

new! Unit Dose




Traxodent®

Hemodent® Paste Retraction System


Saves on chair time and provides predictable results
for your patients' well being.

The #1 retraction paste¹ is now available in
syringe or unit dose, you choose!


Easy to use technique;
patient friendly!

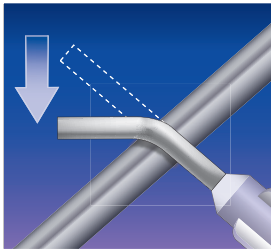


1 Bend tip for optimal access.

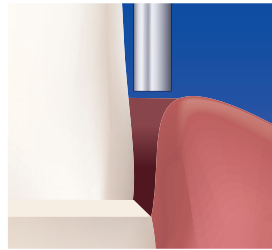


2 Position tip parallel to axial plane of the tooth.

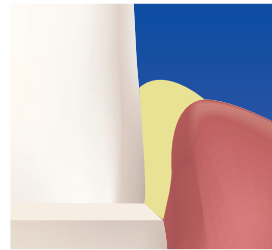


3 Dispense sufficient material to displace tissue.

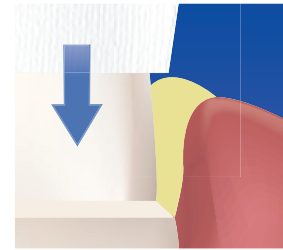

4 Place Premier® Retraction Cap on tooth structure.

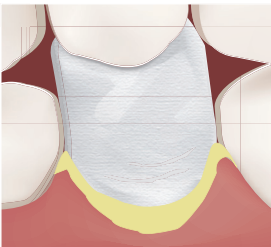


5 Have the patient bite and maintain pressure on the cap. Wait 2 minutes.

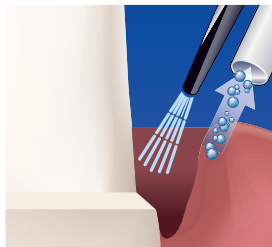


6 Rinse away, leaving sulcus open and accessible.


Optional Cord Technique
Traxodent eliminates the need for a second cord.

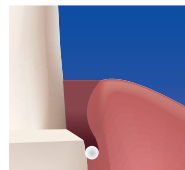

Place a single strand of retraction cord.

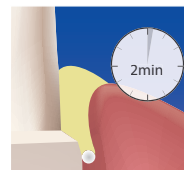

Dispense sufficient material to displace tissue. Wait about 2 minutes.

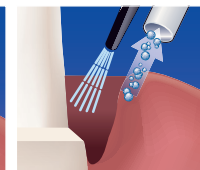

Rinse away, leaving sulcus open and accessible.


"Traxodent works fast, stays put and is easy to use."

- Dean Elledge, DDS, MS Wichita State University, AEGD

"Traxodent can be used in virtually any clinical situation in which control of bleeding is required. When used with a compression cap, it also provides excellent tissue retraction, making it well suited for crown and bridge procedures."

- Richard H. Nagelberg, DDS, PC

Traxodent® from Premier® provides predictable hemostasis and soft-tissue management in minutes.

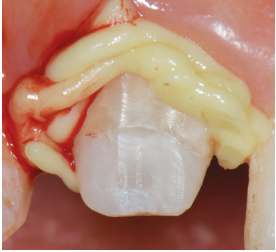

Traxodent applied



Post-op

Photos courtesy of Dr. Yue Weng Cheu, BDS, FRACDS

Easy, effective hemostasis and retraction

The sleek syringe with bendable tip permits easy application of Traxodent directly into the sulcus. After two minutes, it is rinsed, leaving an open, retracted sulcus.

Traxodent is gentle, absorbent and fast.

The soft paste produces gentle pressure on the sulcus while it absorbs excess crevicular fluid and blood. The aluminum chloride creates an astringent effect without irritating or discoloring surrounding tissue. Traxodent provides predictable hemostasis and retraction in less time and with greater comfort.

Traxodent rinses clean without leaving a residue.

Traxodent's strong affinity to aqueous solutions allows the paste to be rinsed away easily and cleanly. There is no residue left behind that can interfere with impression materials and adhesives.

Quick effective retraction

About 2 minutes – no fumbling with hemostatic solutions – save chair time

Superior hemostasis

Stops bleeding and crevicular seepage while retraction occurs - easily absorbs excess crevicular fluid

Increased patient comfort

Avoid tearing gingiva - less pressure on sensitive tissue

Your choice of easy application

Sleek syringe with bendable tips
Ergonomic unit dose dispenser and capsule

Maximum freshness

Each syringe is packaged in resealable foil pouches

"The ability of Traxodent to stop crevicular fluid and bleeding makes it a must-use product during restorative dentistry. When we speak of products that are game-changers, Traxodent is a must-have in anyone's restorative arsenal."

- Ron Kaminer, DDS, Hewlett, NY


View online tutorial at www.premusa.com

Crown and Bridge with Retraction Cap

Using a retraction cap with Traxodent for crown and bridge procedures simplifies moving the paste into the sulcus and provides an atraumatic method for greater retraction. The pressure from the cap quickens and assists hemostasis while the absorbent cotton cap soaks up the fluid in the sulcus. Additionally, the cap quickly brings Traxodent into close contact with the hemorrhaging tissue, which allows the astringent properties to work faster.



Syringe Traxodent and place retraction cap on prep; have patient bite down on cap for 2 minutes.

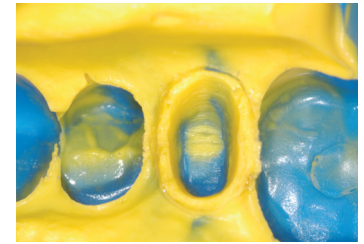

Tooth is prepared for a clean impression.

Photos courtesy of Dean Elledge, DDS, MS Wichita State University, AEGD

Crown and Bridge with Retraction Cord

Traxodent is ideal to replace the second cord in the double cord technique. This technique saves time and lessens the discomfort to the patient while providing greater retraction. Traxodent easily rinses away, eliminating the occurrence of break-through bleeding when removing the second cord.



A single retraction cord is packed around the preps.



Place Traxodent over the cord. Wait 2 minutes.

Photos courtesy of Dean Elledge, DDS, MS Wichita State University, AEGD

Loose Tissue

For difficult tissue that will not stay retracted, Traxodent can be used with retraction cord and cap.



Cord is placed.



Place retraction cap on prep; have patient bite down on cap for 2 minutes.



Impression with well-defined margins

Photos courtesy of William Crinzy, DDS MS

CAD-CAM and Laser Assist Gingival Tissue Procedures

Traxodent is suitable to displace the gingiva before a CAD-CAM scan to get a crisp image without the soft tissue obscuring the view of the margin. Dentists who use lasers for retraction can benefit from Traxodent as it provides excellent post-op hemostasis after laser troughing.



Traxodent is applied around prepared tooth.



After 2 minutes, Traxodent easily rinses away.



Final CAD-CAM restoration.

Photos courtesy of Curtis E. Jansen, DDS

Routine Hemostasis

Traxodent offers a convenient and effective way to stop and control blood and crevicular seepage for all routine hemostasis procedures. The bendable tips can be positioned for areas that are difficult to reach. It is easy to dispense at the site of bleeding and will stay put, giving the astringent properties of the aluminum chloride a chance to stop the bleeding while providing temporary shrinkage of tissue without additional tissue trauma. The absorbent clay base has an affinity towards oral fluids and blood. It will draw in fluids and easily washes away leaving a clean field behind.



Old restoration is removed then Traxodent is placed.



Rinse after 2 minutes. Bleeding is stopped and field is isolated.

Photos courtesy of Richard H. Nagelberg, DDS, PC

Ordering Information

- 9007094 **new!** Traxodent Unit Dose Pack
Contains: 24 capsules
- 9007089 **new!** Unit Dose Syringe Dispenser (3)
- 9007091 Traxodent Value Pack
Contains: 25 syringes and 50 tips
- 9007093 Traxodent Starter Pack
Contains: 7 syringes and 15 tips
- 9007092 Applicator Tips (50)



new! Ergonomic, autoclavable unit dose syringe dispenser

Sleek syringe with bendable tips

Premier® Retraction Caps

Anatomic formed cotton caps work synergistically with Traxodent to quicken and assist retraction and hemostasis.

- Easily guides Traxodent into the sulcus
- Compression assists and quickens hemostasis
- Cotton absorbs fluids
- Anatomic form allows easy placement



- 9048251 Size 1 - Small (120)
- 9048253 Size 3 - Medium (120)
- 9048255 Size 5 - Large (60)
- 9048260 Assorted - (15) #1, (15) #3, (30) #5


Knit-Pak™

Knitted Retraction Cord

Use in combination with Hemodent® or Traxodent® for tissue displacement and hemostasis

- Carries an abundant quantity of hemostatic solution
- Compacts under pressure, making it easier to pack
- Will cut clean and not get entangled in a bur
- Available in 6 color coded sizes, 100% cotton

Each Knit-Pak contains 100 inches (254 cm) of cord.

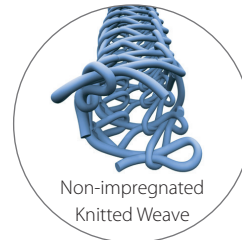

	Size	Color
9007551	000	Green
9007552	00	Brown
9007553	0	Purple
9007554	1	Blue
9007555	2	Orange
9007556	3	Yellow



A stainless steel cutting blade inside the cap automatically cuts the cord upon closing.



The cord piece is held by the cap until removed, preventing it from dropping. Protects cord piece and container from cross-contamination.

For more information on these and additional products, call 610-239-6000.

Order through your authorized dealer.

Please contact your dealer concerning product availability in your country.

premier® Inspired solutions
for daily dentistry™

Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171 • www.premusa.com • E-mail: dentalinfo@premusa.com

1. Strategic Dental Marketing, 2016.



8610410 1117032 Rev13 NKP10M