HELPFUL TIPS TO REMEMBER

- 1 Always keep the tip/toe pointed to the center of the D.I.S.C.
- 2 Always keep the handle (not the shank) of the instrument parallel to the Instrument Reference Lines.
- 3 When sharpening instruments, make sure that only the cutting surface of the working end (as compared to the shank) is touching the stone to avoid weakening the integrity of the shank.
- 4 Avoid re-tipping of instruments, as this can cause stress fractures of the handle.
- 5 No lubrication necessary using the D.I.S.C.
- 6 From time to time, clean the ceramic stones using a mild abrasive pad (without cleaners/chemicals) under running water.

D.I.S.C.™ Scaler & Curette Angles*

Gracey 1-10	G	Jac 34/35	40°	0°
Gracey 11/12	J	H6/H7	F	a
Gracey 13/14	15°	H5/33	10°	
Gracey 15/16	J	U15/33	F	
Gracey 17/18	b (or -10°)	U15/30	15°	
McCalls 13/14	50° -15°	Younger Good 7/8	30°	-5°
McCalls 17/18	H b	Younger Good H-7/8	30°	-5°
McCalls 13S/14S	H b	Neb 128	15°	
McCalls 17S/18S	H b	NV1 Scaler	15°	
Columbia 2L/2R	G a	NV1 Scoop	G**	
Columbia 4L/4R	I b	NV2	65°	-35°
Columbia 13/14	J c	NV4	35°	0°
204S	L d	137 Scaler	15°	
H204S	70° -45°	137 Curette	15°	
H204SD	50° -10°	135	50°	-20°
Barnhart 1/2	50° -20°	Morse 0/00	10°	
Barnhart 5/6	50° -20°	Wil 8	30°	-10°
Langer 1/2	35° 0°	Ball 5/6	50°	-25°
Langer 3/4	40° -20°	UT 107/108	40°	0°
Langer 5/6	20°	S107/S108	70°	-30°
Langer 17/18	40° -20°	GF 21	15°	15°
U15 / Jac 1	F	CK6	80°	-30°
Jac 1/1	F	SIUFW204	40°	-15°
Jac 2/3	K b			

One-Sided Blade

Turn the protractor dial to the appropriate dial setting (Example: "G" for any Gracey 1-10).

Hold the instrument with the handle parallel to the Instrument Reference Lines. Position the working end towards the center.

Keeping the wrist stiff, use a pull-stroke with moderate pressure (repeat 5 or 6 times).

On a curette, roll the instrument as you finish your pull-stroke to avoid sharpening the toe of the instrument.

**There may be slight variations in setting with other manufacturer's instruments. You can fine tune your instrument's settings by following the directions for unlisted blades. **As you pull the disc shaped scoop end across the stone, turn it so that it sharpens around the scoop. At the end of the sweep, you should have turned the scoop 2/3 around. Now, sharpen on other side the same way. You do not want to finish one side at the top of the scoop, or it will start to become a point.*

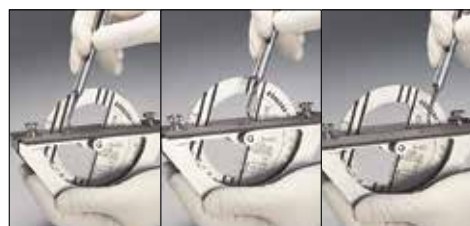
Two-Sided Blade

Turn the protractor dial to the appropriate dial setting for the first cutting surface (Example: "L" for a 204S Scaler).

Hold the instrument with the handle parallel to the Instrument Reference Lines. Position the working end towards the protractor.

Keeping the wrist stiff, use a pull-stroke with moderate pressure (repeat 5 or 6 times).

Change the dial setting for the second cutting surface (Example: "d" for a 204S Scaler). Notice that the Instrument Reference Lines are now going down from left to right.

Move the instrument to the other side of the protractor dial - sharpen the cutting surface with a moderate pull-stroke (keeping the wrist stiff) - notice that the instrument handle will be below the stones.

Unlisted Blade

Identify the cutting surface(s) of the instrument.

Hold the instrument so that the cutting surface is perpendicular to, and above the stone, with the tip/toe pointing towards the protractor.

Lower the instrument handle in a clockwise manner until the stone is in optimal contact with the cutting surface (avoid going too low and making contact with the shank of the instrument). While holding the instrument in this position, turn the protractor dial until the Instrument Reference Lines are parallel to the instrument handle. Sharpen in the usual manner.

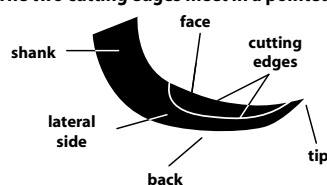
If the instrument has two cutting surfaces, hold the instrument so that the cutting surface is perpendicular to, and below the stone, with the tip/toe pointing towards the protractor. Raise the instrument in a counter-clockwise manner until the stone is in optimal contact with the cutting surface (avoid going too high and making contact with the shank of the instrument). While holding the instrument in this position, turn the protractor dial until the Instrument Reference Lines are parallel to the instrument handle. Sharpen in the usual manner.

Identify the dial setting(s) and record them above.


Parts of an Instrument Blade

SICKLE SCALER

A. The two cutting edges meet in a pointed end.

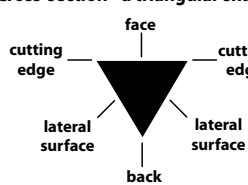
B. Cross-section - a triangular shape.



CURETTE

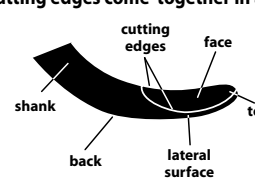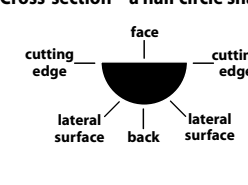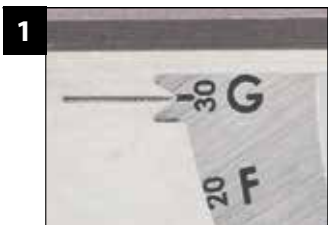A. The cutting edges come together in a spoon shape.

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