PL

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US / UK

Vac Attak™ GREEN
High Performance Evacuation System Cleaner
EPA-Compliant, Powerful Enzymatic Formula, Compatible with Amalgam Separators, Neutral pH, Non-Foaming, Non-Corrosive, Fresh Citrus Scent
Concentrated - Produces 80 Evacuation Cleanings

Directions for Use:
1. Fill evacuation sink or other container with 32 oz. (946 ml) of warm tap water.
2. Remove one scoop (10gm) of cleaning powder from container.
3. Sprinkle powder into water and allow to thoroughly dissolve (30-45 seconds).
4. Flush system with solution.
5. Flush system again with clean tap water after cleaning.

Attention:
Upon initial use, the powerful cleaning action of Vac Attak GREEN may cause accumulated debris to dislodge. Be sure to check the filter/trap after use for maximum efficiency. Highly effective at removing all types of organic debris. Use fresh solution daily.

Caution: May be harmful if swallowed. Eye and skin irritant. Avoid eye and prolonged skin contact.

First Aid:
Eyes: Flush with large volumes of water for 15 minutes. If irritation persists, seek medical attention.
Skin: Flush with large volumes of water.
Ingestion: Drink large volumes of water. Contact physician promptly. Keep out of reach of children.

			
			
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.	MDS& GmbH, Schiffgraben 41		
(888) 670-680 • (610) 239-6000 • premierdentalco.com	30175 Hannover, Germany		
Made in U.S.A. Read SDS before using product. For SDS, visit premierdentalco.com or call Premier at 610-239-6000.	DFU#91115 1118066 Rev0 DCP		

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