

Cleaning and Sterilization Instructions

Carbopost[®], Snowpost[®], and Snowlight[®] (Cure-Thru[®] IntegraPost[®]) Dental Posts



8400 Green Meadows Dr. N.
P.O. Box 545
Lewis Center, OH 43035 USA

USA: 800.964.8324
P: 740.548.4100 F: 740.548.7617

Abrasive Technology Ltd.
Unit 11 Boundary Business Court
92-94 Church Road
Mitcham, Surrey CR4 3TD

P: +44.20.7471.0200
F: +44.20.7471.0202

These devices are NOT supplied sterile. Clean and Sterilize prior to use, as follows.

Use only FDA-cleared sterilizers, and sterilization accessories, per the equipment or accessory manufacturer's instruction.

CLEANING

1. Wipe the post using cotton soaked in ethyl or isopropyl alcohol, minimum 70%.
2. Dry the post with clean, pressurized air that is free of oil and water.

EXAMINATION

Visually examine the devices for contamination. Magnification is recommended.
Repeat the above steps for any devices that are visibly contaminated.

STERILIZATION

Steam sterilize in a sterilization pouch - vacuum, average pressure of 30psi (207 kPa), minimum of 4 minutes, 132°C (270°F), and a minimum dry time of 20 minutes.

STORAGE/PACKAGING

Store the devices in the sterilized pouch.