

The reliable temporary cement.
Now that's peace of mind!



new!
NexTemp
Clear




NexTemp® Temporary Resin Cement

NexTemp® is the reliable temporary cement with firm retention that lasts between visits yet can easily be removed when the time comes. NexTemp is the only eugenol-free resin cement with a unique potassium nitrate, chlorhexidine and fluoride formula, which studies show may help prevent post-op sensitivity.¹

NexTemp creates an excellent seal around margins so there's no need to worry about microleakage or a wash-out while your patients await their final restoration! Affordably priced vs. the leading brand, it's good for your bottom line too!

Keep your patients happy and improve your daily work flow with the temporary cement that delivers predictable results. Every time.

Available in original opaque and **new** clear shade!

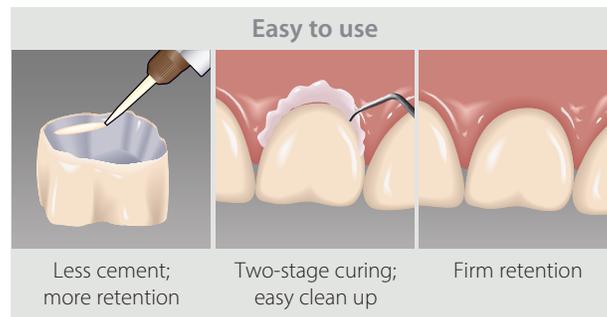

* Total oral set time in 4-5 minutes


premier® Inspired solutions
for daily dentistry®

Premier® Dental Products Company • 888-670-6100 • premierdentalco.com

NexTemp® is easy to use.




Dry and isolate the prepared tooth. Do not desiccate.

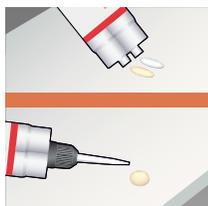


Bleed the syringe. Mount a new automix dispensing tip and bleed the syringe again.



Dispense a **small bead** of cement around the **inside margin**.

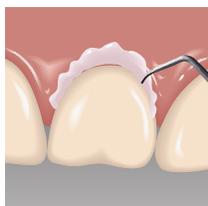

Seat the temporary and hold in place. In approximately 2 minutes, excess cement reaches a distinct gel phase and can be removed safely.

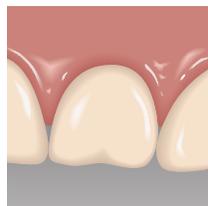

Final set occurs between 4-5 minutes after seating of temporary restoration.*



Use a PerioWise® Periodontal Probe for post-treatment periodontal evaluation.

* Final set time is determined intraorally. Ambient set times vary and are not representative of clinical conditions.

Ordering Information

3001471 NexTemp Standard Kit - Shade: Opaque
Contains: One 5ml Automix Syringe, 10 Mix Tips

3001470 NexTemp Bulk Kit - Shade: Opaque
Contains: Four 5ml Automix Syringes

3001451 Syringe Tips
Contains: 25 Automix Syringe Tips

3001472 **new!** NexTemp Standard Kit - Shade: Clear
Contains: One 5ml Automix Syringe, 10 Mix Tips

3001473 **new!** NexTemp Bulk Kit - Shade: Clear
Contains: Four 5ml Automix Syringes

NexTemp® Clear is a new shade ideal for temporary cementation in the aesthetic zone. It gives you the same reliable NexTemp performance but without the pigment.

Flashing Crown Remover



Flashing Crown Remover • 1003576

As recommended by Dr. Donald Flashing



Flashing Crown Remover

Ideal for removing provisionally cemented crowns without damaging margins and is less traumatic to the patient.

premier® Inspired solutions for daily dentistry®



Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171 • premierdentalco.com • info@premierdentalco.com

1. Jalalian E, Meraji N, Miraei M. A comparison of the efficacy of potassium nitrate and Gluma desensitizer in the reduction of hypersensitivity in teeth with full-crown preparations. J Contemp Dent Pract. 2009 Jan 1;10(1):66-73.
©2020 Premier Dental Co. | All Rights Reserved