Enamelon®

Preventive Treatment Gel

Intended Use: Premier® Enamelon® Preventive Treatment Gel is intended for at home use or by a dental professional as a Preventive treatment to reduce sensitivity, sensitivity, promote remineralization, inhibit demineralization, and soothe and moisturize dry-mouth issues.

Indications for Use: Enamelon Preventive Treatment Gel is indicated to provide a topical coating for high risk caries patients suffering from erosion or xerostomia or patients with special needs such as hypersensitivity or dry mouth.

Description of Device and Contents: Premier® Enamelon® Preventive Treatment Gel is a clear gel available in a clean mint flavor containing fluoride and ACP to be used after teeth cleaning or polishing.

Contraindications:

- Do not use if gingivitis, bleeding, or redness persists for more than 2 weeks.
- Do not use if you have known periodontal disease or symptoms of periodontitis.
- Do not use this product for sensitivity relief for longer than 4 weeks unless recommended by a dentist or doctor.

PRECAUTIONS:

- Supervise children as necessary until capable of using without supervision
- Children under 6 years of age, consult a dentist or doctor prior to using
- Do not eat or drink for 30 minutes following application

WARNINGS:

- Keep out of reach of children under 6 years of age.
- If more than use for brushing is accidentally swallowed, get medical help or contact poison control center right away.
- See your dentist immediately if you have painful or swollen gums, pus from the gum line, loose teeth, or increasing spacing between the teeth.

Adverse Reactions: Choking or poisoning effects may occur from swallowing excess amounts of Enamelon Preventive Treatment Gel during use.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Contents and Setup: Available in 4.0oz (113g) tube

 9007295
Enamelon Preventive Treatment Gel

Materials: Contains 0.40% stannous fluoride

Directions for Use (In-Office):

- Extrude a strip of Enamelon Treatment Gel into the tray or directly apply to the teeth.
- Apply the treatment gel to teeth and let sit for a minimum of 1 minute.
- Ask the patient to expectorate thoroughly and if possible avoid rinsing.
- Any gel remaining on the surface can be left to gradually dissipate.

Directions for Use (At Home): Adults and children 6 years of age and older: use once a day, preferably prior to bedtime, to maximize contact time. Supervise Children as necessary until capable of using without supervision.

Apply the gel to teeth and brush thoroughly for at least 1 minute then spit out. Do not swallow. Do not rinse, eat or drink for as long as possible for maximum benefit – up to 30 minutes after brushing.

Storage: Close cap after each use. Store in temperature conditions of 8°C (46°F) to 25°C (77°F) and away from direct sunlight.

Shelf Life: Shelf life of the product is two years from the date of manufacturing and labeled on the product.

For Further Information: To obtain SDS or a printable version of the IFU, visit www.premierdentalco.com

Symbols	Symbol Meaning
	Temperature Limit
	Keep away from sunlight
	Warning, See IFU
	SKU
	Manufacturer
	Medical Device

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