2pro®

Total Access Disposable Prophy Angle

Disposable Prophy Angle

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US / UK

2pro® Total Access Disposable Prophy Angle

Description:
2pro® is a single-use disposable prophy angle uniquely designed with a prophy cup and Total Access Tip for cleaning and polishing teeth during a dental hygiene appointment. 2pro's prophy cup is ideal for cleaning and polishing facial and lingual tooth surfaces. Remove the prophy cup and use the Total Access Tip to clean and polish gingival and interproximal areas, veneer margins and around implants or orthodontic brackets. 2pro's Total Access Tip can be used to clean and polish occlusal surfaces. 2pro can be used with Enamel Pro® Prophy Paste or any other prophy paste. 2pro is latex free and fits a nose-cone low-speed handpiece.

Intended Use:
A single-use disposable prophy angle uniquely designed to operate a 2-in-1 technique with a prophy cup and Total Access Tip for cleaning and polishing teeth. Dental Use Only.

Indications for Use:
The prophy cup can be removed and the Total Access Tip can be used to clean and polish gingival and interproximal areas, veneer margins, and around implants or orthodontic brackets.

Contraindications:
Do not use 2Pro with high-speed hand pieces (3000+ RPM)

PRECAUTIONS:

- Product is designed as a single-use, disposable device.
- Do not sterilize or re-use.
- Ensure the cup and angle is securely attached, prior to use.

Rx only, Prescription device.

WARNINGS:

- Product is designed for use on a low piece handpiece. Do Not operate 2Pro above 3000 RPM.
- Do Not ReAttach the cup after removing it from the hub of your 2Pro Disposable Prophy Angle's total access tip.

ADVERSE EVENTS:

- Choking may occur from the detachment and swallowing of the disposable prophy angle or cup.
- If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Operating Instructions:

1. Remove 2pro Total Access Disposable Prophy Angle from packaging.
2. Insert the shaft of 2pro into the nose-cone of your low-speed handpiece. Align the slot of the prophy angle with the guide pin on your handpiece and lock into place.
3. Safely operate 2pro for a few seconds outside the mouth to ensure that it is attached securely to the handpiece.
4. To polish facial and lingual tooth surfaces, use the cup on your 2pro Total Access Disposable Prophy Angle. (See Image 1)
5. To clean and polish gingival and interproximal areas, veneer margins and around implants or orthodontic brackets, remove the prophy cup and use the Total Access Tip. (See Image 2, 3, 4 & 5)
6. To clean and polish occlusal surfaces, use the cup or Total Access Tip on your 2pro Total Access Disposable Prophy Angle.
7. Do not operate 2pro above 3,000 rpm.
8. **Do not reattach the cup after removing it from the hub of your 2pro Total Access Disposable Prophy Angle.**
9. 2pro is designed for single-patient use. Do not sterilize or reuse.
10. Dispose of 2pro properly after each patient use.

REF	2pro® with Total Access Tip - Pkg 144	REF	2pro® with Total Access Tip - Pkg 1,000
5500101	2pro® Soft/Short Cup (Purple Cup)	5510001	2pro® Soft/Short Cup (Purple Cup)
5500102	2pro® Firm/Short Cup (Blue Cup)	5510002	2pro® Firm/Short Cup (Blue Cup)
5500103	2pro® Soft/Long Cup (White Cup)	5510003	2pro® Soft/Long Cup (White Cup)
5500104	2pro® Firm/Long Cup (Green Cup)	5510004	2pro® Firm/Long Cup (Green Cup)

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