



Vac Attak™ GREEN & Vacu Blast™

Enzymatic Evacuation System Cleaners



premier Inspired solutions for daily dentistry®



Manufacturer:
Premier® Dental Products Company
1710 Romano Drive,
Plymouth Meeting, PA 19462 U.S.A.
888-670-6100 • 610-239-6000
premierdentalco.com • Made in U.S.A.



MDSS GmbH
Schiffgraben 41
30172 Hannover, Germany



premierdentalco.com/promise

Intended Use:

Vac Attak™ GREEN and Vacu Blast™ are intended to be high-performance evacuation system enzymatic cleaners that are safe to be used with amalgam separators.

Device Description:

Premier® Vac Attak™ GREEN and Vacu Blast are high-performance evacuation system enzymatic cleaners used in dental practices. Vac Attak™ GREEN comes in a powdered form where Vacu Blast is within a liquid form.

Contraindications:

Do not use Vac Attak™ GREEN/ Vacu Blast™ under the following conditions:

- If the container or cap is found broken/damaged
- Should not be used on anodized aluminum instruments

PRECAUTIONS:

- Do not mix with other detergents or contaminants
- Avoid over dilution.
- Keep product stored only in original container.
- Keep containers tightly closed when not in use.
- Do not place a wet scoop into container.
- Wash exposed skin and hands thoroughly after handling.

WARNINGS:

- Remove contact lenses, if exposed.
- EYES – Rinse with large volumes of water for 15 minutes if exposed. If irritation persists, seek medical attention.
- SKIN – Rinse with large volumes of water.
- INGESTION – Drink large volumes of water. Contact physician promptly.

Adverse Events:

- IF SWALLOWED –
 - » Harmful, may cause nausea and vomiting.
 - » May be irritating to mouth and throat.
 - » May cause allergic type reaction.
- May cause skin irritation, eye damage and allergic skin reactions.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Contents and Setup:

Item No.	Product Specifications	Item No.	Product Specifications
9011995	Vac Attak GREEN SAMPLE (1pk)	9011107	Vacu Blast (Case of 12)
9011105	Vac Attak GREEN (CASE OF 6)	9011109	Vacu Blast (Single Bottle)
8101105	Vac Attak GREEN Single Jar	8101107	Vacu Blast Sample

Materials:

Vac Attak Green – EPA complaint Powerful Enzymatic formula and has fresh citrus scent.

Vacu Blast – Phosphate and Chlorine free, Powerful Enzymatic Cleaner and has eucalyptus scent.

DIRECTIONS FOR USE:

Vac Attak™ Green	Vacu Blast™
<ol style="list-style-type: none"> 1. Fill Evacuation sink or other container with 32oz. (946ml) of warm tap water. 2. Remove one scoop (8gm) of cleaning powder from container. 3. Sprinkle powder into water and allow 30-45 seconds to thoroughly dissolve. 4. Flush system with solution. 5. Flush system again with tap water after cleaning. <p>Attention:</p> <ul style="list-style-type: none"> • Upon initial use, the powerful cleaning action of Vac Attak GREEN may cause accumulated debris to dislodge. Be sure to check the filter/trap after use for maximum efficiency. • Highly effective at removing all types of organic debris. • Use fresh solution daily. 	<ol style="list-style-type: none"> 1. Loosen cap on metered-dose chamber (DO NOT REMOVE). 2. Squeeze from middle of bottle until liquid reaches 1/2oz. mark. 3. Remove cap on metered-dose chamber and pour contents into 32oz. (946ml) of water. 4. Stir thoroughly before use. <ul style="list-style-type: none"> • For larger offices, Squeeze from middle of bottle until liquid reaches 1oz. mark. • Recommended to dilute 1oz. of product (Vacu Blast) with 64oz. (1892ml) of water.

Storage:

Store in a cool, dry place away from direct sunlight.

Shelf Life:








Vac Attak™ Green does not have a shelf life period. Vacu Blast™ has a shelf life of three (3) years from the date of manufacturing.

Disposal:

After use, Dispose as biohazardous medical waste in accordance to facility or institutional guidelines/procedures.

For Further Information:

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbols	Symbol Meaning
	Manufacturer
	Batch number
	Warning, See IFU
	Store in dry cool place
	Avoid direct sunlight
	Medical Device
	UKCA