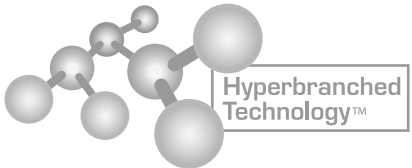
CompCore™ AF Stack

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US / UK

CompCore™ AF Stack

Dual-Cure Composite Core Build-Up Material

Hyperbranched Technology™ • Fluoride releasing • Radiopaque

Intended Use:
CompCore™ AF is an automix, fluoride releasing, resin composite material designed for the direct fabrication of core build-ups.

Product Description:
Premier® CompCore™ AF is a dual cured, fluoride releasing, radiopaque composite core build-up material for use in vital and non-vital teeth. The device is available in a syringe or a dual-barrel cartridge.

Contents:
Premier CompCore AF is available in various kit configurations to match the clinical preferences:

SKU	Product Specifications
3001512 Shade White	SyringeMix STACK: Dual-Cure/Automix
3001513 Shade A3	• 2 – 9g automix syringes • 15 – mix-tips (Single use)
3001514 Shade Blue	• 15 – standard intraoral tips • 10 – extra-fine intraoral tips
3001432 Shade White	Cartridge Refills • 1 – 50g cartridge
3001433 Shade A3	• 25 – mix-tips • 25 – intraoral tips
3001418	SyringeMix Mix-Tips & Intraoral Tips (35)
3001468	SyringeMix Extra-Fine Intraoral Tips (50)
3001417	Cartridge Mix-Tips & Intraoral Tips (35)

Materials:
Composite Core Build-Up material formulation is a unique combination of methacrylate resins, and fillers.

Physical Properties	
Compressive Strength	250 MPa
Radiopacity	400% AI

Contraindications:

- Do not use on patients who have a history of known sensitivity or severe allergic reaction to methacrylate resins or any of the components.
- Do not place CompCore AF, if a dry field is not possible.

Precautions:

- Disposable tips are designed for single patient use only. Do not re-use a mix-tip.
- Working & setting times will vary based on temperature, humidity, and product age and storage conditions.
- Use a protective plastic sheath barrier on the syringe to mitigate potential cross-contamination between patients.
- Avoid contamination of product. Do not mix components with those of other materials. Do not return dispensed components to their original container. Dispose of leftover material properly.
- Avoid sunlight or nearby heat sources. Shelf life will be reduced if not stored properly.

- To prevent lack of setting, ensure all eugenol-containing materials are removed prior to placing CompCore AF.
- Ensure material is at room temperature prior to use to prevent longer cure-time.
- Prescription device, Rx only.

WARNINGS:

- For use by a trained dental professional only.
- Do not use after the shelf-life period indicated on product labeling/packaging.
- Large amounts of excess material may generate heat as part of the normal setting reaction.

Adverse Reactions:
Adverse reactions may include but are not limited to:

- Inflammation, irritation, or damage of soft and hard tissue.
- Allergic reactions to CompCore AF material. Wash exposed area completely to remove material. If an allergic reaction is observed, consult a physician immediately and discontinue use with this patient.
- Pain, discomfort, and/or abnormal sensations due to the presence of CompCore AF.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Directions for Use *(Please follow carefully.)*

- Apply a bonding agent system, such as Premier's IntegraBond. Follow manufacturer's instructions for use with dual-cure composites.
- Select CompCore AF shade and assemble dispenser (cartridge or syringe). Graphic assembly instructions for both the gun and cartridge unit or syringe are enclosed separately with this package. Bleed cartridge or syringe on initial use and bleed through assembled mix-tip and intraoral tip before each use.
- Place intraoral tip into the base of the prepared tooth and dispense an adequate amount of material to complete build-up. Let material rise around the tip while withdrawing to eliminate air bubbles. Cure-Thru® matrix bands can be used to confine the material and define the build-up.

Light-Cure Directions:

- Place the curing-light (Halogen or LED) close to the surface of the core build-up and polymerize for at least 30 seconds. If the core has a depth greater than 8mm, a minimum of 40 seconds is recommended.
- Check for complete hardness. The preparation can be finished immediately with a Two Stripper® diamond or other bur.

Chemical-Cure Directions:

- Do not disturb CompCore AF after the placement or during the gelation stage. Chemical-cure intraoral set-time is approximately 4.5 minutes from start of mix/application. Set time takes longer outside the mouth. Optionally, light-cure for 20 seconds after application to stabilize the core material and to accelerate chemical-cure process.
- Check for complete hardness. Finish preparation with a Two Stripper® diamond or other bur.

Light-cure setting times			
Working time:		2.5 minutes at 72°F (22°C).	
Final Set Time		Shades	
Depth of Cure	A3	White	Blue
6mm	30 sec	30 sec	30 sec
8mm	30 sec	40 sec	60 sec
Chemical-cure setting times			

Working time	2.5 minutes at 72°F (22°C)
Final set	4.5 minutes at 98.6°F (37°C)

Storage:

- CompCore AF can be stored in a temperature range 4°C(39°F) - 24°C(75°F).
- Do not store at temperature over 24°C (75°F).
- Do not freeze.

Shelf Life:

- CompCore AF has a shelf life of 2 years (24 months) from the date of manufacturing.
- Do not use after product has expired. See expiry date on the product label.

For Further Information:
To obtain an SDS visit www.premierdentalco.com or call Premier at 610-239-6000.

Symbols:
A full explanation of symbols used on Premier Dental Packaging and literature is located at premierdentalco.com/symbol-glossary

Symbols	Symbol Meaning
	Expiry Date
	Date and Country of Manufacture
	Manufacturer
	Batch number
	Caution: See instructions for use
	The upper and lower limits of temperature of use, storage and transportation
	Irritant (Skin and Eye) Skin Sensitizer
	Keep away from sunlight
	See IFU on premierdentalco.com
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Catalog Number
	Do not use if package is damaged

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