## ZR-Cem® Self-Adhesive Universal Resin Cement

### Instructions For Use

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Premier® Dental Products Company  
1710 Romano Drive  
Plymouth Meeting, PA 19462 U.S.A.  
(888) 670-6100 • (610) 239-6000  
premierdentalco.com

Made in Israel.  
DFU# 31560 • 0423079 Rev1 BJM

ZR-Cem® Self-Adhesive Universal Resin Cement	US / UK <b>INSTRUCTIONS FOR USE</b>
--	--

#### Intended Use

ZR-Cem is a prescription only, dual cured, radiopaque resin cement intended to be used for cementation of porcelain, ceramic, resin, metal-based inlays/onlays, crowns, bridges, core build up material, posts, and veneers. It is also used for cementation of crown restorations to implants.

#### Product Description:

ZR-Cem is a self-etch, dual cure, adhesive resin cement that is specially formulated for zirconia cementation; however, it is also indicated for definitive cementation of all ceramic, composite and metal restorations.

Examples of Restorative Substrates to be used with ZR-Cem	
<b>Ceramic Materials</b>	<ul style="list-style-type: none"><li>• Zirconia (low and high translucency)</li><li>• Alumina</li><li>• Lithium Disilicate</li><li>• Zirconia-reinforced Lithium Silicate</li><li>• Feldspathic and Leucite-reinforced</li><li>• Ceramics</li></ul>
<b>CAD/CAM milled</b>	<ul style="list-style-type: none"><li>• Composites</li><li>• Ceramics</li></ul>
<b>Metal-based</b>	<ul style="list-style-type: none"><li>• Crown and Bridge Alloys</li><li>• PFMs</li></ul>

#### Contents:

Premier ZR-Cem is available in various kit configurations to match the clinical preferences:

SKU	Description
3001560 Shade White	ZR-Cem Standard Pack <ul style="list-style-type: none"><li>• 1 – 8g Syringe</li><li>• 10 – Mix-tips</li><li>• 10 – Intraoral tips</li></ul>
4001561 Shade A2	
3001562 Shade Translucent	
3001418	ZR-Cem Refill Tips <ul style="list-style-type: none"><li>• 35 – Mix-tips</li><li>• 35 – Intraoral tips</li></ul>

#### Materials:

Non-Eugenol. Non-fluoride adhesive resin formula.

#### Contraindications:

- Do not use on patients who have a history of known sensitivity or severe allergic reaction to methacrylate resins or any of the components.

#### PRECAUTIONS:

- Disposable tips are designed for single patient use only. Do not re-use a mix-tip.
- Working & setting times will vary based on temperature, humidity, and product age and storage conditions.
- Use a protective plastic sheath barrier on the syringe to mitigate potential cross-contamination between patients.
- Avoid contamination of product. Do not mix components with those of other materials. Do not return dispensed components to their original container. Dispose of leftover material properly.
- Avoid sunlight or nearby heat sources. Shelf life will be reduced if not stored properly.
- Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact.
- Prescription device, Rx only.

#### WARNINGS:

- For use by a trained dental professional only.
- Do not use after the shelf-life period indicated on product labeling/packaging.
- Large amounts of excess material may generate heat as part of the normal setting reaction.

#### Adverse Reactions:

Adverse reactions may include but are not limited to:

- Inflammation, irritation, or damage of soft and hard tissue.
- Allergic reactions to ZR-Cem material. Wash exposed area completed to remove material. If an allergic reaction is observed, consult a physician immediately and discontinue use with this patient.
- Pain, discomfort, and/or abnormal sensations due to the presence of ZR-Cem.
- Uncured methacrylate resin may cause contact dermatitis and damage the pulp.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

#### Dual-Cure Physical Properties:

##### Chemical-cure

##### 72°F (22°C) temperature

Working time: 1.5 – 2.5 minutes.  
Higher temperatures accelerate set.

##### 98.6° F (37° C) temperature

Gel phase: approximately 1-1.5 minutes  
Final set: 3.0 – 4.0 minutes.  
Higher temperatures accelerate set.

##### Light-cure setting times:

20 seconds with light intensities of = or > 500 mW/cm<sup>2</sup>  
30 seconds with light intensities of < 500 mW/cm<sup>2</sup>

#### Directions for use: *(Please follow carefully)*

##### 1. Pretreatment of Tooth

- 1.1 Remove the provisional restoration and all temporary cement residue. **Note:** Use only temporary cements that are eugenol- free. The use of eugenol-containing temporary luting materials can inhibit the polymerization process of ZR-Cem.
- 1.2 Mechanically clean prepared tooth (e.g. with pumice paste).
- 1.3 Rinse preparation thoroughly and air dry. Do not desiccate - over drying can lead to post-operative sensitivity.
  - **Note:** Tooth preparation should leave the tooth surface evenly moist. Dry preparation (air-dried or alcohol dried), and wet preparations (with pooled surface water), can reduce adhesion.
  - **Note:** The adjacent teeth and/or the external surfaces of the restoration may be lubricated with a water soluble medium to ease cleanup of excess cement.
- 1.4 Try in the restoration to ensure proper fit.

##### 2. Prepare Restoration

- 2.1 Verify that the laboratory has pre-treated the final restoration in accordance with the manufacturer's instructions.
- 2.2 Clean intaglio surface of restoration with alcohol (or per restoration manufacturer's instructions), then dry well with oil-free air.

##### 3. Apply ZR-Cem Cement

- 3.1 Select desired shade of ZR-Cem cement.
- 3.2 Bleed cartridge before initial use and place mixing tip on Auto-Mix syringe.
- 3.3 Mount the supplied intra oral tip onto the mixing tip. Bleed again through the tips.
- 3.4 Dispense the cement:

<b>Inlay/Onlay</b>	Dispense the cement directly into the cavity preparation covering all surfaces.
<b>Crown/Bridge</b>	Dispense the cement directly into the gingival 3rd of the restoration
<b>Post</b>	Prepare post space in the canal. Dispense cement into the post space while keeping tip immersed in the cement to prevent air entrapment. Fill the post space with ZR-Cem.

##### 4. Seat restoration

<b>Inlay/Onlay</b>	Seat the restoration gently onto the preparation allowing the cement to flow from all sides.
<b>Crown/Bridge</b>	Seat the restoration gently onto the preparation allowing the cement to flow from all margins.
<b>Post</b>	Seat post into post space allowing excess cement to vent from canal space. Stabilize the post by light-curing coronal portion of the post for 2-3 seconds.

##### 5. Clean up Marginal Excess

5.1 After the restoration is properly seated, remove all excess cement. Excess cement is best removed in its gel state with a scaler or explorer.

<b>5.1.a Self-Cure</b>	Excess cement will reach the gelled state in approximately 1 – 1.5 min after application. <ul style="list-style-type: none"><li>• Allow cement to self-cure for 1 – 1.5 min, then remove the excess cement.</li></ul>
<b>5.1.b Light Cure</b>	Under light curing, excess cement will reach the gelled state in a few seconds. <ul style="list-style-type: none"><li>• Within the first minute following intraoral insertion light-cure excess cement at the margin by moving light tip around the buccal and lingual margin for 2-3 seconds.</li></ul>

5.2 Remove excess cement. Note: Cement within the crown or canal has not yet set. Do not move or disturb the crown or post during cleanup.

##### 6. Final Cure

- 6.1 For high-translucency zirconia and other light transmittable restorations (material thickness ≤ 2.5mm), light cure all surfaces including margins for 20 seconds.
- 6.2 For non-light transmittable restorations or light transmissible restorations with material thickness > 2.5mm, allow ZR-Cem to self-cure for 4 minutes from start of mix.
- 6.3 Finish and polish as needed. Use finishing strips for proximal areas. Polish margins with polishing discs, cups, or points.
- 6.4 Check occlusion.

**Note:** Premier Universal Primer can be used to help achieve maximum bond strength for cases that require additional retention (e.g., short and/or over-tapered preparations).

#### Storage:

- Do not store at temperature over 24°C (75°F).
- Keep away from direct sunlight. Store in a cool, dry place.
- If refrigerated, allow the syringe to reach room temperature before use.
- Do not freeze.

#### Shelf Life:

- ZR-Cem has a shelf life of 2 years (24 months) from the date of manufacturing.
- Do not use after product has expired. See expiry date on the product label.

#### For Further Information:

To obtain an SDS visit [www.premierdentalco.com](http://www.premierdentalco.com) or call Premier at 610-239-6000.

#### Symbols:

A full explanation of symbols used on Premier Dental Packaging and literature is located at [premierdentalco.com/symbol-glossary](http://premierdentalco.com/symbol-glossary)

Symbols	Symbol Meaning
	Expiry Date
	Date and Country of Manufacture
	Manufacturer
	Batch number
	Caution: See instructions for use
	The upper and lower limits of temperature of use, storage and transportation
	Irritant (Skin and Eye) Skin Sensitizer
	Keep away from sunlight
	See IFU on <a href="http://premierdentalco.com">premierdentalco.com</a>
<b>Rx ONLY</b>	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Catalog Number
	Do not use if package is damaged

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