ProFlare™

Articulating Disposable Prophylaxis Angle

ProFlare™

Articulating Disposable Prophylaxis Angle

US / UK

INSTRUCTIONS FOR USE

Description:

ProFlare is an articulating disposable prophylaxis angle. Single-use and disposable.

Intended Use:

Premier's ProFlare is designed to aid in cleaning and polishing the surface of teeth. Professional dental use only.

Contents:

ProFlare is individually packaged.

Catalogue Number	Contents
5500150	Package of 125 ProFlare angles

Contraindications:

Do not use ProFlare with high-speed hand pieces (3000+ RPM).

PRECAUTIONS:

Product is designed as a single-use, disposable device.

Do not sterilize, disinfect or re-use.

Ensure the cup is securely attached to the angle prior to use.

WARNINGS:

ProFlare is designed for use on a low speed handpiece. Do not operate ProFlare above 3000 RPM.

Choking may occur from the detachment and swallowing of the disposable prophylaxis angle or cup. Do not use the device if any of the components are loose or have become detached.

Adverse Events:

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Materials:

Cup and body are Latex-Free.

Directions for Use:

1. Remove ProFlare Articulating Disposable Prophylaxis Angle from packaging.
2. Insert the shaft of ProFlare into the nose-cone of your low-speed handpiece. Align the slot of the prophylaxis angle with the guide pin on your handpiece and lock into place.
3. Safely operate ProFlare for a few seconds outside the mouth to ensure that it is attached securely to the handpiece.
4. To polish tooth surfaces, use the cup of the ProFlare.
5. Do not operate ProFlare above 3,000 rpm.
6. ProFlare is designed for single-patient use. Do not sterilize, disinfect or reuse.
7. Dispose of ProFlare properly after each patient use.

Storage:

Always store in the original protective packaging.

Store in a dry environment and protected against contamination.

Disposal:

Dispose of the disposable prophylaxis angles after each patient according to CDC Guidelines for Infectious Waste and federal, state and local regulations.

For Further Information:

To obtain IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbols:

A full explanation of symbols used on Premier Dental Packaging and literature is located at premierdentalco.com/symbol-glossary

Symbols	Symbol Meaning
	Manufacturer
	Batch number
	Catalogue number
	Date and Country of Manufacture
	Single-use only
	Latex free
	Caution: See instructions for use
	See IFU on premierdentalco.com

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