# Hemodent® Buffered Hemostatic Solution

US

## Buffered Hemostatic Solution

**Intended Use:** Hemodent® has been specifically formulated to provide gingival retraction and hemostasis. It is intended for use in restorative and operative dentistry, optionally with gingival retraction cord, prior to taking an impression, cementation, cavity preparation or whenever hemostasis and retraction is required.

**Device Description:** Premier® Hemodent liquid is a buffered aluminum chloride solution, without epinephrine, that is used to suppress bleeding during crown preparations, before taking impressions, and cementation or bonding procedures.

### Contents:

| Base Unit - Hemodent |                        |
|----------------------|------------------------|
| Item No.             | Product Specifications |
| 9007071              | Hemodent – 10cc bottle |
| 9007072              | Hemodent – 20cc bottle |
| 9007073              | Hemodent – 40cc bottle |

| Available Accessories (Optional)       |          |         |        |
|----------------------------------------|----------|---------|--------|
| Knit-Pak™ Non-impregnated knitted cord |          |         |        |
| 9007551                                | Size 000 | 9007554 | Size 1 |
| 9007552                                | Size 00  | 9007555 | Size 2 |
| 9007553                                | Size 0   | 9007556 | Size 3 |

| Premier Retraction Caps |                                  |
|-------------------------|----------------------------------|
| 9048251                 | Premier Retraction Caps Size 1   |
| 9048253                 | Premier Retraction Caps Size 3   |
| 9048255                 | Premier Retraction Caps Size 5   |
| 9048260                 | Assorted Premier Retraction Caps |

### Contraindications:

Do not use Hemodent in the following conditions:

- Do not use on persons with known allergy to aluminum chloride
- Patients with significant periodontal disease or furcation involvement.
- An intraoral site that has active infection or inflammation

### PRECAUTIONS:

- Do not use if the bottle seal is damaged/broken prior to use.
- Keep product stored in original packaging. Seal after use
- Be sure to rinse treated area before taking impression to avoid inhibition of polymerization of impression material
- Ensure all instruments or applicators that come into contact shall be sterile, reuse or contact of a re-used applicator can expose patients and staff to cross-contamination
- Rx only, Prescription device.

### WARNINGS:

- Ensure all aluminium chloride is remove by rinsing prior to dental impression to avoid inhibition of polymerization of impression material
- Do not use on persons with known allergies to Aluminum Chloride
- Aluminum Chloride is irritating to skin and eyes.

### Adverse Reactions:

Adverse reactions may include but are not limited to:

- Recession of the gums from prolonged use or excessive force in sulcus
- Additional procedures may be necessary if impression material was inhibited due to residual Aluminum chloride.
- Allergic type of reactions such as swelling, inflammation, irritation, and pain could occur in persons with known allergy to Aluminum chloride.
- Infection within the gums is possible if applicators are re-used within the same solution.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

### Materials:

- 21.3% Buffered Aluminum Chloride Epinephrine-free Hemostatic Liquid

### Directions for Use:

- Pour appropriate amount of Hemodent into dappen dish.
- Soak applicator within the dappen dish and apply to area needing hemostasis.
- Apply to bleeding area with pressure until hemorrhaging is arrested. Should a small dark coagulum appear, flush it away with water. This will have no effect on hemostatic action.
- When hemostasis has been achieved, remove applicator containing Hemodent and thoroughly rinse area using an air/ water spray to clean and dry preparation.
- Check for hemostasis. If bleeding occurs, repeat steps 2-4. Recheck with air/ water spray until bleeding has stopped.

**A full explanation of symbols used on Premier Dental packaging and literature is located at [premierdentalco.com/symbol-glossary](http://premierdentalco.com/symbol-glossary).**

| Symbols        | Symbol Meaning                                                                             |
|----------------|--------------------------------------------------------------------------------------------|
|                | Expiry date                                                                                |
|                | Date and country of manufacture                                                            |
|                | Manufacturer                                                                               |
|                | Batch number                                                                               |
|                | Indicates the upper limit of temperature to which the device can be safely exposed.        |
|                | Irritant                                                                                   |
|                | Caution: See instructions for use                                                          |
| <b>Rx ONLY</b> | “CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician |
|                | Consult IFU                                                                                |
|                | Medical Device                                                                             |
|                | Do Not Reuse                                                                               |
|                | Catalog number                                                                             |
|                | European Union authorized representative                                                   |
|                | Swiss Union authorized representative                                                      |

### Storage:

- Do not store at temperature over 28°C (82°F).
- Do not freeze.
- Store in a dry place

### Shelf Life:

- Hemodent has a shelf life of 3 years (36 months) from the date of manufacturing.
- Do not use after product has expired. See expiry date on the product label.

**For Further Information:** To obtain SDS or IFU, visit [www.premierdentalco.com](http://www.premierdentalco.com) or call 610-239-6000

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Plymouth Meeting, PA 19462 U.S.A.
Made in U.S.A.