Knit-PakTM

Impregnated Knitted Cord

US INSTRUCTIONS FOR USE

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Intended Use:

Knit-Pak+ is a prescription-only, Aluminum Chloride Hexahydrate impregnated cord is used for temporary chemo-mechanical retraction of gingival tissues and hemostasis of the gingival margin around the tooth. It is intended for accurate and enhanced results when acquiring dental impressions, for cavity preparation or wherever hemostasis and retraction is required.

Device Description:

Premier® Knit-Pak+ is an aluminum chloride impregnated gingival retraction cord created from knitted microfibers for temporary chemo-mechanical retraction of gingival tissues and hemostasis of the gingival margin around the tooth.

Contents:

100 inches (254 cm) Aluminum Chloride impregnated cord each in self-cutting container.

| Item no. | Product Specifications |
|----------|--|
| 9007560 | • Color – Dark Green • Size – 000 |
| 9007561 | • Color – Green • Size – 000 |
| 9007562 | • Color – Brown • Size – 00 |
| 9007563 | • Color – Purple • Size – 0 |
| 9007564 | • Color – Blue • Size – 1 |
| 9007565 | • Color – Orange • Size – 2 |
| | AlCl ₃ Content (wt.) - 0.5±0.3 mg/inch |
| | 1.0±0.4 mg/inch |

Contraindications:

Knit-Pak+ is impregnated with aluminum chloride.

Do not use on persons:

- With known allergy to aluminum chloride.
- With cases of acute gingivitis or acute necrotizing ulcerative gingivitis.
- With an intraoral site that has active infection or inflammation.
- With significant periodontal disease or furcation involvement.

PRECAUTIONS:

- Be sure to rinse treated area before taking impression to avoid inhibition of polymerization of impression material.
- The reuse of a single-use device can expose patients and staff to cross-contamination.
- Federal law restricts this device to sale by or on the order of a dentist.

WARNINGS:

- Do not apply excessive force and violate the biological width when packing the retraction cord into sulcus.
- Do not leave the cord in the gum for more than 20 minutes as this may cause a lasting recession of the gum.
- Ensure all aluminum chloride is removed by rinsing prior to dental impression to avoid inhibition of polymerization of impression material.
- Do not reuse cord. Cords are single-use only.
- Do not use in patients with known allergies to Aluminum Chloride.
- Aluminum Chloride is irritating to skin and eyes.

Adverse Reactions:

Adverse reactions may include but are not limited to:

- Recession of the gums from prolonged use or excessive force in sulcus.
- Additional procedures may be necessary if impression material was inhibited due to residual Aluminum chloride.
- Allergic type of reactions such as swelling, inflammation, irritation, and pain could occur in persons with known allergy to Aluminum chloride.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Materials:

NP Conjugated dyed Yarn (Nylon-6, PET) Impregnated with Aluminum Chloride.

| Item no. | Product Specifications |
|----------------------|------------------------|
| Body/Round container | PE (Polyethylene) |
| Cap | PE (Polyethylene) |
| Cutter | Ceramic Cutter |

Directions for Use:

- Evaluate the health of the gingiva and the depth of the sulcus.
- Select the impregnated cord of appropriate-size and cut the cord to required length by closing the cap (ceramic blade in cap will sever the cord).
- Apply the cord tightly to the cervix of the tooth and carefully insert into sulcus by means of a cord packer hand instrument.
- Leave the cord(s) in the place and wait one (1) to three (3) minutes to displace the sulcus and stop any bleeding caused by preparation (do not exceed 20 minutes).
- Before removing the cord, it should be wetted/rinsed with water so it won't grab and tear gingival tissue when removed creating more bleeding.
- Remove cord; rinse the area with a firm air/water spray to complete removal of Aluminum Chloride and dry preparation.
- Examine the treatment site to verify hemostasis.
- If the bleeding is persistent when the cord is removed, repeat the procedure from step 2 to step 7 until hemostasis is achieved.

Storage:

Close the cap after use and store in a dry place.

Shelf Life:

- Knit-Pak+ has a shelf life of 3 years (36 months) from the date of manufacturing.
- Do not use after device has expired. See expiry date on the device label.

For Further Information:

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

A full explanation of symbols used on Premier® Dental packaging and literature is located at premierdentalco.com/symbol-glossary

| Symbols | Symbol Meaning |
|---------|---|
| | Expiry date |
| | Manufacturer |
| | Batch number |
| | Catalog Number |
| | Irritant |
| | Date and Country of Manufacture |
| | Do Not Reuse |
| | Caution: See instructions for use |
| | CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician |
| | Keep dry |
| | Keep away from sunlight |
| | Medical Device |
| | See IFU on premierdentalco.com |

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CH REP MDSS CH GmbH • Laurenzenvorstadt 61
5000 Aarau, Switzerland

UK Responsible Person:
MDSS-UK RP Ltd. • 6 Wilmslow Road, Rusholme,
Manchester M14 5TP, United Kingdom

Premier® Dental Products Company
1710 Romano Drive
Plymouth Meeting, PA 19462 USA
888-670-6100 • 610-239-6000
premierdentalco.com

IFU# 97560 1123059 Rev8 PDF

European Union authorized representative

EC REP

European Union authorized representative

CH REP

Swiss Union authorized representative

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