**Traxodent® Unit Dose**

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**Traxodent® Unit Dose**

US/UK Hemostatic Retraction Paste System

## INSTRUCTIONS FOR USE

## Intended use / Indications for use:

Traxodent™ has been specifically formulated to provide gingival retraction and hemostasis. It is intended for use prior to taking an impression, for cavity preparation or wherever hemostasis and retraction is required.

## Product Description:

Traxodent™ is a product designed for dental hemostasis and/or retraction. The two main components consists of clay and 15% aluminum chloride. When the clay is in the sulcus, it bulk provides retraction. The aluminum chloride is an effective and proven dental astringent-hemostatic agent.

## Contents:

Item No.	Product Configurations
9007094	Traxodent Unit Dose Pack (24)

## Accessories:

Item No.	Product Configurations
9007089	Traxodent Unit Dose Syringe Dispenser

## Contraindications:

Do not use Traxodent in cases of:

• Do not use on persons with known allergy to aluminum chloride

• Patients with significant periodontal disease or furcation involvement.

• An intraoperative site that has active infection or inflammation

## Precautions:

• Unit dose capsules are designed for single patient use only. Do not reuse unit dose capsules

• Do not use the unit dose dispenser or capsule if broken/damaged.

• Do not insert capsule dispensing tip into the sulcus.

• Avoid contamination of product. Do not return dispensed material (paste) to their original container. Dispose of leftover material properly.

• Avoid sunlight or heat sources. Shelf life will be reduced if not stored properly.

• Prescription device, Rx only.

## WARNINGS:

• Do NOT swallow retraction paste.

• Ensure all paste is removed by rinsing prior to dental impression, to avoid inhibition of polymerization of impression materials.

• Do not use on persons with known allergies to Aluminum Chloride

• Aluminum Chloride is irritating to skin/eyes.

• Do not use the shelf life period indicated on product labeling/packaging.

• Do not freeze.

## Adverse Reactions:

Adverse reactions may include but are not limited to:

• Recession of the gingiva from prolonged use or excessive force in sulcus

• Additional procedures may be necessary if impression material was inhibited due to residual paste.

• Allergic type of reactions such as swelling, inflammation, irritation, and pain could occur in persons with known allergy to Aluminum chloride.

• Cross-contamination and infection within the gingiva is possible if single use capsules are reused with multiple patients.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

## Materials:

15% Aluminum Chloride Hexahydrate Epinephrine-free hemostatic agent in a clay matrix.

## Directions for Use (Please follow carefully)

FOR UNIT-DOSE

1. Remove a Unit Dose Traxodent retraction capsule from the individually wrapped foil pack.

2. Firmly seat the Unit Dose retraction capsule in the Premier Unit Dose Syringe dispenser.

3. Remove the dispensing cap from the Unit Dose capsule tip (immediately before application only) to access the dispensing tip.

Note: To prevent sharp edges, do not modify the capsule tip.

4. For initial use, dispense a small aliquot of the Unit Dose Traxodent paste onto a mixing pad.

5. Rinse and air dry prepared tooth. Extrude Traxodent paste slowly into the sulcus while maintaining the dispensing tip just above the sulcus and align approximately parallel to the gingival crest. This will allow for a more even and rapid retraction.

6. Ensure the retraction is of sufficient volume to displace the gingiva into the sulcus and around the prepared tooth to achieve adequate tissue retraction.

7. Allow Traxodent paste to remain in the sulcus for two to three minutes. Often, the marginal gingiva will grow pale when retraction is achieved.

8. After hemostasis has been achieved, Traxodent Paste should be removed thoroughly by rinsing with a mixture of air-water and with the aid of saliva ejector or suction tip.

9. Examine treatment site to verify complete removal of the Traxodent prior to taking the impression.

10. Dispose the unit dose capsule as per local laws.

## Preprocessing: (ONLY for Syringe Dispenser)

Cleaning:

Inspect and sterilize before each use. Do not use phenols or iodophors

containing cleaning agents and do not heat above 177°C (350°F).

Sterilization:

1. Place each syringe-dispenser in a separate paper/plastic steam sterilization pouch. If using a sterilizing cassette, ensure that the sterilizer's maximum load is not exceeded.

2. Placed bagged syringe-dispenser into a steam autoclave for the autoclave manufacturer's instructions.

a. Pre-vacuum Steam Sterilization - Full Cycle: 132 °C (270°F) for 4 minutes

3. To dry use the drying cycle of the steam autoclave. Set cycle for 20 to 30 minutes. Do not exceed 137°C (28°F).

## Storage:

• Do not store at temperature over 28°C (82°F).

• Do not freeze.

• Store in a dry place.

## SheLF Life:

• Traxodent has a shelf life of three (3) years from the date of manufacturing.

• Do not use after product has expired. See expiry date on the product label.

## Further Information:

To obtain an SDS or IFU visit www.premierdentalco.com or call Premier at 610-239-6000.

## Symbols:

A full explanation of symbols used on Premier Dental packaging and literature is located at premierdentalco.com/symbol-glossary.

Symbols

Symbol Meaning

Expiry Date

Date and Country of Manufacture

Manufacturer

Batch number

Catalog Number

Caution: See instructions for use

Indicates the upper limit of temperature to which the device can be safely exposed.

! Irritant (Skin and Eye) Skin Sensitizer.

Keep away from sunlight

Rx ONLY CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician

Single Use

Medical Device

European Union authorized representative

CH REP Swiss Union authorized representative

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Description of the product:

Traxodent™ is a product designed for the hemostasis and the retraction dentales.

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