2pro®

Total Access Disposable Propy Angle


U.S. Patent #7,104,796
U.S. Patent #7,153,133

Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
610-239-6000 • 888-670-6100 • premierdentalco.com

DFU# 55001 1123047 Rev12 JC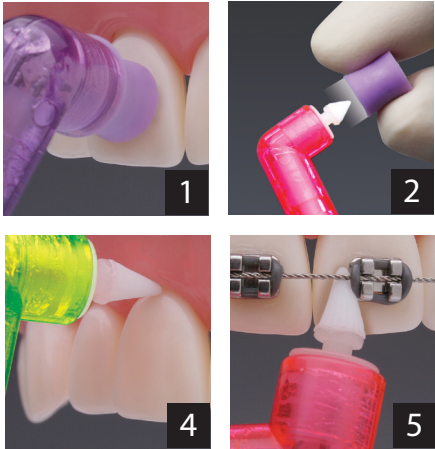

2pro®
Total Access Disposable Propy Angle

US

INSTRUCTIONS FOR USE

Description:
2pro® is a single-use disposable prophy angle uniquely designed with a prophy cup and Total Access Tip for cleaning and polishing teeth during a dental hygiene appointment. 2pro's prophy cup is ideal for cleaning and polishing facial and lingual tooth surfaces. Remove the prophy cup and use the Total Access Tip to clean and polish gingival and interproximal areas, veneer margins and around implants or orthodontic brackets. 2pro's Total Access Tip can be used to clean and polish occlusal surfaces. 2pro can be used with Enamel Pro® Prophylaxis Paste or any other prophylaxis paste. 2pro is latex free and fits a nose-cone low-speed handpiece.

Intended Use:
A single-use disposable prophy angle uniquely designed to operate a 2-in-1 technique with a prophy cup and Total Access Tip for cleaning and polishing teeth. Dental Use Only.

Indications for Use:
The prophy cup can be removed and the Total Access Tip can be used to clean and polish gingival and interproximal areas, veneer margins, and around implants or orthodontic brackets.

	Item no.	Product Specifications		Item no.	Product Specifications
Bag of 144	5500101	Soft/Short Cup (Purple Cup)	Bag of 1,000	5510001	Soft/Short Cup (Purple Cup)
	5500102	Firm/Short Cup (Blue Cup)		5510002	Firm/Short Cup (Blue Cup)
	5500103	Soft/Long Cup (White Cup)		5510003	Soft/Long Cup (White Cup)
	5500104	Firm/Long Cup (Green Cup)		5510004	Firm/Long Cup (Green Cup)

Contraindications:
Do not use 2Pro with high-speed hand pieces (3000+ RPM)

PRECAUTIONS:

- Product is designed as a single-use, disposable device.
- Do not sterilize or re-use.
- Ensure the cup and angle is securely attached, prior to use.

Rx only, Prescription device.

WARNINGS:

- Product is designed for use on a low piece handpiece. Do Not operate 2Pro above 3000 RPM.
- Do Not ReAttach the cup after removing it from the hub of your 2Pro Disposable Propy Angle's total access tip.

ADVERSE EVENTS:

- Choking may occur from the detachment and swallowing of the disposable prophy angle or cup.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Materials:
Cup and body are Latex-Free.

- Operating Instructions:**
1. Remove 2pro Total Access Disposable Propy Angle from packaging.
 2. Insert the shaft of 2pro into the nose-cone of your low-speed handpiece. Align the slot of the prophy angle with the guide pin on your handpiece and lock into place.
 3. Safely operate 2pro for a few seconds outside the mouth to ensure that it is attached securely to the handpiece.
 4. To polish facial and lingual tooth surfaces, use the cup on your 2pro Total Access Disposable Propy Angle. (See Image 1)
 5. To clean and polish gingival and interproximal areas, veneer margins and around implants or orthodontic brackets, remove the prophy cup and use the Total Access Tip. (See Image 2, 3, 4 & 5)
 6. To clean and polish occlusal surfaces, use the cup or Total Access Tip on your 2pro Total Access Disposable Propy Angle.
 7. Do not operate 2pro above 3,000 rpm.
 8. **Do not reattach the cup after removing it from the hub of your 2pro Total Access Disposable Propy Angle.**
 9. 2pro is designed for single-patient use. Do not sterilize or reuse.
 10. Dispose of 2pro properly after each patient use.

Storage:

- Always store in the original protective packaging.
- Store in a dry environment and protected against contamination.

Disposal:
Dispose the 2Pro Propy angle after each patient-use in accordance to CDC Guidelines for Infectious Waste and federal, state and local regulations.

For Further Information:
To obtain IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbols:
A full explanation of symbols used on Premier® Dental packaging and literature is located at www.premierdentalco.com/symbol-glossary

Symbols	Symbol Meaning
	Date and Country of Manufacture
	Manufacturer
	Batch number
	Caution: See instructions for use
	Do Not Reuse
	Latex-Free
	See IFU on premierdentalco.com
	Medical Device
Rx ONLY	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Catalog Number
	Do not use if package is damaged

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