Premier X5 Sectional Matrix System®

Comprehensive Solution for Class 2 Interproximal Restorations

Mesial & Distal Ring Placement: The ring may be placed mesially and/or distally for better access such as the case with MOD restoration. (Figure 7)

Dual-Mesial Ring Placement: Premier X5 ring may be stacked over each other for better access such as the case with MOD restoration. (Figure 8)

3. Performance Restoration: Proceed to fill entire cavity preparation with desired composite material.

4. Premier X5 Removal: Remove ring using Ring Forceps, remove wedge(s) using Pin-Holder, separate matrix from composite using a thin bladed instrument by pushing the instrument into the embrasure. Fold matrix wings backwards and composite again from buccal and lingual surfaces. Remove matrix with Pin-Holder; grip one of the matrix holes with the Pin-Holder and gently wiggle the matrix out. (Figure 9)

If needed, you may use another Pin-Holder on the "opposite" side of the matrix and move the matrix back and forth, or upward, until loose.

5. Contour & Polish Final Restoration:

Reprocessing Instructions for Rings and Instruments

Manual Cleaning:

1. Prepare enzymatic solution per manufacturer's recommendation.

2. Using a lint-free cloth dampened with the prepared detergent thoroughly wet the rings and instruments with the prepared detergent.

Note: If needed, using a soft bristled brush, brush the ring until all visible soil is removed

3. Rinse the rings and instruments under potable running tap water for a minimum of 30 seconds to remove detergent residue.

4. Dry the rings and instruments using a clean lint-free cloth.

5. Visually inspect each ring and instrument for visible soil. If soil is seen, repeat the cleaning procedure.

Steam Autoclave Sterilization:

1. Place each ring and instrument (Forcips & Pin Holder) in a separate paper/plastic steam-sterilizing pouch. If using a sterilizing cassette, ensure that the sterilizer's maximum load is not exceeded.

2. Place bagged rings and instruments (Forcips & Pin Holder) into a steam autoclave, per the autoclave manufacturer's instructions.

3. Rings and instruments should be steam autoclaved at 134°C (273.2°F) for at least 3 minutes for sterility prior to clinical use. Do not use chemical sterilization techniques. Other sterilization methods are at the discretion and sole responsibility of the dental practitioner.

4. Dry after sterilization.

5. Visually inspect to ensure that all contamination has been removed. Check for distortion, damage or wear. Discard damaged or worn rings.

NOTE: The rings and instruments should remain bagged until ready for use. Recommended ring use is 5 sterilization cycles before discarding. Excessive clinical use may result in insufficient interproximal separation force, resulting in less desirable clinical results.

Storage: The rings, wedges, matrices, and instruments should remain in original protective packaging until ready for use.

Disposal: Dispose of the rings, wedges, matrices and instruments in accordance to facility or institutional guidelines/procedures.

Further Information: To obtain EU, visit premierdentalco.com or call Premier at 60-239-6000.

Symbols: A full explanation of symbols used on Premier Dental packaging and literature is located at premierdentalco.com/symbol-glossary

Symbols	Symbol Meaning
	Date and Country of Manufacture
	Manufacturer
	Batch number
	Catalog number
	See IFU on www.premierdentalco.com
	Medical device
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Single Use, Do Not Re-Use
	Non-Sterile
	Steam Sterilize
	Swiss Union authorized representative
	European Union authorized representative
	Do not use if package is damaged

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