



## WATCH VIDEO

Scan the QR code to see Premier's extensive technique video library that includes the latest tips and tricks for using VeneerNow in a wide range of clinical scenarios.



## PREMIER DENTAL

# A Same-Day Solution for Chairside Veneers

*Composite veneer system paves the way for efficient, predictable, and affordable chairside restorations that send patients home with highly esthetic results*

**P**orcelain veneers are often considered the epitome of cosmetic dentistry. But if these beautiful restorations have any kryptonite at all, it can largely be traced to 2 givens: a hefty price tag bound to be a strain on many patients' wallets, and a process that usually spans at least 2 office visits.

VeneerNow eliminates those concerns with an injectable matrix system that reduces chair time, simplifies workflows, and offers highly esthetic, natural-looking results. It's a solution that's "quick and cost-efficient with very good esthetics—all in a single visit," according to Jason Goodchild, DMD, Vice President of Clinical Affairs at Premier Dental.

### Fast, Predictable Results in 1 Visit

"Porcelain veneers offer top-of-the-line esthetics, but you need at least 2 visits," Dr. Goodchild continued. "They're expensive and may require invasive tooth preparation. VeneerNow is affordable and offers very good esthetics in a single visit, often with no preparation at all."



The images above show teeth Nos. 6–11 before and after restoration using VeneerNow.



Versatile and compatible with any flowable or packable composite, VeneerNow features a full anatomical design that replicates natural tooth shape, as well as a natural cervical line for a precise emergence profile. The closed-system matrix creates a smooth, void-free composite layer for easier finishing and polishing, while an injection stalk ensures optimal placement. Plus, built-in side wings enhance stability and minimize interproximal cleanup.

"VeneerNow really shines because it has wings that go between the teeth, which allow me to inject the composite without leaving much excess," he said.

### Versatile & Efficient

VeneerNow can be used in a wide range of clinical scenarios, including peg laterals, fractured incisors, diastemas, composite crowns, misalignments, rampant caries, and Class V restorations.

"This system works well when a patient walks in with a broken tooth on a Friday and has big plans for the weekend, because I've got the solution right there, ready to use," Dr. Goodchild added. "To me, efficiency means being able to do the restoration right the first time without requiring multiple steps, and with VeneerNow, I can easily place something that looks great right off the bat."

## FOR MORE INFORMATION:

888.670.6100

premier  
dentalco.com