

The #1 Choice

Reliable Hemostasis & Retraction



Traxodent®

Hemostatic Retraction Paste System

Award-winning Traxodent is the #1 choice¹ among dental professionals because it works fast, helps capture accurate impressions and rinses easily.

Only Traxodent delivers...



Effective hemostasis & retraction
Predictable results & accurate impressions



Fast and easy handling
Works within 2-3 minutes



Rinses clean with no residue
No interference with future bonding & cementation



Versatility
Use alone, with retraction cap or cord

Count on Traxodent for fast hemostasis & retraction!



1. SDM Northcoast,
North America 2024.



premier®

Inspired solutions
for daily dentistry®

Premier® Dental Products Company • 888-670-6100 • premierdentalco.com

©2023 Premier Dental | All Rights Reserved

Traxodent® is ideal for digital and conventional impressioning.

Tooth # 3 was restored and then prepped for a full contour zirconia crown.




Traxodent® was applied around the margin of tooth then a Premier® Retraction Cap was placed over the tooth. The patient was asked to bite down.



After 2-3 minutes, the retraction cap was removed and Traxodent® rinsed off. After gently drying the area, you can see the field is clean, the tissue is no longer bleeding, and the prep is ready to be scanned.



Tooth #3 was scanned. You can see views of the prep and occlusion reduction. The scans were sent to the dental laboratory for creation of the definitive crown.

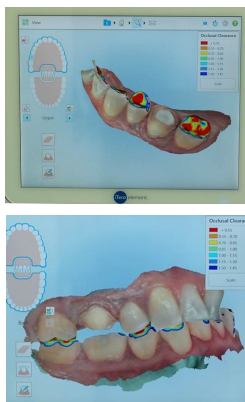

Photos courtesy of Jason Goodchild, DMD




Available in unit dose or sleek syringe - you choose!

- 9007089 Unit Dose Syringe Dispenser (3)
- 9007090 Unit Dose Value Pack - Contains: 50 capsules
- 9007091 Traxodent Value Pack - Contains: 25 syringes and 50 tips
- 9007092 Applicator Tips (50)
- 9007093 Traxodent Starter Pack - Contains: 7 syringes and 15 tips
- 9007094 Traxodent Unit Dose Pack - Contains: 24 capsules

Traxodent used with retraction cap



Syringe Traxodent; place retraction cap; have patient bite down for 2 minutes.

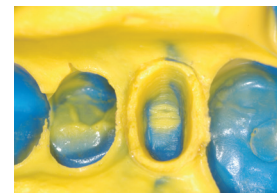

Tooth is prepared for a clean impression.

Photos courtesy of Dean Elledge, DDS, MS, Wichita State University

CAD-CAM and laser assist gingival tissue procedures



Traxodent is applied around prepared tooth.



After 2 minutes, Traxodent easily rinses away.



Final CAD-CAM restoration.

Photos courtesy of Dr. Curtis E. Jansen, DDS

Traxodent used with retraction cord



A single retraction cord is packed around the preps.



Place Traxodent over the cord. Wait 2 minutes and rinse.

Photos courtesy of Dean Elledge, DDS, MS, Wichita State University

Traxodent for routine hemostasis



Old restoration is removed; Traxodent is applied.



After 2 minutes, bleeding is stopped and field is isolated.

Photos courtesy of Richard H. Nagelberg, DDS, PC

premier Inspired solutions for daily dentistry®



Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171 • premierdentalco.com • E-mail: info@premierdentalco.com