premier

Traxodent®

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Made in Israel.

Traxodent®

Hemostatic Retraction Paste System

Intended Use /Indications for Use

Traxodent® has been specifically formulated to provide gingival retraction and hemostasis. It is intended for use prior to taking an impression, for cavity preparation or wherever hemostasis and retraction is required.

Product Description

Traxodent® is a product designed for dental hemostasis and/or retraction. The two main components consists of clay and 15% aluminum chloride. When the clay is in the sulcus, its bulk provides retraction. The aluminum chloride is an effective and proven dental astringent-hemostatic agent.

Contents:

Item No.	Product Configurations
9007091	Traxodent Value Pack (25)
9007093	Traxodent Starter Pack (7)
900-7093	Traxodent Starter Promo Pack (7+2)
9007097	Traxodent Professional Pack (2)

Accessories:

Item No.	Product Configurations
9007092	Traxodent Applicator Tips (50)

Contraindications

Do not use Traxodent® in cases of:

- Do not use on persons with known allergy to aluminum chloride
- Patients with significant periodontal disease or furcation involvement.
- An intraoral site that has active infection or inflammation.

PRECAUTIONS

- Disposable applicator tips are designed for single patient use only. Do not reuse applicator tips.
- Do not use, if the applicator tip is broken/damaged prior to use.
- Do not insert canula applicator tip into the sulcus.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.
- Avoid contamination of product. Do not return dispensed material (paste) to their original container. Dispose of leftover material properly.
- Avoid sunlight or nearby heat sources. Shelf life will be reduced if not stored properly.
- Prescription device, Rx only.

WARNINGS

- Do NOT swallow Traxodent paste.
- Ensure all paste is removed by rinsing prior to dental impression, to avoid inhibition of polymerization of impression material.
- Do not use on persons with known allergies to Aluminum Chloride
- Aluminum Chloride is irritating to skin/eyes
- Do not use after the shelf life period indicated on product labeling/packaging.
- Do not freeze

Adverse Reactions

- Adverse reactions may include but are not limited to:
 - Recession of the gums from prolonged use or excessive force in sulcus
- Additional procedures may be necessary if impression material was inhibited due to residual paste.
- Allergic type of reactions such as swelling, inflammation, irritation, and pain could occur in persons with known allergy to Aluminum chloride.
- Cross-contamination and Infection within the gums is possible if single use applicator tips are reused with multiple patients.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Materials

15% Aluminum Chloride Hexahydrate Epinephrine-free hemostatic agent in a clay matrix.

Directions for Use

- Remove a Traxodent syringe from the individually wrapped foil pack.
- Remove the cap from the syringe. Be sure to retain cap for subsequent storage of the material.
- Place annealed luer dispensing canula onto end of the syringe tip, twist lock into place. The soft luer of the annealed canula allows bending of dispensing tip for optimal access. A mirror handle can assist in bending tip to desired angle.
- Rinse and air dry prepared tooth. Extrude Traxodent paste slowly into the sulcus while maintaining the dispensing tip above the sulcus and aligned approximately parallel to the axial plane of the tooth preparation. Be careful not to jam dispensing tip into sulcus.
- Ensure you extrude sufficient material into the sulcus to achieve adequate tissue retraction.
- Allow Traxodent paste to remain in the sulcus for at least two to three minutes. Often, the marginal gingiva will grow pale when retraction is achieved.
- After hemostasis has been achieved, the Traxodent should be removed by thoroughly rinsing using air-water syringe and saliva ejector or suction tip.
- Examine treatment site to verify complete removal of the Traxodent prior to taking the impression.
- Remove canula dispensing tip and dispose of properly after use. Re-cap Traxodent syringe for storage in a sealable foil bag.

Storage

- Do not store at temperature over 28°C (82°F).
- Do not freeze.
- Store in a dry place

Shelf Life

- Traxodent® has a shelf life of three (3) years (36 months) from the date of manufacturing.
- Do not use after product has expired. See expiry date on the product label.

For Further Information

To obtain an SDS or IFU visit premierdentalco.com or call Premier at 610-239-6000.

Symbols:

A full explanation of symbols used on Premier Dental packaging and literature is located at premierdentalco.com/symbol-glossary.

Symbols	Symbol Meaning
	Expiry Date
	Date and Country of Manufacture
	Manufacturer
	Batch number
	Catalog Number
	Caution: See instructions for use
	Indicates the upper limit of temperature to which the device can be safely exposed.
	Irritant (Skin and Eye) Skin Sensitizer.
	Keep away from sunlight
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Single Use
	European Union authorized representative
	Swiss Union authorized representative

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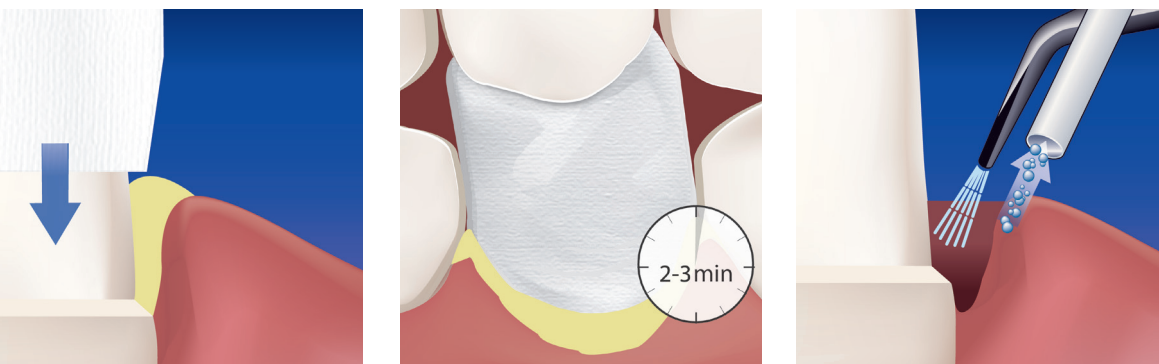
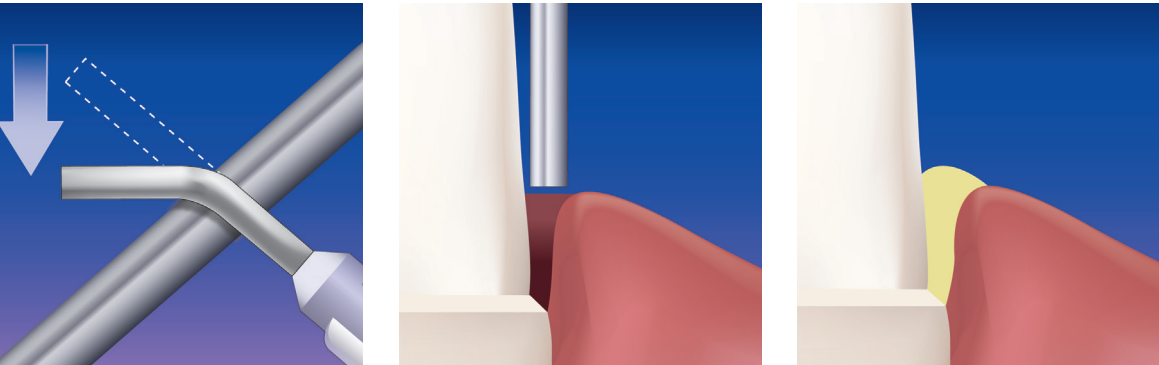
UK Responsible Person:
MDSS-UK RP Ltd.
6 Winslow Road, Rusholme,
Manchester M14 5TP,
United Kingdom

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