Traxodent® Gel FS

20% Ferric Sulfate Hemostatic System

Distributed by:

Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
US: (610) 239-6000 | International: +1 (888) 670-6100
premierdentalco.com

Inter-Med, Inc.
2200 South St., Racine, WI 53404
Tel: +1-262-636-9755

Made in USA

IFU#97107 0525076 Rev0 VA

EN

INSTRUCTIONS FOR USE

Traxodent® Gel FS

20% Ferric Sulfate Hemostatic System

Intended Use:

Traxodent Gel FS is intended for sulcus retraction prior to impression making and to control bleeding and gingival oozing in restorative and operative dentistry used with gingival retraction cord. Traxodent Gel FS facilitates the insertion of the cord into the sulcus.

Device Description:

Traxodent Gel FS is a 20% ferric sulfate gel used for immediate and profound control of bleeding. It works on both hard and soft tissue.

Contents and Setup:	
SKU	Description
9007106	Traxodent Gel FS Bulk Syringe Kit
9007105	Brush Tips Refill (100)

Contraindications:

- Do not use Traxodent Gel FS on patients with known allergies to ferric sulfate or chemical sensitivities.

PRECAUTIONS:

- Hemostatics can prevent quality adhesion and lead to staining microleakage under “direct bonded” restorations. They must be thoroughly cleaned with an air/water spray and pumice.
- For definitive bonding and luting of restorations, thoroughly scouring and cleaning the preparations are crucial to avoid microleakage and decreased bond strength.
- Hemostatic gel must be washed from the preparation site with an air/water spray to avoid staining.
- When using self-etch bonding agents, the tooth/preparation surface must be scrubbed with pumice and thoroughly washed. This is not necessary when a phosphoric etch bonding system is being used.
- To avoid blue discoloration, do not mix with epinephrine or any epinephrine-impregnated cord.
- Always thoroughly wash hemostatic from tissue before using other chemicals on the site.
- Soft tissue discoloration is temporary.
- Prefilled syringes may be used several times as long as they are properly sealed with a syringe cap and disinfected after each usage or used with a barrier.
- Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check

WARNINGS:

- For professional use only.
- Do not allow product to be ingested.
- Use protective clothing and eyewear (patient and user) when loading and handling Traxodent Gel FS
- Avoid contact with eyes and skin.
- Dispose of used tips and empty syringes properly. Please refer to individual state laws to dispose of correctly.
- Use only recommended tips. To avoid cross contamination, do not re-use tips.
- Keep out of reach of children.

Materials:

- 20% Ferric Sulfate

Directions for Use:

Preparation

- For 30mL Prefilled Syringe
- Remove luer cap from 30mL prefilled syringe.
- Attach one end of the docking port to the 30mL prefilled syringe and attach the other end of the docking port to a 1.2mL syringe.
- Slowly depress 30mL prefilled syringe plunger while guiding 1.2ml syringe plunger to desired fill.
- Separate syringes from the docking port and re-cap 30mL prefilled syringe.
- Securely attach brush tip applicator to 1.2mL syringe.
- Verify flow prior to intraoral use

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