UniClean™

Universal Cleaner

Distributed by:

Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
US: (610) 239-6000 | International: +1 (888) 670-6100
premierdentalco.com

 Inter-Med, Inc.
2200 South St., Racine, WI 53404
Tel: +1-262-636-9755

Made in USA
IFU#97100 0825012 Rev1

UniClean™

Universal Cleaner

EN

INSTRUCTIONS FOR USE

Intended Use:

Cleaning of ceramic and metal substrates prior to bonding or cementing in place.

Description of Device and Contents:

UniClean™ is an aqueous-based universal cleanser suitable for non-abrasive cleaning of the intaglio surface of prosthetic restorations after intra-oral try-in.

Contents and Setup:

SKU	Contents
9007100	5g Bottle

Contraindications:

Contraindicated for in vivo use; only intended for extraoral use.

PRECAUTIONS:

1. Ensure adequate ventilation during use.
2. Wear protective eye wear and gloves when using UniClean™.
3. To avoid cross contamination, recap and wipe product with a suitable medical grade disinfectant between uses.
4. Wash hands and other exposed areas with mild soap and water before eating, drinking or smoking and when leaving work. Wash contaminated clothing before reuse.
5. Handle in accordance with good industrial hygiene and safety procedures.

WARNINGS:

1. Only intended to be used by dental professionals.
2. Application tips or brushes are single use only. Reuse of these items may result in cross-contamination between patients.
3. Avoid contact with eyes, skin, and soft tissue.
4. Do not eat, drink or smoke when using this product.
5. Always wash hands after handling the product.
6. Keep out of reach of children.

Adverse Reactions:

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the member state where the device was used.

Directions for Use:

1. After intra-oral try-in and determining the restoration is ready to deliver, thoroughly rinse with water to remove any contamination, and dry with oil-free air.
2. Dispense several drops of UniClean™ into a well. Using an applicator brush, or micro applicator, liberally apply UniClean™ to the entire intaglio surface of the restoration using a gentle brushing motion.
3. Allow to dwell for 20 seconds, then thoroughly rinse with water and dry with oil-free air.
4. Use *Premier® Universal Primer* for superior results.

Storage:

- Store between 2-30°C (36-86°F) in a dry place.

Shelf Life:

The product has a shelf life of 2 years after the date of manufacturing.

Disposal:

Disposal should be carried out in compliance with federal, state and local regulations.

For Further Information:

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbols

A full explanation of symbols used on Premier® Dental Packaging and literature is located at premierdentalco.com/symbol-glossary

Symbols	Symbol Meaning
	Catalog number
	Batch number
	Expiration Date
	Manufacturer
	Irritant (Skin and Eye) Skin Sensitizer
	Caution: See instructions for use
	Do not use if package is damaged
Rx ONLY	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Indicates the lower and upper limit of temperature to which the device can be safely exposed.
	See Instructions For Use on premierdentalco.com

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