Dual-Arch Tray

Triple Tray®/NeoTray®/Posterior Bite Tray

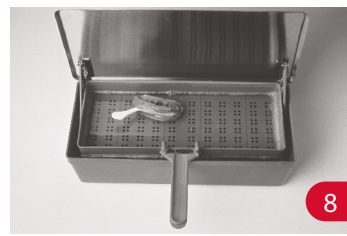
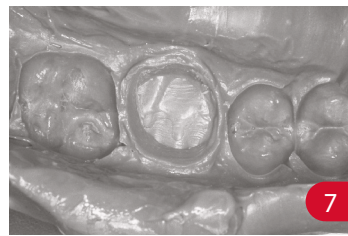
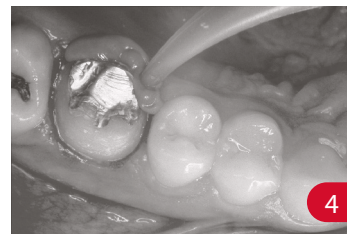
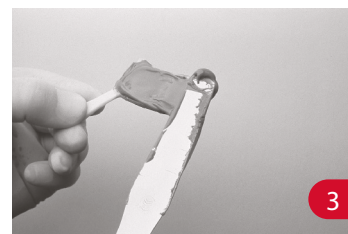
US/UK Instructions for Use – Dual-Arch Impression Trays

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Triple Tray®/NeoTray®/Posterior Bite Tray US/UK

Intended Use: Premier® Dual-Arch Trays® are intended to carry, confine and control impression material, while simultaneously capturing master/counter impression and bite registration.

Indications: The dual-arch trays are indicated for crown and bridge procedures - for one or two unit cases per quadrant. Trays are also used in the creation of veneers, inlays, onlays, provisional restorations and single tooth implant crowns and bridges.

Device Description: Premier® Dual-Arch Trays® are designed with thin transparent webbing or mesh so it can allow the capture of occlusal details during impressioning. The mesh also offers firm support of impression material before and after curing/setting. The combine dual-arch tray and impression material captures the three-dimensional shape of the teeth and oral tissue in the maxillary, mandibular arches and the bite registration.

Contents:

Description	SKU	SKU	SKU
Triple Tray® Posterior – Case of 10	1006203	Posterior – Single Box	8106203
Triple Tray® Anterior – Case of 10	1006213	Anterior – Single Box	8106213
Triple Tray® Quadrant – Case of 10	1006223	Quadrant – Single Box	8106223
Triple Tray® Assorted – Box of 35	1006233		
Triple Tray® Sideless Posterior – Case of 10	1006243	Sideless Posterior – Single Box	8106243
Triple Tray® Extended Quadrant – Case of 10	1006253	Extended Quadrant – Single Box	8106253
Triple Tray® Three-Quarter Arch – Case of 10	1006273	Three-Quarter Arch – Single Box	8106273
Triple Tray® Wide Body Posterior – Case of 10	1006283	Wide Body Posterior – Single Box	8106283
NeoTray® Posterior – Box of 48	2201NT	Sideless Posterior – Box of 48	2202NT
NeoTray® Quadrant – Box of 40	2203NT	Anterior – Box of 30	2204NT
NeoTray® Extended Quadrant – Box of 35	2207NT	Wide Posterior – Box of 48	2208NT
Posterior Bite Tray Box of 50	9048076		

Contraindications: Do not use Triple Tray/NeoTray/Posterior Bite Tray under the following conditions:

- Dental impression trays should not be used on patients with mouth/oral cancer.
- Patient is bleeding intraorally or has crevicular fluid seepage.
- If there is a missing tooth directly above or below the tooth being prepped.
- If there is a missing or absent tooth adjacent to the tooth being prepped on both (mesial and distal) sides
- The tray has become damaged or distorted.
- Patient has lost or compromised dentition or restorations

Precautions

- Tray should fit comfortably prior to use to ensure there is no impingement on the tray
- Proper hygiene and tissue management shall be performed prior to taking an impression
- Premature removal from the mouth.
- Not allowing enough working time.
- Ensure tray adhesives are used.

Warnings

- The reuse of a single-use device can expose patients and staff to cross-contamination.
- Ensure patient has no allergies to adhesives or impression materials prior to use.

Adverse Reactions

- Incorrect removal technique may tear gingival tissue upon removal from mouth.
- If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Materials	
Triple Trays – HDPE (High Density Polyethylene) as per FDA 21 CFR Part 177.1520 (Olefin Polymers)	Mesh (Triple Tray) – Cerex Nylon type 23 (Olefin Polymers)
NeoTray - K-Resin Styrene-Butadiene Copolymer Posterior Bite Tray – Polypropylene Co-Polymer	Mesh NeoTray, Posterior Bite Tray) – Fabloc knitted nylon 1518

Directions for Use

1. Place unloaded dual-arch tray into proper position. When using the posterior, quadrant or Sideless dual-arch tray, make sure the crossbar of the tray is distal to the last maxillary tooth and the tray is centered buccally/lingually.
2. Have the patient "close" on the unloaded tray and check the opposite side of the patient's mouth to make sure that the patient has closed into the correct occlusion.
3. Paint impression adhesive on tray walls only, not on webbing. Overload both sides of dual-arch tray. **Note:** dual-arch tray must be used with a firm setting material. It also may be used with hard-setting Heavy Body/Light Body one-step techniques, not two-step techniques.
4. Remove retraction cord, if used. Syringe impression material around important cervical, interproximal and edentulous areas.

Lab Techniques UK/US

Evaluate impression for any visible plastic pressure areas which may affect the accuracy of the impression.

1. Trim excess impression material from the borders of the impression. **Helpful hint:** Before pouring, mark the handle of the upper arch. This will eliminate articulating the model upside down.
2. Box impression containing die with wax and position dowel pin. Seal off any holes that stone may leak through. First, the working side is poured, adding stone or epoxy in small increments. Next, the counter side is poured. Allow to set. Pour base and allow to set. Remove boxing wax.
3. Articulate model and let plaster set.
4. Separate casts by pulling apart carefully.
5. Continue with normal crown and bridge procedures.

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premier® Inspired solutions for daily dentistry™

Premier® Dental Products Company
1710 Romano Drive,
Plymouth Meeting, PA 19462 U.S.A.
888-670-6100 • 610-239-6000
premierdentalco.com

Made in USA
Fabriqué aux États-Unis
Hecho en EE. UU.
Hergestellt in den USA
Fabbricato negli USA

premierdentalco.com/promise

EU REP MDSS GmbH • Schiffgraben 41
30175 Hannover, Germany

IFU# 106229 1225063 Rev1 PDF

View reverse side for more languages.

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