Spot-On™

37% Phosphoric Acid Dentin/Enamel Etch Gel

Premier® Etch

37% Phosphoric Acid Dentin/Enamel Etch Gel

US/UK

Indications For Use:

Premier etch products are prescription-only, 37% phosphoric acid etchant in a gel form that is used by the dental professional for etching dentin or enamel prior to tooth restoration.


Premier etch products are intended to be used in dental clinics for the purpose of enamel etch or the total etch technique for composite restorations, adhesive luting of inlays, crowns, bridges, veneers, adhesive bridges, root canal posts, splints, luting of orthodontic appliances, and fissure sealing.

Contraindications and Cautions

- If accidental contact occurs, rinse with a copious amount of water and seek medical attention immediately. Avoid contact with skin, oral soft tissue and eyes.
- Do not use Premier etch products if the patient is allergic to phosphoric acid based ingredients. If an allergic reaction occurs, seek medical attention immediately.
- Rx Only.
- Disposable tips are designed for single patient use only.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

Instructions for Use

- Prepare tooth and cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophyl paste. Rinse and dry thoroughly.
- Cover the dispensing syringe with a protective plastic sheath barrier in order to mitigate potential cross-contamination between patients.
- Securely attach luer lock applicator tip onto syringe. **Start flow on pad; not in patient's mouth. Do not push on plunger with thumb.** For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel.
- Recommended etching time for enamel is 30 seconds and for dentin is 15 seconds. To the extent possible, apply etch first to enamel and then to dentin. Leave on dentin for 15 seconds. Cap syringe immediately after use.

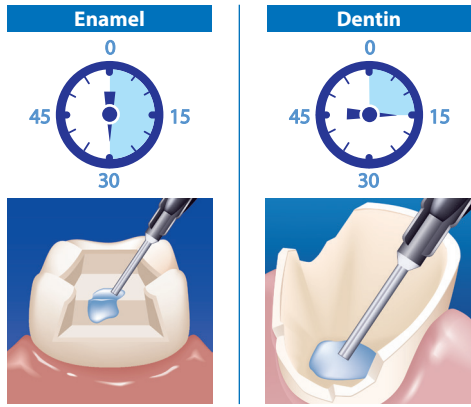- Rinse off etch with copious amounts of water. Do not disturb this surface.
- Leave surface wet or dry surface with clean, uncontaminated air, per instructions for the bonding system you are using.
- Re-cap syringe immediately after use.

Note:

- Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds and rinse with copious amounts of water.

Storage and Use

For single use only per disposable tips provided. The syringe must be closed immediately after each use. Premier Etch products should not be sterilized in the autoclave or by dry heat. Store in a dry place. Use only as directed and store as directed.

Store at room temperature between 10°C to 24°C (or 50 °F to 75°F). Do not store above 24°C (75 °F) and avoid direct sunlight.

Shelf Life: see expiry date on the label (2 years from the date of manufacture). Etch should not be used after the expiration date.

Ordering Information: Spot-On™ high viscosity 37% phosphoric acid dentin/enamel etch gel

REF 3001461 Spot-On™ Etch Intro

5 1.2ml filled syringes

20 bent disposable applicator tips

REF 3001460 Spot-On™ Etch Bulk

1 12g jumbo filled syringe with luer lock adapter

10 1.2ml empty syringes

50 1.2ml disposable applicator tips

Ordering Information: Premier® Etch 37% phosphoric acid dentin/enamel etch gel

REF 3001422 Premier® Etch 12g

1 12g jumbo filled syringe with luer lock adapter

10 1.2ml empty syringes

50 disposable applicator tips

For Further Information:

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbol Meaning

A full explanation of symbols used on Premier® Dental packaging and literature is located at premierdentalco.com/symbol-glossary

| Symbols | Symbol Meaning |
|----------------|---|
| | Catalog Number |
| | Expiry Date |
| | Date and Country of Manufacture |
| | Manufacturer |
| | Batch number |
| | Caution: See instructions for use |
| | The upper and lower limits of temperature of use, storage and transportation |
| | Keep away from sunlight |
| | Corrosive |
| Rx ONLY | CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician |
| | Consult instructions for use |
| | Do not use if package is damaged |
| | Medical Device |

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