

Inspired Innovation with Premier®

new!

MultiMatch Flow™

Chameleon Antimicrobial Composite
with BioArmor™

See page 1.



BioArmor™
Antimicrobial Protection

Premier® Restorative Products



What's New from Premier?

Lumira™ • MultiMatch™ • Bond-PR™
StackFlow™ • UniClean™ • Traxodent® Gels
See pages 1–8.



new!

MultiMatch™ Flow

Chameleon Antimicrobial Composite

Protects. Adapts. Lasts.

Experience the next evolution in flowable composite performance. MultiMatch™ Flow Chameleon Composite with BioArmor™ Antimicrobial Protection is formulated to help reduce bacterial load^{1,5} and demineralization², supporting healthier restorations that stand the test of time.

The nano-hybrid formula offers optimal handling, is self-leveling, and provides excellent adaptation – even in micro-preps and tight margins. With strong physical properties and low shrinkage³, it delivers reliable performance across everyday restorative procedures.



BioArmor™
Antimicrobial Protection



BioArmor™ Antimicrobial Protection

Lowers bacterial loads^{1,5} with proven activity across a broad pH range (2.5–10.4).⁶



Built-In Defense Against Acid Challenges

Reduces demineralization²



Optimized Handling

Easy-flowing, self-leveling for smooth placement and excellent adaptation



Low Shrinkage + Strong Physical Properties

Helps maintain marginal integrity and long-term durability³



EZ-Blend™ Shade Simplicity

One universal shade that blends to match the 16 VITA® classical A1-D4® shades⁴

PRO-TIP:

One Solution, Multiple Applications

Cleared for Class I-V restorations, pit and fissure treatments, and orthodontic aligner buttons.


3005230 Shade: Universal

Contains: 2 x 2.3g Syringes, 20 Applicator Tips (20 gauge)

3005235 Applicator Tips



Contains: 100 Applicator Tips (20 gauge)

Antimicrobial Protection*

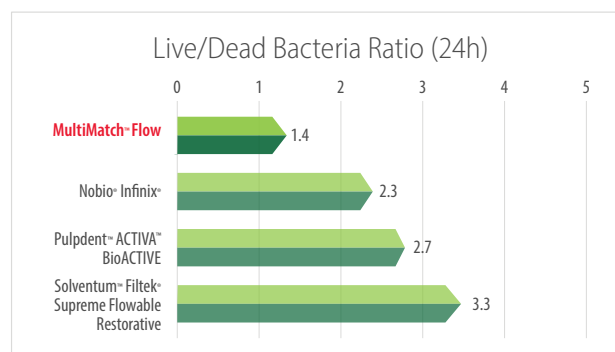

Control composite with no antimicrobial properties

Premier® MultiMatch™ Flow with BioArmor™

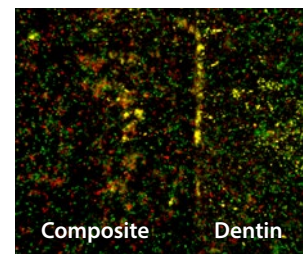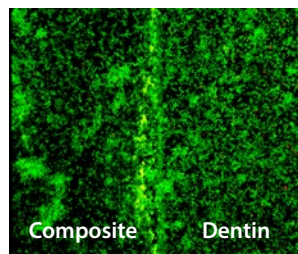

Dead bacteria (red) | Live bacteria (green)


Teeth incubated in an inoculated biofilm fermenter for 24 hrs at 37°C

VITA® and VITA classical A1-D4® are not trademarks of Premier Dental Products Company.
1-6. Data on file. / * Data on file. Sponsored study conducted by Temple University. / Nobio®
Infinix, Pulpdent® ACTIVA™ BioACTIVE, Solventum™ Filtek® Supreme Flowable Restorative
are not trademarks of Premier® Dental Products Company.

EZ-Blend™ TECHNOLOGY

Proprietary Shade Technology

And with our proprietary EZ-Blend™ shade technology, a single universal chameleon shade naturally adapts across all 16 VITA classical A1-D4® shades.⁴

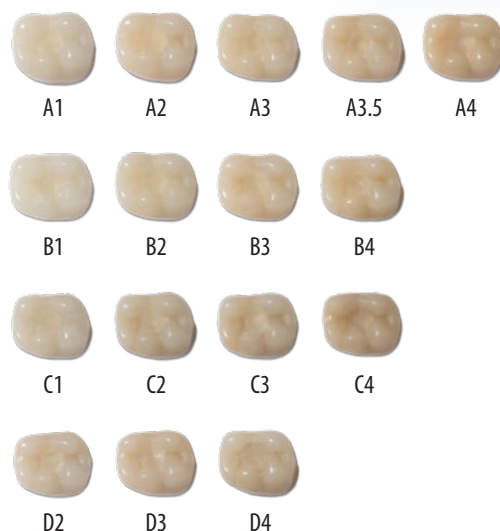

Indications for use

- Direct restorations in all cavity classes (I-V)
- Base / liner under direct restoration
- Veneering of discolored anterior teeth
- Splinting of mobile teeth
- Extended pit and fissure sealing in molars and premolars
- Repair of composite/ceramic veneers
- Retention of thermoplastic aligners

The addition of antimicrobial particles to MultiMatch™ Flow Chameleon Antimicrobial Composite reduces demineralization and the total amount of bacteria at the material surface, which is part of the caries-formation process.

Antimicrobial Protection


BioArmor™ antimicrobial-infused silica contains Octenidine Dihydrochloride (OCT), an antimicrobial agent that disrupts microbial cell membranes, leading to rapid cell lysis and death. BioArmor™ is effective against a range of pathogenic microorganisms, such as bacteria, fungi and viruses, that make up biofilm. Its broad-spectrum efficacy and long history of safe, effective use make it a trusted technology in dental applications. It helps reduce bacterial load and demineralization.




A positively charged component of BioArmor™, Octenidine dihydrochloride (OCT) leaves the composite surface and enters the biofilm.



It penetrates the negatively charged microbial cell membranes disrupting their integrity. This process continues as OCT diffuses through Biofilm.

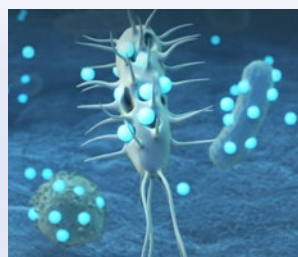

OCT kills bacteria and critically disrupts fungus and virus, stopping them from growing and multiplying.

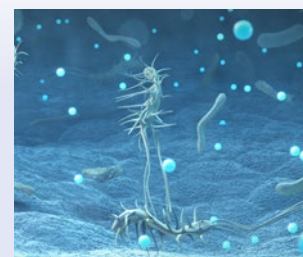

This protective process works for the life of the restoration, with proven activity across the broad pH range (2.5-10.4).

new!

MultiMatch™

Universal Chameleon Restorative Composite



Simplified Shade Matching.
Beautiful Results.

MultiMatch™ Universal Chameleon Restorative Composite takes the complexity out of shade selection – covering all VITA® classical A1–D4® shades with just three EZ-Blend™ options – Light, Medium and Dark. Its advanced optical properties deliver a seamless, natural blend that patients notice – or don't notice – for all the right reasons. And you'll appreciate the ease of placement, superior handling, and excellent physical properties.³ Talk about workflow efficiency!²



Chameleon Blending Properties

*Outstanding shade match²
& long-lasting color stability³*



Simplified Shade Selection

*3 EZ-Blend™ shades =
16 classic VITA® shades²*



Exceptional Handling

*Easy to place, sculpt,
finish & polish¹*



Universal

*For all cavity classes,
anterior & posterior*



Excellent Strength & Durability

Resists chipping, fractures & wear⁴

EZ-Blend™ TECHNOLOGY

EZ-Blend's three chameleon shades deliver a more genuine, effortless result versus a single-shade composite.

Composite Shade	VITA® Shade Guide Equivalent
LIGHT	A1, A2, B1, B2, C1, C2, D2
MEDIUM	A3, A3.5, B3, B4, D3, D4
DARK	A4, C3, C4

**3 shades
equal**
16 VITA®
shades

MultiMatch™ Syringe

Contains one 4g syringe

3005200 Light Shade
3005201 Medium Shade
3005202 Dark Shade

MultiMatch™ Capsule

Contains 20-pack unit dose capsules

3005210 Light Shade
3005211 Medium Shade
Contains 10-pack unit dose capsules
3005212 Dark Shade

new!

Bond-PR™

Universal Adhesive



You will appreciate the performance and reliability of Bond-PR™ universal adhesive. This versatile solution delivers high-strength adhesion across all restorative substrates¹⁻⁴ with the flexibility to work seamlessly in any etching technique.²⁻⁵

Its 10-MDP-enhanced formulation ensures durable bonding to zirconia¹⁻³ and ceramics – no separate primers required.¹⁻⁵

The result is a streamlined workflow, with a formulation designed to help reduce or eliminate post-operative sensitivity and deliver predictable outcomes you can rely on.



**One Product,
All Techniques**

Compatible with
self-, selective-,
& total-etch


3005250 Bond-PR™ Standard Pack

Contains: 1 x 5 ml bottle

3005255 Bond-PR™ Value Pack


Contains: 3 x 5 ml bottles

Bond-PR™ Shear Bond Strength, MPa*

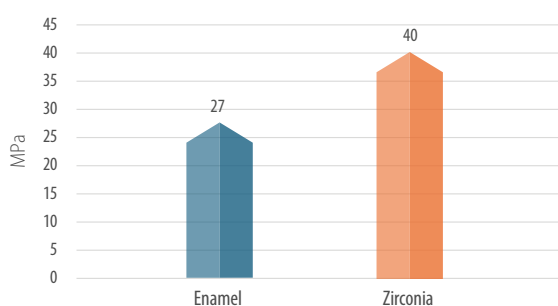

*Data source: Independent testing conducted by The Dental Advisor.
Study sponsored by B.J.M. Laboratories Ltd.

Bond-PR™ Shear Bond Strength to Unetched Dentin**

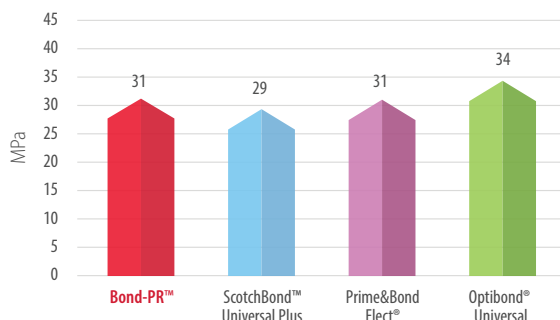

**Data source: Independent testing conducted by the University of Alabama.
Study sponsored by Premier Dental. / ScotchBond™, Prime&Bond Elect®,
and Optibond® are not trademarks of Premier Dental.



Contains 10-MDP

For bonding to zirconia¹ and hydroxyapatite in enamel and dentin.^{1,7}



Universal Compatibility

Optimized for use with all restorative substrates.¹⁻⁵



Streamlined Workflow

Eliminates need for separate primers or activators.¹⁻⁵



Reliable Performance

Delivers high, dependable bond strength you can rely on.¹⁻⁵



Reduced Sensitivity

Lowers post-operative sensitivity in self-etch mode.⁶



One Product, All Techniques

Compatible with self-, selective-, and total-etch.²⁻⁵

1-6. Data on file. 7. Carrilho E, Cardoso M, Marques Ferreira M, Marto CM, Paula A, Coelho AS. 10-MDP Based Dental Adhesives: Adhesive Interface Characterization and Adhesive Stability—A Systematic Review. Materials. 2019; 12(5):790.

new!

Lumira™


LED Curing Light



Technology Inspires Clinical Confidence.

Designed for today's clinicians, the Lumira™ LED curing light delivers consistent performance and durability. Lumira's lightweight, all-metal construction withstands the rigors of daily dental use while reducing hand fatigue. Its innovative low-profile head rotates 360° for effortless access and optimal illumination throughout the oral cavity. Lumira combines advanced optical technology with clinical versatility, offering a broad wavelength range to effectively polymerize CQ products, a maximum output of up to 3000 mW/cm² for rapid curing, and a high-intensity collimated beam for uniform light distribution and increased depth of cure.

With seven curing modes – including Normal, High, Turbo, Ortho, Soft, Pulse, and Check (Transillumination) – the Lumira LED curing light provides a comprehensive solution for delivering predictable, reliable cures, even in challenging deep restorations.

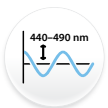

Wave Range

- 440–490 nm
- Polymerizes CQ products



Intensity

- Max irradiance 3000 (mW/cm²)
- Matches premium brands



Collimated Beam

- Less light loss¹
- Reliable depth of cure



7 Clinical Modes

- Align with restorative workflows
- Normal, High, Turbo, Ortho, Soft, Pulse & Check



Streamlined Design

- Durable all metal design
- Ergonomic & lightweight (108g)



3006100 Lumira™ LED Curing Light System

Contents:

- 1 x Lumira™ Curing Light Unit
- 1 x LED Light Head
- 1 x Charging Station
- 1 x Adapter
- 1 x Light Shield
- 1 x Point Cure Lens




new!

StackFlow™

Injectable Light-Cured Composite

NEW!

Stacks when you want.
Flows when you need.

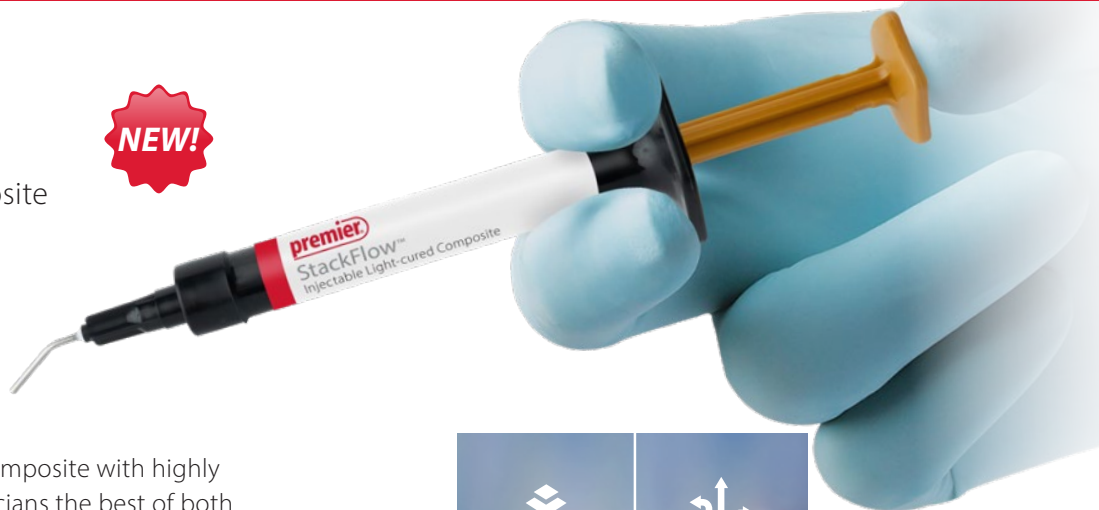

StackFlow™ is a versatile flowable composite with highly thixotropic behavior that offers clinicians the best of both worlds: control when you want it and adaptability when you need it.

A proprietary hydrophobic silane surface-treatment applied to nano fillers improves dispersion and strengthens the resin–filler interface, supporting excellent mechanical integrity and smooth, controlled handling.

StackFlow places precisely and polishes to a natural, long-lasting gloss. The result is a streamlined workflow and durable, aesthetic restorations across a wide range of indications.



Versatile Flowable Composite

Suitable for a wide range of indications



Thixotropic Handling

Holds its shape when dispensed yet flows on demand for efficient adaptation



Hydrophobic Silane Surface Treatment

Promotes uniform filler dispersion for enhanced strength & predictable handling



Strength Equivalent to a Conventional Composite

Delivers durability without sacrificing ease of placement



Zero-Bubble Syringe Tip

Ensures smooth placement while helping minimize bubbles and oozing

StackFlow™ Standard Pack

Contains: 2 x 2.5g Syringes, 20 Applicator Tips

3005270 Shade: Light


3005271 Shade: Neutral

3005272 Shade: Warm

3005273 Shade: Bleached

3005275 Applicator Tips

Contains: 100 Zero-Bubble Applicator Tips (20 Gauge)

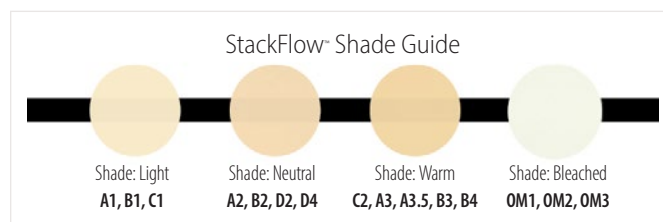

new!

UniClean™

Universal Restorative Cleaner




UniClean™ effectively removes salivary contamination from the intaglio surface after try-in – a critical step for achieving strong, long-lasting bonds. Its advanced formulation binds salivary proteins and creates a rinseable precipitate, ensuring a thoroughly clean surface.


- ✓ Advanced formulation – *binds salivary proteins ensuring a thoroughly clean surface*
- ✓ Compatible with all restorative materials: zirconia, glass ceramics, gold, metal, & composite
- ✓ Cleaning with UniClean™ promotes strong, reliable bonding performance – especially with zirconia.
- ✓ Fast 20-second application
- ✓ Ready to go from the bottle – *no shaking required*
- ✓ Optimized viscosity for accurate placement & easy handling
- ✓ Orange tint ensures visibility for coverage and thorough rinsing



SBS to Saliva Contaminated Zirconia After Cleaning With Different Cleaners*

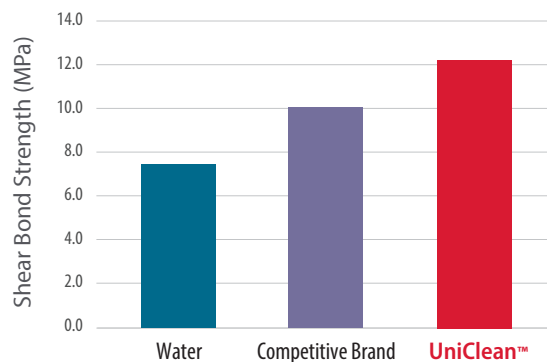

Phosphate Contamination on Zirconia vs. Cleaner*

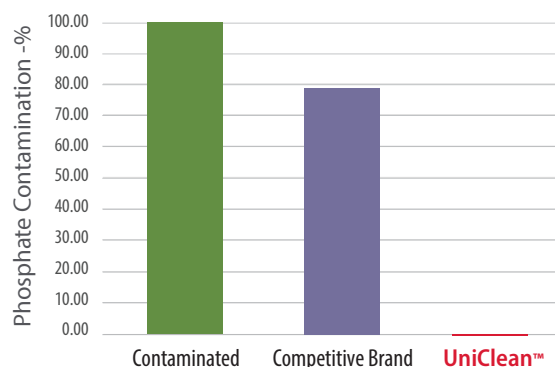

9007100 UniClean™
Contents: 5g bottle

**Orange
tint ensures
visibility!**

*Data on file. / IvoClean® is not a trademark of Premier.

Traxodent® Gel AlCl

25% Aluminum Chloride Hemostatic System

NEW!

Clear Control. Proven Performance.

- ✓ 25% aluminum chloride for effective hemostasis & fluid control
- ✓ Innovative formula boosts gel penetration for faster, more effective results
- ✓ Excellent visibility during placement & rinse
- ✓ Syringe delivery for easy, accurate application



9007101 Traxodent® Gel AlCl

Contents: 30cc (30mL) Filled Bulk Syringe, 20 Brush Tips, 20 x 1.2ml Syringes, 20 x 1.2ml Syringe Caps, Docking Port

9007105 Traxodent® Gel Brush Tips

Contents: 100 Brush Tips

Traxodent® Gel FS

20% Ferric Sulfate Hemostatic System

NEW!

Targeted Hemostasis with the Strength of Ferric Sulfate

- ✓ 20% ferric sulfate gel with optimized handling
- ✓ Rapid hemostasis & sulcular fluid control for procedural efficiency
- ✓ Excellent visibility during placement & rinse
- ✓ Syringe delivery for easy, accurate application



9007106 Traxodent® Gel FS

Contents: 30cc (30mL) Filled Bulk Syringe, 20 Brush Tips, 20 x 1.2ml Syringes, 20 x 1.2ml Syringe Caps, Docking Port


9007105 Traxodent® Gel Brush Tips

Contents: 100 Brush Tips



Traxodent®

Hemostasis Retraction Paste System

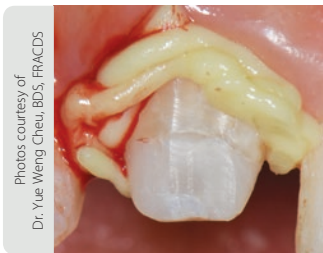

Traxodent applied



Post-op

The #1 Choice for hemostasis & retraction¹

Award-winning Traxodent® is the #1 choice¹ among dental professionals because it works fast, helps capture accurate impressions and rinses easily.

And since Traxodent doesn't irritate or discolor surrounding tissue, patients leave happier too.

Count on Traxodent for fast hemostasis & retraction!



**Effective Hemostasis
& Retraction**



**Fast & Easy
Handling**



**Rinses Clean
With No Residue**



Versatility



View Traxodent® Videos

Scan QR or visit
premierdentalco.com/Traxodent

*"The product provided excellent
retraction and is easy to use
and gentle to the tissue."*

-Mark P. Cohen, DDS
Roselle Park, NJ

Unit dose
capsules



Sleek syringe with bendable tips

1. SDM Northcoast, North America 2025.



Unit Dose Value Pack
(50 Capsules)

Routine Hemostasis



Old restoration is removed
then Traxodent is placed.



Rinse after 2 minutes. Bleeding
is stopped and field is isolated.

Photos courtesy of
Richard H. Nagelberg, DDS, PC

Ordering Information

- 9007094 Traxodent Unit Dose Pack
Contains: 24 capsules
- 9007090 **NEW!** Traxodent Unit Dose Value Pack
Contains: 50 capsules
- 9007091 Traxodent Value Pack
Contains: 25 syringes and 50 tips
- 9007093 Traxodent Starter Pack
Contains: 7 syringes and 15 tips
- 9007092 Applicator Tips (50)

Premier® Retraction Caps

Anatomic formed cotton caps work synergistically with Traxodent to quicken and assist retraction and hemostasis.

- ✓ Easily guides Traxodent into the sulcus
- ✓ Compression assists and quickens hemostasis
- ✓ Cotton absorbs fluids
- ✓ Anatomic form allows easy placement

- 9048251 Size 1 - Small (120)
- 9048253 Size 3 - Medium (120)
- 9048255 Size 5 - Large (60)
- 9048260 Assorted - (15) #1, (15) #3, (30) #5



Crown and Bridge with Retraction Cap



Syringe Traxodent and place retraction cap
on prep; have patient bite down on cap for
2 minutes.



Tooth is prepared for
a clean impression.

Photos courtesy of
Dean Ellledge, DDS, MS

Crown and Bridge with Retraction Cord



A single retraction cord is packed
around the preps.



Place Traxodent over the cord.
Wait 2 minutes.

Photos courtesy of
Dean Ellledge, DDS, MS

Loose Tissue



Cord is placed.



Place retraction cap
on prep; have patient
bite down on cap for
2 minutes.



Impression with
well-defined margins

Photos courtesy of
William Crinzy, DDS, MS

new!

VeneerNow™

Injectable Matrix System



Simplify your anterior composite restorations!

Elevate your dental practice and improve your workflows with the VeneerNow™ Injectable Matrix System – your solution for quickly creating chair-side attractive, natural-looking composite veneers and anterior restorations. Designed for ease of use, it enables composite injection with minimal clean-up.

VeneerNow also works with packable composite, offering versatility. This essential addition to your operatory toolkit makes anterior restorations fast with predictable results.

VeneerNow™ Fast, Predictable Results!



Full anatomical design
Replicates natural tooth shape for superior results



Side wings
Enhance stability & minimize interproximal clean-up



Closed system matrix
Provides a void-free smooth composite layer for easier finishing and polishing



Natural cervical line
For a precise anatomical emergence profile



Injection stalk for flowable composite
Provides easy injection for optimal placement



Versatile
Use with flowable or packable composite

Ordering information

9069237 Starter Kit

Contains:

32 Small/Medium - SM Teeth 5-12, Palmer 4 - 4 (4 ea.)

16 Medium/Large - ML Teeth 5-12, Palmer 4 - 4 (2 ea.)

9069244 Mini Kit

Contains:

8 Small/Medium - SM Teeth 7-10, Palmer 2 - 2 (2 ea.)

4 Medium/Large - ML Teeth 7-10, Palmer 2 - 2 (1 ea.)



Refills:

Small/Medium - SM

Contains: 6 matrices

9069268	Tooth 5	Palmer <u>4</u>	FDI 14
9069275	Tooth 6	Palmer <u>3</u>	FDI 13
9069282	Tooth 7	Palmer <u>2</u>	FDI 12
9069299	Tooth 8	Palmer <u>1</u>	FDI 11
9069305	Tooth 9	Palmer <u>1</u>	FDI 21
9069312	Tooth 10	Palmer <u>2</u>	FDI 22
9069329	Tooth 11	Palmer <u>3</u>	FDI 23
9069336	Tooth 12	Palmer <u>4</u>	FDI 24

Refills:

Medium/Large - ML

Contains: 6 matrices

9069343	Tooth 5	Palmer <u>4</u>	FDI 14
9069350	Tooth 6	Palmer <u>3</u>	FDI 13
9069367	Tooth 7	Palmer <u>2</u>	FDI 12
9069374	Tooth 8	Palmer <u>1</u>	FDI 11
9069381	Tooth 9	Palmer <u>1</u>	FDI 21
9069398	Tooth 10	Palmer <u>2</u>	FDI 22
9069404	Tooth 11	Palmer <u>3</u>	FDI 23
9069411	Tooth 12	Palmer <u>4</u>	FDI 24



Refill



View VeneerNow™
Instructional Videos

For additional video instruction by indications, visit premierdentalco.com/VeneerNow or scan QR code.

Hemodent®

Buffered Aluminum Chloride Hemostatic Solution



For over 50 years, the world's most sought after hemostatic dental product.

- ✓ Fast & effective
- ✓ Unique formulation
- ✓ Wide range of uses
- ✓ 3-Year shelf life


Hemodent® effectively stops minor gingival bleeding. It contains no epinephrine to help avoid cardiac reactions. Hemodent is stable and offers a long shelf life.

Recommended to arrest bleeding during tooth crown preparations, before dental impression taking, placing fillings, dental cement bases, or temporary cementing inlays or tooth crowns; also after minor dental surgery, curettage, gingivectomies, gingivoplastys and deep scaling.

9007071 10cc bottle
9007072 20cc bottle
9007073 40cc bottle

Cure-Thru® IntegraPost®

Light-transmitting Zirconia Glass Fiber Post

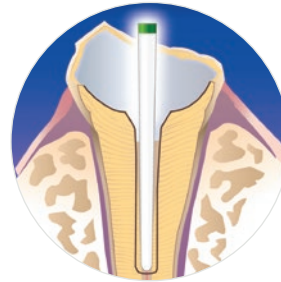

- ✓ Safe and Simple to Use – Composite Zirconia Glass Fiber composition won't fracture root and provides easy non-traumatic removal.
- ✓ Aesthetic – White color of post guarantees no grey shadow or show through.
- ✓ Strong and Flexible – Elastic modulus and flexural strength more like tooth structure allowing the post to respond like the tooth to traumatic stresses. (E-modulus 52 GPa/Flexural Strength 1650 MPa)
- ✓ Factory Silanated for Secure Adhesion – Post is chemically treated to bond with both resin cement and composite core build-up material.
- ✓ Light Transmitting – Photo curing through the post assures complete polymerization of cement around and down the post length.
- ✓ Four Color Coded Post Sizes – Multiple sizes for easy fit in all teeth and canal space. Color coding make size selection easy.
- ✓ Tapered Tip Design – Parallel body with slightly tapered tip conforms to apical root anatomy for safe and effective adaptation of post.




Refills (10 pack)

3001610	Size 4	(red)
3001611	Size 4.5	(purple)
3001612	Size 5	(blue)
3001613	Size 6	(green)

Not available in Canada.



Non-impregnated
Knitted Weave

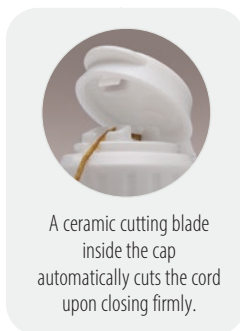

A ceramic cutting blade
inside the cap
automatically cuts the cord
upon closing firmly.



Impregnated
Knitted Weave



Knit-Pak™

Knitted Retraction Cord

Use in combination with Hemodent® or Traxodent® for tissue displacement and hemostasis.

- ✓ Absorbent knitted cotton can take in an abundant amount of hemostatic solution
- ✓ Compacts under pressure, making it easier to pack
- ✓ Will not get entangled in a bur
- ✓ Measures 100 inches / 254 cm

	Size	Color
9007551	000	Green
9007552	00	Brown
9007553	0	Purple
9007554	1	Blue
9007555	2	Orange
9007556	3	Yellow

Knit-Pak™+

Impregnated Knitted Cord

Unique Knitted Microfiber with Superior Absorbency

Knit-Pak+ is knitted impregnated retraction cord manufactured from unique microfibers.

- ✓ Aluminum chloride impregnated cord (AlCl₃-6H₂O)
- ✓ High durability and retention properties
- ✓ Resists fraying and easily packs into sulcus
- ✓ Measures 100 inches / 254 cm

	Size	Color
9007560	000.0	Dark Green
9007561	000	Green
9007562	00	Brown
9007563	0	Purple
9007564	1	Blue
9007565	2	Orange

IntegraBond™

Universal (5th gen) Adhesive System

Clinically proven IntegraBond⁷⁻¹⁵ - provides outstanding performance for all restorative challenges!

- ✓ Offers outstanding shear bond strength to all popular light, dual-or auto-cure composites.

Use IntegraBond after acid-etching.



3001411	IntegraBond - 7ml
3001412	Auto-Cure Activator - 7ml for use when light-curing is not possible
3001413	IntegraBond - 4ml

Premier® Universal Primer

Zirconia and Glass Ceramics,
Metal and Composite Primer

- ✓ Dual coupling agents works with all substrates
- ✓ Easy to use – just apply, wait 60 seconds, and dry
- ✓ No pretreatment of restoration required
- ✓ Can be used with adhesive and self-adhesive resin cement to achieve maximum bond strength



3001568	5ml bottle
---------	------------

Not available in Canada. 7–15. Full bibliography available.


Not available in Canada.

CompCore™ AF

Dual-Cure Composite Core Build-up Material

Faster & stronger
A better core experience!

Build cores with confidence! CompCore™ AF resin composite is easy to place, shape and finish. The composite cuts like dentin with excellent visibility on radiographs while its superior mechanical properties and fluoride release deliver long-term clinical success.

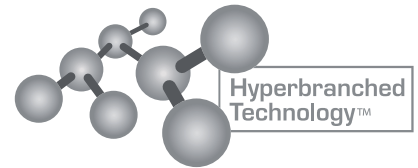


Proprietary Polymer Strengtheners



**Outstanding
Handling**



**Optional
Dual-Cure**

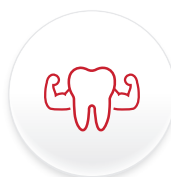

**Superior Mechanical
Properties**



**Minimum Shrinkage
of < 2.0%**



CompCore™ AF SyringeMix STACK:

9g Automix Syringes/Dual-Cure

3001512 SyringeMix STACK / Shade White

3001513 SyringeMix STACK / Shade A3

3001514 SyringeMix STACK / Shade Blue

Contains: two 9g automix syringes, 15 mix tips,
15 standard and 10 extra-fine intraoral tips.

Refill Kit

3001432 Shade White

3001433 Shade A3

Contains: 50g cartridge of CompCore AF Dual-Cure,
25 mix tips, 25 intraoral syringe tips

Accessories

3001418 SyringeMix Mix-Tips & Intraoral Tips (35 each)


3001468 SyringeMix Extra-Fine Intraoral Tips (50)

3001417 Cartridge Mix-Tips & Intraoral Tips (35 each)



Bonding Agent

(use only bonding agents compatible with dual-cure composites, such as IntegraBond™)

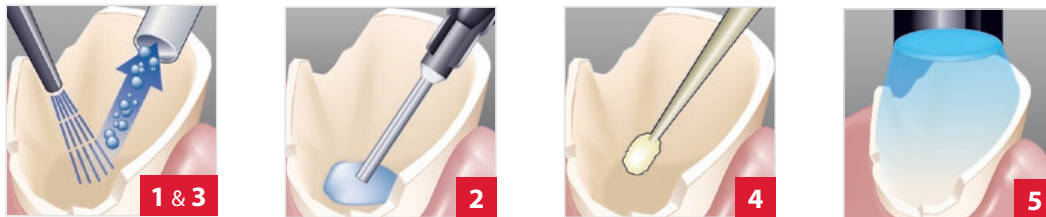

1. Clean and dry prepared tooth surface.
2. Apply 37% phosphoric acid to the cavosurface for 10 to 15 seconds.
3. Rinse and dry cavo surface, do not desiccate the tooth.
4. Apply first coat of IntegraBond™ on prepared cavosurface. Allow to sit for 15 seconds, and then air dry. Apply 2 to 3 more coats; without waiting, dry again after last application.
5. Light-cure for 20 seconds.

Core Build-up



1. Bleed syringe. Assemble intraoral tip to mix-tip, attach to syringe and bleed tip.
2. Place intraoral tip into the base of the prepared tooth and dispense. Let material rise around the tip while withdrawing to eliminate air bubbles.
3. Light-cure for 30 seconds. If the core is larger than 8mm, light-cure for at least an additional 10 seconds.
4. Check for complete hardness. The preparation can be finished immediately with a Two Striper® bur. Alternately, the core can be chemical-cured, final set-time is 4.5 minutes.

NexTemp®

Non-Eugenol Temporary Resin Cement

Your go-to temporary cement
Stays Put, Removes Easily



Now in
**Clear & Opaque
Shades!**

NexTemp is the reliable temporary cement you can count on. That's because NexTemp creates an excellent marginal seal with firm retention to help prevent premature dislodgment or microleakage.

What's more, NexTemp is the only eugenol-free resin cement containing potassium nitrate, chlorhexidine and fluoride – which studies show help mitigate post-op sensitivity.¹⁶



**Temporary
Restorations Stay Put**



**Eugenol-
Free**



**Proprietary
Formula**



**Affordably
Priced**



Opaque shade


3001470 Bulk pk.: 4-5ml automix syringes

3001471 Standard pk.: 1-5ml automix syringe, 10 mix tips

Clear shade

3001473 Bulk pk.: 4-5ml automix syringes

3001472 Standard pk.: 1-5ml automix syringe, 10 mix tips

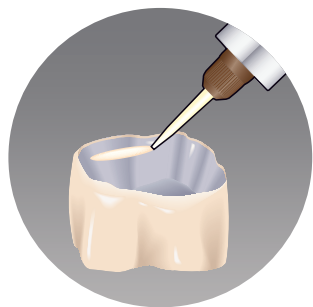


Greater Retention, Less Cement!

*Just dispense a small bead of cement
around the inside margin.*


*"This is such a superior product
to all the other temp cements
we have tried over the years."*

– Peter Schott, DMD
Middletown, RI


Premier® NexTemp® Temporary Cement is easy to use

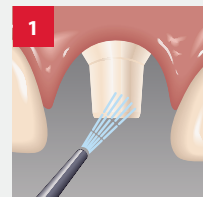


Dry and isolate the
prepared tooth.

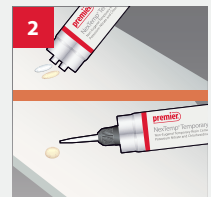


Bleed the syringe.
Mount a new automix
dispensing tip and bleed
the syringe again.

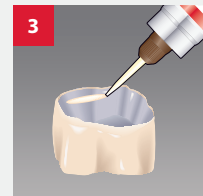


Dispense a small bead
of cement around the
inside margin.

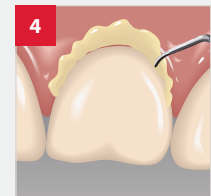

Seat the temporary
and hold in place. In
approximately 2 minutes,
excess cement reaches a
distinct gel phase and can
be removed safely.

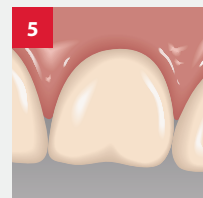

Final set occurs
between 4-5 minutes
after seating of
temporary restoration.†

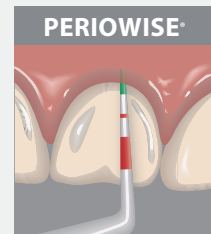

Use a PerioWise®
Periodontal Probe
for post-treatment
periodontal evaluation.

*An independent non-profit dental education and product testing foundation, December 2006 & 2007 / †Final set time is determined intraorally. Ambient set times vary and are not representative of clinical conditions. / 16. Jalalian E, Meraji N, Miraei M. A comparison of the efficacy of potassium nitrate and Gluma desensitizer in the reduction of hypersensitivity in teeth with full-crown preparations. J Contemp Dent Pract. 2009 Jan 1;10(1):66-73.

ZR-Cem®

A better zirconia cementation solution is here!

New ZR-Cem® Self-adhesive resin cement



ZR-Cem® is an innovative self-adhesive universal resin cement specially formulated for zirconia restorations. Dual-cured ZR-Cem enables a strong bond to all ceramic materials, dentin and enamel - ensuring superior retention and marginal integrity.



**Innovative
MDP Formula**



**Lasting
Color Stability**



**Virtually No
Post-op Sensitivity**



**No
Pre-treatment**



Standard Pack

1 - 8g syringe, 10 mix-tips and 10 intraoral tips

3001560 Shade Opaque White

3001561 Shade A2

3001562 Shade Translucent

Refill Tips

3001418 35 mix-tips & 35 intra-oral tips

Light cure

2-3 seconds tack cure - easily remove excess cement

For high-translucency zirconia and other light-transmittable restorations (material thickness ≤ 2.5mm), light cure all surfaces including margins for 20 seconds to accelerate final set.

Chemical cure

72°F (22°C) temperature

Working time: 1.5 - 2.5 minutes.

Higher temperatures accelerate set.

98.6°F (37°C) temperature

Gel phase: approximately 1-1.5 minutes

Final set: 3.0 - 4.0 minutes.

Higher temperatures accelerate set.

Physical Properties


Compressive strength: 150 MPA

Flexural strength: 100 MPA

Film thickness: 10 µm

Radiopacity: 250% Al

Shelf life: 2 years

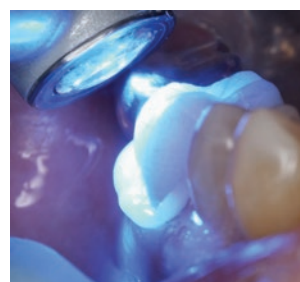


3-second tack cure



Easy clean-up

Photos courtesy of Timothy M. Bizga, DDS

ZR-Cem: superior bond strength vs. other leading self-adhesive cements to zirconia†

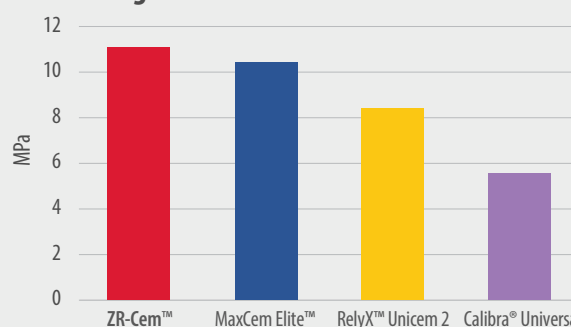

MaxCem Elite, RelyX Unicem 2 and Calibra Universal are not trademarks of Premier Dental.
† Latta, M. Creighton University School of Dentistry Study 2019.

Not available in Canada.

Premier® Implant Cement™ Plus


The ultimate solution for implant crown cementation.

- ✓ Secure retention – *for long-lasting hold*
- ✓ Retrievable – *without damage to the crown, abutment or implant*
- ✓ Easy to use – *no etch, primers or adhesive required*
- ✓ Unsurpassed radiopacity* – *excellent visibility on radiographs*

Unsurpassed
Radiopacity



Premier® Implant Cement™ is easy to use

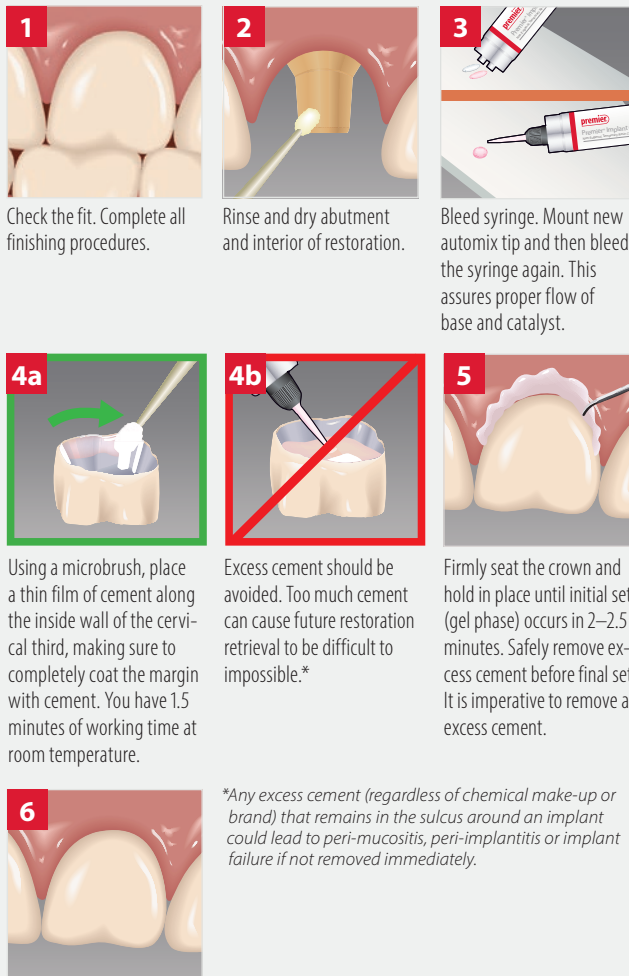


1 Check the fit. Complete all finishing procedures.

2 Rinse and dry abutment and interior of restoration.


3 Bleed syringe. Mount new automix tip and then bleed the syringe again. This assures proper flow of base and catalyst.




Using a microbrush, place a thin film of cement along the inside wall of the cervical third, making sure to completely coat the margin with cement. You have 1.5 minutes of working time at room temperature.

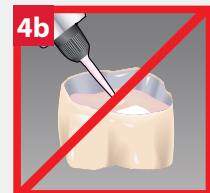

Excess cement should be avoided. Too much cement can cause future restoration retrieval to be difficult to impossible.*

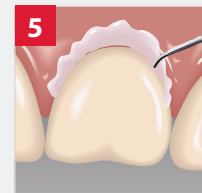


Firmly seat the crown and hold in place until initial set (gel phase) occurs in 2–2.5 minutes. Safely remove excess cement before final set. It is imperative to remove all excess cement.

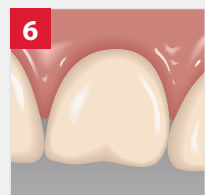

Final cement is 4–5 minutes, intraorally.

*Any excess cement (regardless of chemical make-up or brand) that remains in the sulcus around an implant could lead to peri-mucositis, peri-implantitis or implant failure if not removed immediately.

Premier Implant Cement Plus offers unsurpassed radiopacity* with excellent visibility on radiographs.

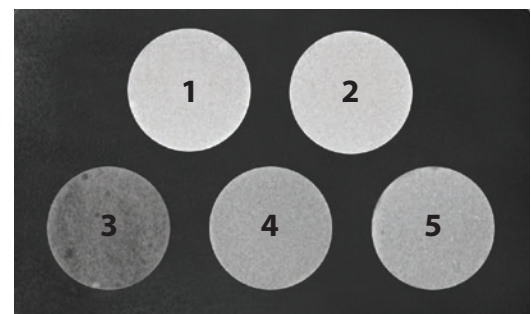

- 1 Premier® Implant Cement™ Plus - Shade Original (Pink)
- 2 Premier® Implant Cement™ Plus - Shade White
- 3 Improv®
- 4 Implantlink® Semi Xray
- 5 Retrieve™ DC

*As compared to other implant cements. Improv, Implantlink Semi Xray and Retrieve DC are not trademarks of Premier® Dental Products Co.

Standard Pack

1-5ml Automix Syringe, 10 Mix Tips
3001550 Shade Original (Pink)
3001551 Shade White

Value Pack

3-5ml Automix Syringes, 25 Mix Tips
3001555 Shade Original (Pink)
3001556 Shade White

Accessories

3001451 Automix Dual-barrel Syringe Tips (25)

Not available in Canada. Ask your Canadian dealer about Premier Implant Cement.

Triple Tray®

Proven Accurate in Over 185 Million Impressions

- ✓ Three techniques in one
– *Saves time, money and materials.*
- ✓ Adaptable, ultra-thin webbing
– *Provides accurate bite registration and low water sorption¹⁷ reducing the possibility of distortion*
- ✓ Disposable – *No cleaning involved, aseptic*
- ✓ Seven configurations – *Designed to address all clinical cases*

1006203	Posterior - Box of 48
1006243	Sideless Posterior - Box of 48
1006283	Wide Body Posterior - Box of 48
1006213	Anterior - Box of 35
1006223	Quadrant - Box of 40
1006253	Extended Quadrant - Box of 35
1006273	Three-Quarter Arch - Box of 35
1006233	Assorted - Box of 35 Contains 5 each of designs listed above



¹⁷ Independent Research Organization, July 2000.

T-LOC™ Triple Tray®

Adhesive-free Dual-arch Impression Trays

- ✓ Patented adhesive-free design
- ✓ Easier handling
- ✓ Accurate bite
- ✓ Greater rigidity in posterior
- ✓ Ensure accuracy and save time

1006204	Posterior - Box of 35
1006214	Anterior - Box of 28
1006274	Three-Quarter Arch - Box of 22



Premier X5 Sectional Matrix System®

Reliability. Convenience. Value.

A superior sectional matrix system is here.

Premier X5 – an innovative, complete, high-quality, 5 component system designed to achieve accurate Class 2 composite restorations.



9061150 X5 Intro Kit

Contains:

- (1) Matrix Pin Holder
- (1) Matrix Ring Forceps
- (10) Matrix Rings (gray)
- (25) each

Wedges

- Small (yellow)
- Medium (green)
- Large (blue)

Matrices

- 3.5mm
- 4.5mm
- 5.5mm
- 6.5mm

9061150 X5 Essential Kit

Contains:

- (10) Matrix Rings (gray)
- (25) each

Wedges

- Small (yellow)
- Medium (green)
- Large (blue)

Matrices


- 3.5mm
- 4.5mm
- 5.5mm
- 6.5mm

Reorder

-  9061190 X5 Matrix Ring (35)
-  9061170 X5 Wedge - Small (yellow) (100)
-  9061171 X5 Wedge - Medium (green) (100)
-  9061172 X5 Wedge - Large (blue) (100)
-  9061160 X5 3.5mm Matrix (25)
-  9061161 X5 4.5mm Matrix (80)
-  9061162 X5 5.5mm Matrix (80)
-  9061163 X5 6.5mm Matrix (80)

- 9061180 Matrix Pin Holder
- 9061181 Matrix Ring Forceps



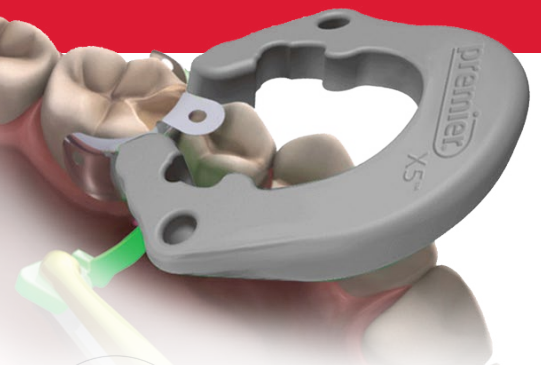

Strong



Contouring/Contacts



Compatible



Easy




Priced Right

There's never been a better time to make the switch to X5 and save without compromising quality results.




1a
Placement of Matrix. Select matrix based on the approximate occlusogingival height. Place prongs of Pin-Holder through corresponding holes in the matrix with the gold side visible to easily disengage the instrument once the matrix is in place.


When placing the matrix before the wedge, place a finger at the occlusal aspect of the matrix to hold it in place & prevent it from dislodging, or use Pin-Holder to secure matrix, proceed with wedge placement.

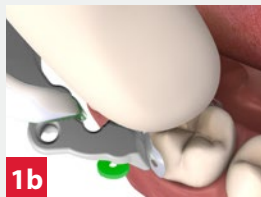

1b
Fold the matrix tab towards you so you can insert the matrix apically into the gingival crevice & adjust matrix to the proper height to obtain ideal benefit from the contour of matrix.

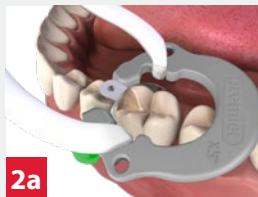


2a
Placement of Premier X5™ Ring. Use the ring forceps to grasp and slightly expand the ring. Carefully place the ring as low as possible with the tines of the ring close to the gingival margin and straddling the wedge(s).

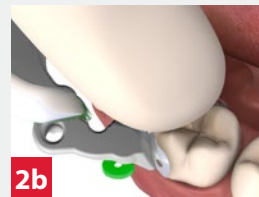

2b
When you are ready to release the ring from the Ring Forceps, place your finger and press down on the matrix tab to prevent any unwanted movement.




2c
Premier X5™ Ring in the ideal position from a buccal view.

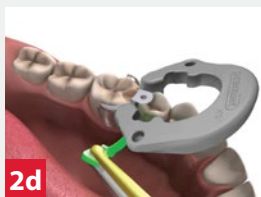


2d
Dual-Wedging: The Premier X5™ system allows for the insertion of a second wedge, to prevent any leakage, to be placed from the opposite side without removing the ring. The second wedge should be inserted underneath the first wedge.



2e
Mesial & Distal Ring Placement: The ring may be placed mesially and/or distally for better access such as the case with MOD restoration.

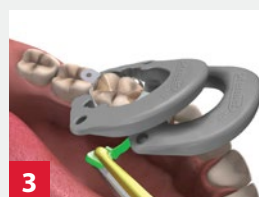

3
Dual-Mesial Ring Placement: Premier X5™ ring may be stacked over each other for better access such as the case with MOD restoration.

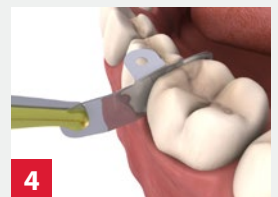

4
Premier X5™ Removal: Remove ring using Ring Forceps, remove wedge(s) using Pin-Holder, separate matrix from composite using a thin bladed instrument by pushing the instrument into the embrasure. Fold matrix wings backwards and cure composite again from buccal and lingual surfaces. Remove matrix with Pin-Holder; grip one of the matrix holes with the Pin-Holder and gently wiggle the matrix out.

If needed, you may use another Pin-Holder on the "opposite" side of the matrix and move the matrix back and forth, or upward, until loose.

NEW!

PremierAir®

Titanium Implant Scalers
4 designs crafted
specifically for supra and
subgingival maintenance



Barnhart 5/6 Ti • 9061460



NV4 Ti • 9061461



Langer 3/4 Ti • 9061462



Neb 128B/Langer 5 Mini Ti • 9061463

Premier® Implant Scalers

Made from graphite reinforced resin that will not scratch implant surfaces.



137 Anterior • 9061421 (2-pack) • 9061420 (5-pack)



204 Posterior
• 9061423 (2-pack) • 9061422 (5-pack)



Facial (Goldman-Fox)
• 9061402 (2-pack) • 9061403 (5-pack)



Universal (4L/4R)
• 9061401 (2-pack) • 9061400 (5-pack)

Premier® PerioWise®

"The Friendly Probe"®

- ✓ Easy to read
- ✓ Safe around implants
- ✓ Autoclavable



Original 3-5-7-10mm • 9006107 (3-pack) • 9006102 (6-pack) • 9006105 (12-pack)

3-6-9-12mm • 9006108 (3-pack) • 9006104 (6-pack) • 9006106 (12-pack)

1mm increments
up to 12mm

UNC 12 • 9006120 (3-pack) • 9006121 (6-pack) • 9006122 (12-pack)



Scan to view full
line of PremierAir®
instruments!



Composite Manipulation (CMI)



Amalgam Carrier



Spot-On™ Etch

37% Phosphoric Etch Gel

Spot-On™ Etch is a 37% high viscosity phosphoric etch gel for all your routine and selective-etch procedures. It's designed with enhanced viscosity that will prevent it from dripping or slumping. Spot-On Etch easily flows through a small applicator tip, washes off readily and resists drying out.

- ✓ Ideal for routine etching and selective-etch procedures
- ✓ Blue color provides a strong contrast to enamel and dentin
- ✓ No slump formula
- ✓ Easy handling
- ✓ Rinses clean without leaving residue



Spot-On™ Etch no-slump viscosity will stay put. Only etch the areas you desire!

Spot-On™ Etch clinical image courtesy of Shalom Mehler DMD, Teaneck, NJ.



- 3001460 Spot-On™ Etch Bulk
One 12g jumbo filled syringe with luer lock adapter
Ten 1.2ml empty syringes
Fifty 1.2ml disposable applicator tips
- 3001461 Spot-On™ Etch Intro
Five 1.2ml filled syringes
Twenty bent disposable applicator tips

Premier® Etch

37% Phosphoric Acid Dentin/ Enamel Etch Gel

Premier® Etch contains phosphoric acid for etching tooth enamel and dentin when preparing surfaces for the application of composites.




- 3001422 37% Dentin/Enamel Etch Kit
Contains: one 12g jumbo syringe, with luer lock adapter, ten empty 1.2ml syringes and 50 disposable applicator tips for the ten 1.2ml syringes


Stop Strip

Anterior Bands with
Integrated Stopper

- ✓ Turns a three-handed procedure into a two-handed one
- ✓ Light-cure Class III and IV restorations with ease

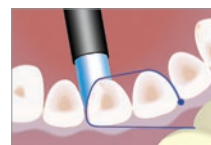

Unique integrated stopper
secures the matrix band
between adjacent teeth.

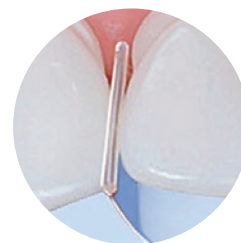


9061380 8mm (red) (100)

9061375 10mm (blue) (100)

Sycamore Wood Wedges

The Original Anatomically Carved

- ✓ Round tip easily passes over gingiva
- ✓ Concave sides conform matrix band to tooth's anatomy
- ✓ Square end is easily inserted and removed

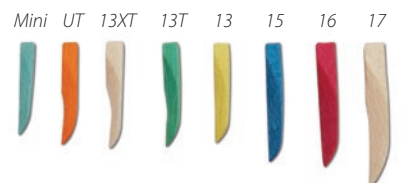

Wedge length in mm
(13=13mm)
Mini = 11mm
UT = 13mm

Eight color-coded sizes (400 count)

9061109 Mini (Light Blue)
9061106 UT - Ultra Thin (Orange)
9061100 13XT - Extra Thin (Neutral)
9061101 13T (Green)
9061102 13 (Yellow)
9061103 15 (Blue)
9061104 16 (Red)
9061105 17 (Neutral)
9061108 Assorted
(UT, 13, 13T & 15)

Cure-Thru® Clear Cervical Matrices

Transparent Pre-contoured Matrix Forms

- ✓ Flexes to the exact contour of the tooth for dense restorations with excellent marginal integrity
- ✓ Compresses restorative material as you light-cure
- ✓ Helps eliminate air inhibited layer
- ✓ Reduces time to final polish

Actual Size



719 720 721 722 723 Anterior/
Premolar Molar/
Universal

Available in boxes of 150

9061012 719CT
9061013 720CT
9061014 721CT
9061015 722CT
9061016 723CT
9061017 Anterior/Premolar
9061018 Molar/Universal
9061340 Positioning Instrument - double-ended, autoclavable
9061011 Assorted Box - Contains: 275 matrix forms, positioning instrument, gingival retraction instrument



Pre-Contoured Metal Bands

- ✓ Anatomical shape achieves better contact points
- ✓ Ideal for large Class II restorations with incremental curing technique
- ✓ Finished edges ensure patient comfort
- ✓ Measures 0.035mm thickness

9061070 Box of 30



BioGREEN™ Micro-Applicators

Bendable. Economical. Eco-Friendly.

- ✓ Available in regular, fine and super-fine sizes
- ✓ Eco-Friendly
- ✓ Gluten-free & Lint-free brush tips

3005100 Regular (green)

3005101 Fine (blue)

3005102 Super-Fine (yellow)

Contents 100 micro-applicators



ComfortView®

Lip & Cheek Retractor

- ✓ Retracts lips and cheeks for optimal buccal and gingival access
- ✓ Maximum visibility and patient comfort
- ✓ Suitable for use in diagnostic, preventative, therapeutic dental applications, in-office whitening, digital impression scanning and intra-oral photography
- ✓ Autoclavable, latex-free

9061381 Universal ComfortView Lip and Cheek Retractor, 2 per box (white)

9061383 Small ComfortView Lip and Cheek Retractor, 2 per box (blue)

9061382 ComfortView Refill Cushions, 10 per pack



premier Inspired solutions
for daily dentistry®



Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171 • premierdentalco.com • info@premierdentalco.com

©2026 Premier Dental Co. | All Rights Reserved

8610307 0526025 Rev17 PDF