AeroPro™ Barrier

Disposable Sleeve for use with AeroPro™ Cordless Prophy System

Directions for Use

Indications for use:

Premier® AeroPro™ Barrier sleeves are to serve as a protectant barrier for use with the AeroPro™ Cordless Prophy System. This device is non-sterile and is intended for single patient use only. The AeroPro Barrier sleeve is designed to protect the motor component from debris and bio burden.

Contraindications:

Single use only – discard after use

Precautions:

DO NOT install the outer sheath onto the motor component without using an AeroPro Barrier sleeve. ***The barrier sleeve is placed between the AeroPro motor component and the outer sheath. DO NOT place the barrier sleeve over the outer sheath. The outer sheath should be sterilized after each use.***

This polyethylene barrier does not replace cleaning and disinfection of the AeroPro system and its components. Follow the infection control procedures for each component after each patient as detailed in the AeroPro Cordless Prophy System's Directions For Use.

Directions for use:

1. Insert the AeroPro motor component into the AeroPro barrier sleeve. Ensure that the nose of the motor is all the way to the end of the barrier sleeve.
2. Place the AeroPro outer sheath over the barrier sleeve and onto the motor component and push together until locked into the proper positioning.

REF 5500530

Premier® Dental Products Company

1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.

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