

Routine Use

4 Pre-Operational Checks and Handpiece Assembly

Insert the charged Motor Component (A) into a custom-fitted AeroPro Disposable Barrier (F). Ensure that the nose of the Motor Component (A) is fully aligned with the opening of the disposable barrier.*



Motor Component (A) inserted into AeroPro™ Disposable Barrier (F).

Place autoclaved Outer Sheath (B) onto Motor Component (A) after covering with the Disposable Barrier (F).



Attach a 2pro® (G) or equivalent disposable prophylaxis angle (DPA) into the Outer Sheath (B) nose cone.




Shown - fully assembled CDC-compliant handpiece

*Use only AeroPro™ Disposable Barriers (F) for CDC-compliance

5 Operating the AeroPro™ Handpiece

To turn unit on:
Once assembled: Press the ON/OFF/MODE button once quickly using your thumb or index finger. The DPA tip will begin to rotate at low speed of 500 rpm.

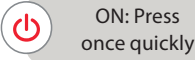


Use the low speed for dipping the DPA into the prophylaxis paste. Press the cup of the DPA into the prophylaxis paste, applying a downward force and light pressure. The low speed can also be used for very light stains and pedodontic patients.

Insert AeroPro™ into the patient's mouth and, if desired, change to high speed (2800 rpm) by pressing ON/OFF/MODE button once quickly and start polishing.




To turn handpiece off:
Press and hold ON/OFF/MODE button for 1 second.

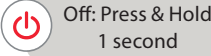

Note: The handpiece always returns to low speed upon powering on from off state.



After 20 continuous seconds of non-use, AeroPro™ will automatically enter the sleep mode phase and the indicator lights will turn off. The indicator lights will illuminate again once you press the ON/OFF/MODE button.

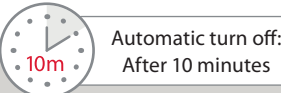

AeroPro™ will automatically turn off after 10 minutes of continuous use. Press ON/OFF/MODE button to restart.

6 After Treatment / Cleaning and Sterilization

Pull DPA straight off to remove and discard.



Disassemble the handpiece and follow Infection Prevention Procedures. (See Section 4.8 in Directions for Use)



Remove Outer Sheath (B) for sterilization by steam autoclaving. See 2 Autoclave. Autoclave AeroPro™ Stand (C) if used.



Remove the AeroPro Disposable Barrier (F) from the Motor Component (A) and discard the barrier.




Disinfect the Motor Component (A), Charging Station (D), and AC Adapter (E) using a CDC intermediate level disinfectant. See Step 2. (See Section 4.8.2 in Directions for Use for disinfection instructions.)



Place disinfected Motor Component (A) back into the Charging Station (D). See 3 Charge.



7 Changing Speed Modes

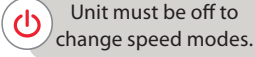

Factory setting: 2-speed mode (low to high speed).
(L → H → L → H)
If a greater range of speeds is preferred, the 3-speed mode can be selected by following the instructions below.

3-speed

*Beep 3x



To change the speed mode from 2-speed to 3-speed, press and hold ON/OFF/MODE button for 3 seconds. Three green indicator lights will flash and beep three times. AeroPro™ is switched to 3-speed mode.

2-speed

*Beep 2x



To change the speed mode from 3-speed to 2-speed, press and hold ON/OFF/MODE button for 3 seconds. Two top green indicator lights flash and beep two times. AeroPro™ is switched to 2-speed mode.

Speeds and Speed Modes

Operating Speeds	2-Speed Mode (factory setting)	3-Speed Mode
Low	500 rpm	500 rpm
Medium	—	1500 rpm
High	2800 rpm	2800 rpm

When operating in 3-speed mode, switch speeds by quickly pressing ON/OFF/MODE button.
(L → M → H → L)



The selected speed mode will be saved until a new speed mode selection is made. (See Section 4.5 in Directions for Use.)

Quick Start Guide

AeroPro™
Cordless Prophylaxis System

Please read carefully. Refer to the Directions for Use manual for complete product details.



Get Started!

Initial Use: Follow steps 1–3.



Routine Use: Follow steps 4–7.





- A Motor Component
- B Autoclavable Outer Sheath
- C Stand
- D Charging Station
- E AC Adapter
- F AeroPro™ Disposable Barriers
- G 2pro® Disposable Propphy Angles
- H Replacement O-rings and Tool for Maintenance

Initial Use

1 Unlock






AeroPro™ is shipped in a “Locked” position and must be “Unlocked” prior to initial use.

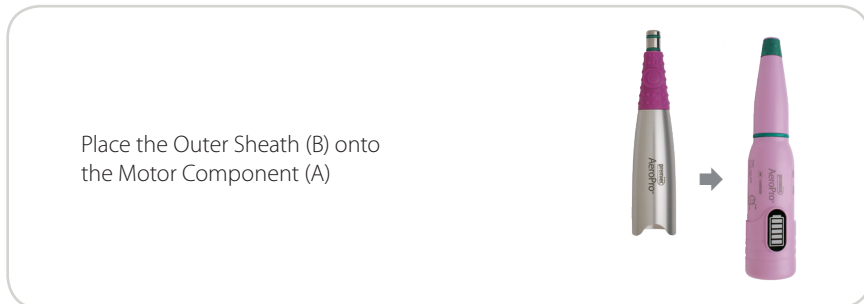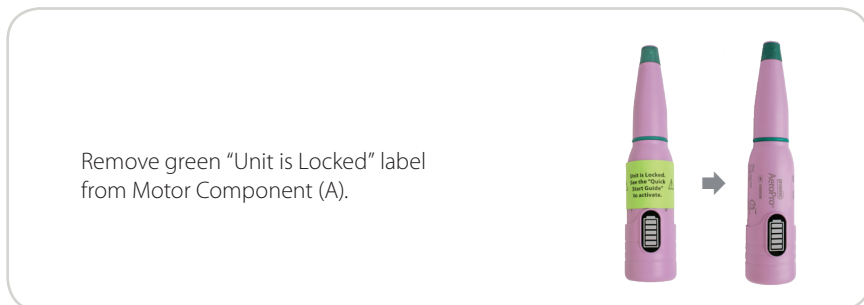

Press & Hold
Press and hold the ON/OFF/MODE button for 3 seconds.

Listen & Look
AeroPro will beep 6 times and all the indicator lights will flash simultaneously 6 times to indicate the system is unlocked.

2 Autoclave

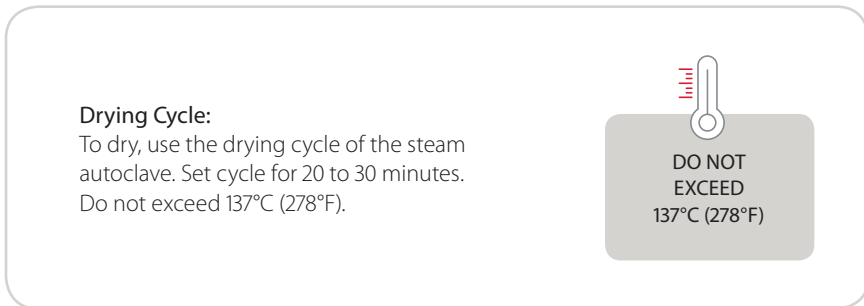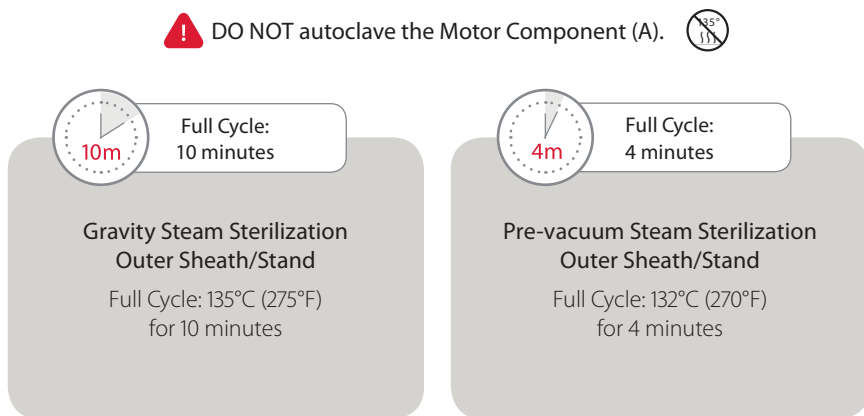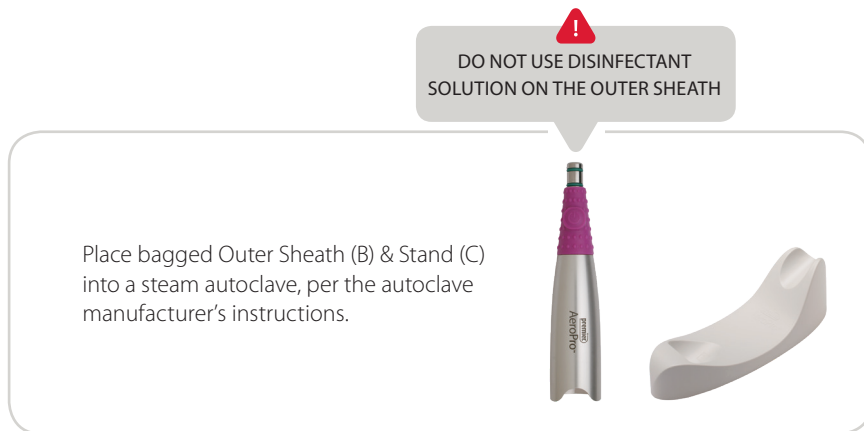




For non-autoclavable components

The Motor Component (A), Charging Station (D), and AC Adapter (E) are **not sterilizable by autoclave**. Use only water-based non-immersion disinfectant with a CDC intermediate level rating (i.e. chlorine disinfectant wipes with a tuberculocidal claim).

3 Charge



Charging AeroPro™ for the first time:

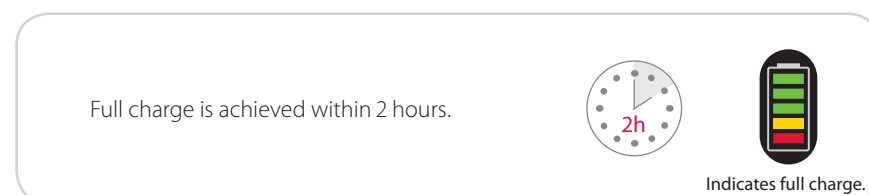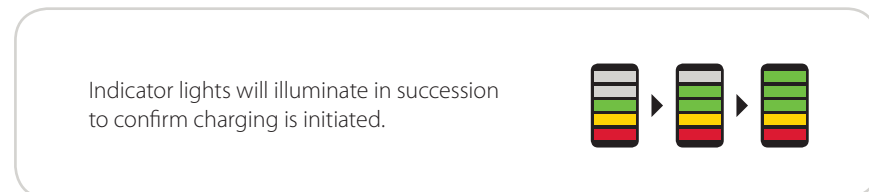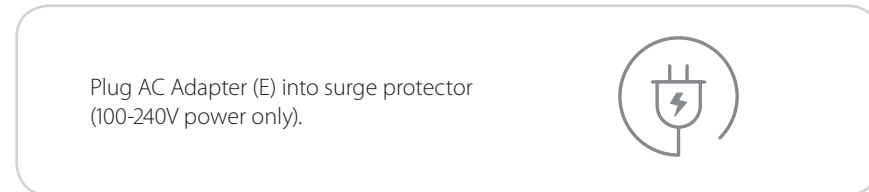

AeroPro features ChargeSMART™ Technology
Engineered to optimize battery life so you don't have to worry about running out of power during the work day.
See Section 4.6 in *Directions for Use* or learn more at premierdentalco.com

For instructions on operating AeroPro, continue to Step 4 on reverse side.

Basic Kit #5500500	Complete Package #5500510
1 Motor Component	1 Motor Component
1 Autoclavable Outer Sheath	3 Autoclavable Outer Sheaths
1 Charging Station	1 Charging Station
1 AC Adapter	1 AC Adapter
1 Stand	1 Stand
1 Box of 500 Disposable Barriers	1 Box of 500 Disposable Barriers
1 Bag of 48 Units 2pro® Disposable Propphy Angles	2 Bags of 144 Units 2pro® Disposable Propphy Angles
3 Extra Set of O-rings and Tool for Maintenance (6 Small O-rings for the Outer Sheath and 3 Large O-rings for Motor Component)	3 Extra Set of O-rings and Tool for Maintenance (6 Small O-rings for the Outer Sheath and 3 Large O-rings for Motor Component)

Accessories*

- 5500520 AeroPro™ Autoclavable Outer Sheath - Single
- 5500530 AeroPro™ Disposable Barriers - Box of 500

2pro® Disposable Propphy Angle		Bag of 144	Bulk Package (1,000 angles)
Soft/Short	Purple cup	5500101	5510001
Firm/Short	Blue cup	5500102	5510002
Soft/Long	White cup	5500103	5510003
Firm/Long	Green cup	5500104	5510004

*Order through your authorized dealer.

Warranty Registration

Please register your AeroPro Cordless Propphy System and retain your dealer invoice. This will ensure prompt and accurate service should your AeroPro Cordless Propphy System require evaluation, repair or replacement during the applicable warranty period, and will enable Premier to send you important product information.

Register online at: premierdentalco.com/aeroprowarrantyregistration

premier Inspired solutions
for daily dentistry®



Manufacturer:
Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171
premierdentalco.com • info@premierdentalco.com

Made in PRC

DFU# 55506 0118041 Rev0 NKP