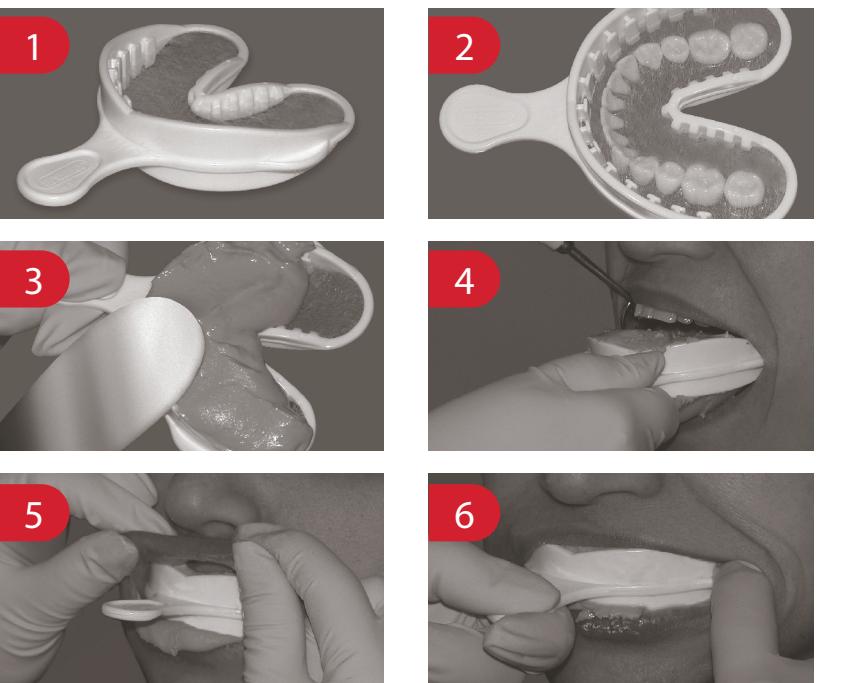
ALFA™ TRIPLE TRAY®

INDICATIONS: All alginate preliminary impressions - ideal for fabrication of custom trays, tooth whitening trays, temporaries, study models, mouth guards/night guards, removable prosthetics and orthodontics. Alfa Triple Tray can also be used with alginate substitute vinyl polysiloxane (VPS. material). Do not use a rigid setting or heavy body VPS material.

CONTRAINDICATIONS: Should not be used on edentulous patients. Do not use a rigid setting or heavy body VPS material. Block out any undercuts and do not use if the patient has loose restorations of any kind.

DIRECTIONS FOR USE

1. Select appropriate size tray. Medium fits most adult patients. Premier logo on top of handle ensures proper maxillary/mandibular orientation.
2. Try in selected tray by inserting into the patient's mouth. It should fit passively, with the crossbar at the back of the tray distal to the most posterior teeth in arch. As with any impression making technique, ensure patient can breathe comfortably through nose with mouth closed.
3. Prepare 2-3 scoops of alginate, depending on tray size (two for the small, three for medium or large). Load alginate into tray along outer walls to ensure it flows into retentive undercuts. Remove excess; alginate should extend 1-2mm just beyond tray wall. It is recommended that the maxillary anterior section be slightly overfilled to record the buccal vestibule.
4. Place loaded tray into the mouth. Retract the cheek with a dental mirror to provide easier access. The tray should be centrally placed and not impinge on tissue. Have the patient place their tongue into the posterior notch of the tray.
5. If necessary, assist the patient by pulling their lips beyond and over tray wall as they close into a natural bite.
6. Remove the set impression alginate impression by placing your finger on either purchase point and pressing down and out as you ask the patient to open. Once the suction is broken, the tray will be easily removed. Disinfect and inspect the impression. Pour the stone as per the manufacturer's directions.

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Alfa™ Triple Tray®

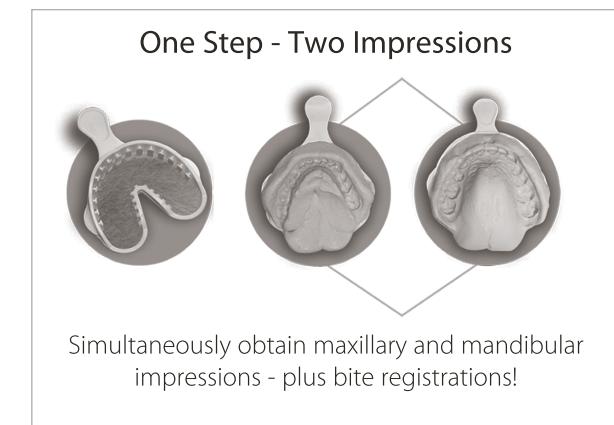
Alginate Full-Arch Closed-Mouth Impression Tray

TO SIMULTANEOUSLY OBTAIN

THE MAXILLARY IMPRESSION,

MANDIBULAR IMPRESSION

AND BITE REGISTRATION.

premier Inspired solutions
for daily dentistry®

Manufacturer:
Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
888-670-6100 • 610-239-6000 • www.premusa.com

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U.S. Pat. Nos. D464,138;
D413,386; D403,771

US/UK: LAB TECHNIQUES

1. Mix the stone and pour up one arch, making sure that no bubbles are incorporated into the stone. If needed, apply tray to vibrator to eliminate bubbles from stone.

2. Wrap the tray in a damp towel to preserve the integrity of the alginate while mixing stone for the second arch.

3. Once the first side is set, pour the second arch without removing the first arch from the tray. If needed, apply tray to vibrator to eliminate bubbles from stone. Allow stone to set.

4. Remove tooth models from Alfa Triple Tray. Trim as needed.

SUGGESTIONS: Consider using cold water when mixing alginate to increase working time. Apply a separating medium to the impression to facilitate easier removal of models from tray. Use model formers to create a model base. To save time, use a fast-set stone when pouring up the models.

If the stone from the second pour wraps around the tray and joins the first pour, separate the two arches using a lab bur/diamond to remove them from the tray.

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