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#### Warranty:

Premier® ultrasonic inserts are warranted for 90 days against manufacturer's defect.

#### Big Easy® Ultrasonic Inserts

REF	1005300	#10 Universal 25K
REF	1005310	#10 Universal 30K
REF	1005305	#1000 Triple Bend 25K
REF	1005315	#1000 Triple Bend 30K
REF	1005320	#100 Thin 25k
REF	1005330	#100 Thin 30K

#### Resin Grip Inserts

REF	1006800	#10 Universal 25K
REF	1006900	#10 Universal 30K
REF	1006802	#1000 Triple Bend 25K
REF	1006901	#1000 Triple Bend 30K
REF	1006820	#100 Thin 25K
REF	1006920	#100 Thin 30K

Made in the U.S.A.

DFU10536 1119001 Rev4 PMM

Made in the U.S.A.

**premier**  
Manufacturer:  
Premier® Dental Products Company  
710 Romano Drive,  
Plymouth Meeting, PA 19462, U.S.A.  
610-239-6000  
premierdentalco.com

**premier**

## Big Easy® Implant Ultrasonic Insert

REF 1005336 30K

For use in  
magnetostrictive  
ultrasonic scalers.

USE ON LOW TO  
MEDIUM POWER ONLY

For intended use by dental  
professionals only.

## US/UK

### For use in magnetostrictive ultrasonic units

#### Indications:

The ultrasonic scaler and inserts are intended for use by medical professionals during dental cleaning and periodontal therapy to remove calculus and tartar deposits and stains from teeth / prosthesis using the application of an ultrasonically vibrating tip.

#### Contraindications:

Use of any ultrasonic device is not recommended in cases where the operator or patient has been fitted with a cardiac pacemaker.

#### Caution

- Inserts are intended for dental hygiene (cleaning) of dental implants and prostheses only as a preventive procedure
- Not intended to be used as tool to assist other restorative, surgical, operative, or similar dental procedures
- Discard insert if excessive wear or metal show-through is observed around the PEEK tip.
- Rx Only

#### PEEK tip

The yellow PEEK tip is permanently attached to the metal connector body with a specific torque. Do not attempt to remove the tip. PEEK is a biomaterial polymer used in medical devices. It can be repeatedly autoclaved, is fracture resistant and will not scratch implants.

#### Cleaning/ Sterilization:

- Inspect, clean, and sterilize prior to each use. Do not use chemical disinfectants prior to sterilization or leave residual cleaning solutions on insert. Carefully remove gross debris, rinse and dry inserts completely prior to sterilization. *Premier® Dental has independently validated their instruments to be sterile when using a pulsed pre-vacuum steam sterilization process for five minutes at 134°C (273°F).*
- Do not use Rapid Heat Transfer (dry heat) sterilization method.
  - Do not expose inserts to temperatures above 350°F/177°C or to chemical vapor concentrations exceeding manufacturer's recommendations.
  - Do not use phenols.
  - Do not use a bleaching agent.
  - Do not use cold disinfectant/cold sterilization solutions.
  - Do not sterilize in 100% plastic sterilization pouches. (A combination paper/plastic pouch is acceptable, paper side down.)

#### Optimum Performance:

- Insert stacks should be straight and parallel at all times. Bent or warped inserts should be repaired or discarded.
- Worn tips prevent effective operation of the ultrasonic insert and are subject to breakage. Inserts should be replaced when tip-wear reduces size and clinical access from original condition.
- Water lines to scaler should be flushed and handpiece should be filled with water prior to use for most efficient results.
- The O-ring should be carefully maintained to ensure the longest possible life. Maintenance includes water lubrication of O-ring as well as proper seating of insert within the hand-piece.
- Damaged or broken (torn) O-rings should be replaced prior to use.

#### Warranty:

Premier® ultrasonic inserts are warranted for 90 days against manufacturer's defect.

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