BioCoat®

Bioactive Resin Pit and Fissure Sealant

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BioCoat® Bioactive Pit and Fissure Sealant, a prescription only medical device used for prophylactic sealing of pits and fissures. It may also be used for micro-restorative or as an "initial layer" for composite restorations.

Contraindications

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product with this patient.

Cautions:

- Keep out of reach of children.

Important Interactions and Side Effects

- No known systemic side effects.
- In rare cases components of BioCoat may cause sensitization. In case of sensitization stop using the product.

Note:

- Disposable tips are designed for single patient use only.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

Instructions for Use: Pit and Fissure Sealant:

1. Attach dispensing tip to BioCoat syringe.

NOTE: Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

- Clean fissures using oil-free pumice, air polishing prophylaxis system, or thin diamond fissure/prep bur.
- Isolate teeth being treated to avoid saliva contamination.
- Apply etch to fissures for 20 seconds. Thoroughly rinse and dry. NOTE: If using sodium bicarbonate in air polishing system, repeat with a 5 second etch, thoroughly rinse and dry.
- Upon initial use express a small drop of the BioCoat on a mixing pad prior to applying intra-orally.
- Using the syringe needle tip or a brush, slowly introduce sealant into the pits and fissures. Do not let sealant flow beyond the etched surfaces. Stirring/agitating the sealant with the syringe-tip during or immediately after placement will help eliminate any possible bubbles, and enhance the flow into the pit and fissures. An explorer may also be used. Avoid pooling of resin.
- Light cure with a Curing Light on Standard Power for 20 seconds for each surface (For lights with output 600mw/cm2 cure for 20 seconds). The tip of the light should be held as closely as possible to the sealant, without actually touching the sealant. When set, the sealant forms a hard, opaque film, is white in color with a slight surface inhibition
- Evaluate Sealant. Inspect sealant for complete coverage and voids. If surface has not been contaminated, additional sealant may be added. If contamination has occurred re-etch, rinse, and dry prior to placing more sealant.
- After confirming placement of the fully cured sealant, wipe the sealant with a cotton applicator to remove the thin sticky surface film. Check occlusion and adjust as required.
- Immediately after use, the dispensing tip should be removed, discarded, and the cap screwed back on to avoid prolonged air exposure.

Super Adaptive Initial Layer for Restorative:

- Apply dentin bonding agent according to manufacturer's instructions.
- Attach tip to BioCoat syringe.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.
- Verify flow prior to applying intra-orally.
- Apply first layer of BioCoat 0.5 – 1.5mm thick and light cure 20 sec.
- Follow with 2-4mm increments as needed.
- Light cure with a curing light on Standard Power for at least 20 seconds. For detailed anatomy, apply paste composite for final layer. Carefully shape restorations with finishing burs and polish.

- Storage and Use**
 - Avoid excess heat. Keep away from direct sunlight.
 - For optimum freshness, refrigeration is optional. Do not freeze. If refrigerated, allow the syringe to reach room temperature.
 - Use only as directed, per indications for use. Tightly recap after use.
 - To ensure optimal performance of the product, use immediately after dispensing and limit the amount of time of air exposure.
 - Disinfect this syringe/product using an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control. Guidelines for Infection Control in Dental Health-Care Settings – MMWR, December 19, 2003:52(RR-17), Centers for Disease Control and Prevention.
 - Do not use after expiration date. See expiry date on the label.
 - Shelf life is 2 years after the date of manufacturing.

To obtain SDS visit premierdentalco.com or call Premier at 610-239-6000.

Disposal

See the Safety Data Sheet (available at premierdentalco.com or through your local subsidiary)

Made in Israel

Premier Dental
1710 Romano Drive Plymouth Meeting, PA 19462
610-239-6000 premierdentalco.com

Warranty and Limitation of Liability

Premier Dental Products Company ("Company") warrants that the Product will be free from defects in material and manufacture. Company makes no other warranties. In particular, and without limitation, Company makes no warranty, implied or otherwise, as to the Product's merchantability or fitness for any particular purpose.

For any patient on whom use of the Product is contemplated, user is solely responsible for determining whether the Product is suitable for such use according to the information set forth in the enclosed directions for use.

If the Product is defective within the warranty period, the sole remedy available to user, and the sole obligation of Company, shall be the Product's repair or replacement. Unless otherwise required by law, Company will not be liable for any loss or damage arising from this Product or its use, whether such loss or damage is direct, indirect, special, incidental or consequential, and regardless of the theory of loss or damage asserted (which theories include, without limitation, warranty, contract, negligence and strict liability). Unless otherwise required by law, no party other than the Company is authorized to provide to any other party any information concerning the Product which deviates from the information set forth in the enclosed directions for use.

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