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Distributed by:

Premier[®] Dental Products Company

1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.

888-670-6100 • 610-239-6000 • www.premusa.com

DFU#40083 0617063 Rev3 BJM

Perfecta[®] Block-Out Light-Cure Block-Out Resin

Use: To create optional gel reservoirs in the bleaching tray, apply Perfecta[®] Block-Out Resin to the facial surfaces of selected teeth of the model.

To Apply: After the model is dry, apply an even coat (approximately 0.5mm thick) of block-out resin to the selected facial surface(s). Stay approximately 1.5mm from the gingival margin. Do not apply to incisal edges or occlusal surfaces. Discard applicator tip after use.

Cure: Using an intraoral curing light, cure each tooth for 20 seconds or until hard. A light-cure box may be used to quicken the process. As with all Bis-GMA resins, this material will have a thin oil film after curing. This is the oxygen-inhibited layer and may be wiped or washed off.

Warning: Appropriate personal protective equipment should be used at all times. If accidental exposure to eyes occurs, rinse exposed eye(s) for 15 minutes.

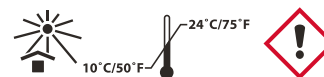
Contents: Two 3cc syringes of light-cure block-out resin and 20 applicator tips.

To obtain an SDS, visit Premier[®] Dental at www.premusa.com or call 610-239-6000.

REF 4008300 Perfecta[®] Block-Out
Contains two 3cc syringes
of light-cure block-out resin
and 20 applicator tips.

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