CompCore™ AF Stack

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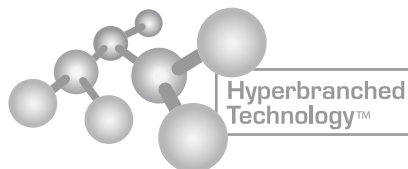
Manufacturer:
Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
(888) 670-6100 • (610) 239-6000 • www.premusa.com

To obtain an SDS visit www.premusa.com
or call Premier at 610-239-6000.

Made in Israel.

DFU# 30107 • 1118070 Rev8 BJM

US / UK

CompCore™ AF Stack

Dual-Cure Composite Core Build-Up Material

Hyperbranched Technology™ • Fluoride releasing • Radiopaque

Indications for Use:

CompCore™ AF is an automix, fluoride releasing, resin composite material designed for the direct fabrication of core build-ups.

Dual-Cure Physical Properties:

Light-cure setting times:

Working time: 2.5 minutes at 72°F (22°C).
Higher temperatures accelerate set.

Final set: 30 seconds if depth of core is less than 8mm - a minimum of 40 seconds if depth is greater than 8mm.

Chemical-cure setting times:

Working time: 2.5 minutes at 72°F (22°C).
Higher temperatures accelerate set.

Final set: 4.5 minutes at 98.6°F (37°C).
Higher temperatures accelerate set.

Precautions:

- Disposable tips are designed for single patient only.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.
- Eugenol-containing materials could inhibit the polymerization of CompCore AF.
- Placement of CompCore AF is contraindicated if a dry field is not possible.
- Caution- U.S. Law restricts this device for sale by or on order of a dental professional.

Directions for Use: Bonding Agent Instructions

If you are using another manufacturer's bonding agent system follow manufacturer's instructions for use with dual-cure composites.

1. Clean all freshly instrumented enamel and dentin with a spray of oil-free water followed by a spray of oil free air to dry surface. Be careful not to desiccate the tooth.
2. Isolate the tooth from oral fluids. A rubber dam is recommended.
3. Acid etch the entire cavo surface for 10 to 15 seconds with Premier Etch, 37% phosphoric acid.
4. Thoroughly rinse the cavo surface for at least 20 seconds with spray of oil-free water followed by a spray of oil free air to dry surface. Be careful not to desiccate the tooth.
5. If using IntegraBond™ bonding agent, apply first coat on prepared cavo surface. Allow IntegraBond to penetrate the prepared surface for 15 seconds then dry with a full blast of oil-free air. Apply an additional 2 to 3 more coats of IntegraBond; dry with a full blast of oil-free air after the final application. Look for light reflection (shiny surface) from the bonded surface as an indication of sufficient coating. If necessary, apply additional coats of IntegraBond until light-reflection is achieved.
6. Light-cure IntegraBond for 20 seconds after final application. Follow respective manufacturer directions for light-curing if using another bonding agent.

Directions for Use: Core Build-Up Instructions

1. Select CompCore AF shade and assemble dispenser (cartridge or syringe). Graphic assembly instructions for both the gun and cartridge unit or syringe are enclosed separately with this package. Bleed cartridge or syringe on initial use and bleed through assembled mix-tip and intraoral tip before each use.
2. Place intraoral tip into the base of the prepared tooth and dispense an adequate amount of material to complete build-up. Let material rise around the tip while withdrawing to eliminate air bubbles. Premier Accor® crown-form matrices or Cure-Thru® matrix bands can be used to confine the material and define the build-up.

Light-Cure Directions:

1. Place the curing-light (Halogen or LED) close to the surface of the core build-up and polymerize for at least 30 seconds. If the core has a depth greater than 8mm, a minimum of 40 seconds is recommended.
2. Check for complete hardness. The preparation can be finished immediately with a Two Stripper® diamond or other bur.

Chemical-Cure Directions:

1. Do not disturb CompCore AF after the placement or during the gelation stage. Chemical-cure intraoral set-time is approximately 4.5 minutes from start of mix/application. Check for complete hardness. Set time takes longer outside the mouth. Optionally, light-cure for 20 seconds after application to stabilize the core material and to accelerate chemical-cure process.
2. Finish preparation with a Two Stripper® diamond or other bur.

Contraindications:

Do not use on patients with known sensitivity or allergy to resins. Wash exposed area completely to remove material. If an allergic reaction is observed, consult a physician immediately and discontinue use with this patient.

Storage:

Do not store CompCore™ AF over 75°F (24°C); Refrigerate for maximum shelf-life. Allow material to warm to room temperature before use. Do not freeze.

Shelf life: 2 years when stored at room temperature 75°F (24°C)
To obtain an SDS, visit Premier Dental at www.premusa.com or call 610-239-6000.

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