# premier® Cure-Shield

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**premier®** Inspired solutions  
for daily dentistry®

Manufacturer:

Premier® Dental Products Company  
1710 Romano Drive  
Plymouth Meeting, PA 19462, USA  
610-239-6000 • [www.premusa.com](http://www.premusa.com)

\* References dealing with ultraviolet and blue light exposure. / 1. Clinical Research Associates Newsletter - Vol. 8, Issue 2, February, 1984 / 2. Ocular Hazard of Light Sources: Review of Current Knowledge, Journal of Occupational Medicine / 3. U.S. Army Environmental Hygiene Agency: Standards for Use of Visible and Non-visible Radiation on the Eye, 1983

**US** - Cure-Shield effectively blocks out potentially harmful\* ultraviolet and blue light at wavelengths shorter than 525 nanometers which, after prolonged exposure, may cause photochemical burn of the retina. Cure-Shield gives you that extra margin of safety.

#### **Directions for Use:**

- Premier® Cure-Shield is held to shield eyes from direct or reflected visible light beam.
- Can be held by assistant, patient or the operator.
- Position the Premier® Cure-Shield over the patient's mouth to protect the dentist and assistant simultaneously.
- Premier® Cure-Shield *should be cleaned with soap and water and dried with a soft cloth.* Avoid scratching the shield with abrasive materials. Can be sterilized in cold solutions.
- Scratches can be removed with fine flour of pumice followed with a final polishing material.

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