# premier® Enamel Pro® Varnish

EN	5% Sodium Fluoride Varnish - ACP formula
FR	Vernis de fluorure de sodium 5 <span> </span> % - Formule ACP
IT	Smalto a base di fluoruro di sodio al 5% - Formula ACP
ES	Barniz de fluoruro sódico al 5% - Fórmula del ACP
NL	5% natriumfluoridelak - ACP-formule (amorf calciumfosfaat)
DE	Natriumfluoridlack 5% - ACP-Formel
CZ	Umělá sklovina s 5 <span> </span> % fluoridu sodného - S použitím technologie ACP (Amorphous Calcium Phosphate)
FI	5 <span> </span> % natriumfluoridilakka - ACP-koostumus
PL	Lakier z 5% zawartością fluoru sodu - Preparat oparty na technologii amorficznego fosforanu wapnia
LT	5% natrio fluorido lakas - ACP receptūra
SE	5 <span> </span> % natriumfluoridlack - ACP-formel
HU	5%-os nátrium-fluoridos máz - ACP formula
PT	Verniz com 5% de fluoreto de sódio - Fórmula ACP
GR	Βερνίκι φθοριούχου νατρίου 5% - Σύνθεση ACP
TR	%5 Sodyum Floridli Diş Glasi - ACP formül
KO	5% <b>플루오르화 나트륨 바니시</b> - ACP 처방서

<b>premier®</b>	Inspired solutions for dental dentistry- Premier® Dental Products Company
	1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A. (888) 670-6100 • (610) 239-6000 www.premusa.com

<b>Made in U. S. A.</b>	<b>Rx ONLY</b>
To obtain a SDS visit www.premusa.com or call Premier at 610-239-6000.	
31926	DFLX 33011 101014 Rev2 MPL

<b>US / UK</b>	
----------------	--

**Premier® Enamel Pro® Varnish 5% Sodium Fluoride Varnish - ACP formula**
**Description:** Enamel Pro Varnish is a 5% sodium fluoride varnish which has a strong desensitizing action when applied to enamel and dentin surfaces. Enamel Pro Varnish may be used on moist teeth and leaves a film of varnish which sets in the mouth to allow for visual control and verification of application.

**Indications for Use:** Enamel Pro Varnish is a fluoride-containing preparation for the treatment of partial hypersensitivity and for the reduction of post-operative sensitivity. The varnish can be placed on sensitive root surfaces and under temporary restoratives and cements in order to seal exposed dentinal tissue.

**Dosage:** For the treatment of dentin hypersensitivity, in most cases, two applications at several days' interval are sufficient for effective treatment. The fluoride content of Enamel Pro Varnish is dosed in such a way that neither acute nor chronic side effects are to be expected if applied according to instructions.

**Directions for Use: Please observe when treating hypersensitive teeth.**

1. A "pre-treatment" prophylaxis cleaning and polishing are not necessary before varnish application; however, treated area should be free of excessive plaque build-up.

2. Peel off the foil top of the unit-dose container to expose the varnish and applicator brush. Stir varnish with applicator brush. **Mix thoroughly as some phase separation of varnish components may occur during storage.**

3. Apply a very thin coat of varnish evenly to the surface being treated.

4. Enamel Pro Varnish hardens on contact with saliva and/or water. The patient may leave immediately following application of the product.

5. Instruct patient to avoid hard food and hot beverages for the next 2 hours and avoid brushing and flossing for 4 hours after application.

**Note:** Enamel Pro Varnish also covers moist teeth with a coating of protective film for several hours; however, it is best to avoid applying the varnish over excessive saliva. It is recommended that patient be instructed to eat only soft foods for two hours after treatment with varnish. Longer contact time provides greater protection. Patient should avoid brushing the treated teeth for four hours after application. Alcoholic beverages, mouthwashes or rinses containing alcohol can accelerate the dissolution of the varnish and also should be avoided for at least four hours after application.

**Contraindications:** Ulcerative gingivitis, stomatitis, known sensitivity to colophony (kolophonium), rosin or similar agents. Direct ingestion should be avoided. Not for systemic treatment.

**Interactions:** When Enamel Pro Varnish is applied, other fluoride preparations such as fluoride gels should not be administered during the same day. The routine use of

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