premier® Enamel Pro®

IT

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Premier Dental®

Made in U.S.A.

Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
(610) 239-6000 • www.premusa.com

Medical Products Laboratories, Inc., Philadelphia, PA U.S.A.

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premier® Enamel Pro®

US/UK Professional Prophylaxis Paste (Amorphous Calcium Phosphate)

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US/UK

These instructions cover all Enamel Pro Prophylaxis Paste flavors and grinds.

Indications for Use

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. Enamel Pro pastes containing fluoride are not considered a replacement for other forms of fluoride therapy. Enamel Pro is for use only by individuals trained to perform a dental prophylaxis. In packages indicating that Enamel Pro contains fluoride, it is present at a concentration of 1.23% fluoride ion.

Precautions

Excessive ingestion of fluoride can cause nausea. Enamel Pro pastes containing fluoride should be expectorated and NOT SWALLOWED.

Enamel Pro Professional Prophylaxis Paste is available in the following grits:

Fine – For gentle polishing. Ideal for younger patients.

Medium – For normal adult cleaning procedures where a high level of polish and plaque removal is desired.

Coarse – For adult patients with substantial stain build-up.

Extra Coarse – For those select patients who have stain build-up difficult to remove.

Directions for Use

Unit dose cups: Each cup contains enough Enamel Pro paste for one (1) prophylaxis procedure.

Using any commercially available prophylaxis angle-rubber cup combination, fill the cup by pressing the face of the cup into the paste while running the angle at low speed. The "Swirl" design is for aesthetic purposes only. All contents in the cup will provide the same benefits to your patient's teeth. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

Caution

U.S. Federal Law restricts this device to sale by or on the order of a dentist.

Store at room temperature. Keep out of direct sunlight. Do not use after expiration date.

Warranty and Limitation of Liability

Premier Dental Products Company ("Company") warrants that the Product will be free from defects in material and manufacture. Company makes no other warranties. In particular, and without limitation, Company makes no warranty, implied or otherwise, as to the Product's merchantability or fitness for any particular purpose.

For any patient on whom use of the Product is contemplated, user is solely responsible for determining whether the Product is suitable for such use according to the information set forth in the enclosed directions for use.

If the Product is defective within the warranty period, the sole remedy available to user, and the sole obligation of Company, shall be the Product's repair or replacement.

Unless otherwise required by law, Company will not be liable for any loss or damage arising from this Product or its use, whether such loss or damage is direct, indirect, special, incidental or consequential, and regardless of the theory of loss or damage asserted (which theories include, without limitation, warranty, contract, negligence and strict liability). Unless otherwise required by law, no party other than the Company is authorized to provide to any other party any information concerning the Product which deviates from the information set forth in the enclosed directions for use.

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