US / UK

Buffered Hemostatic Solution

For Topical Application – Contains no Epinephrine

INDICATIONS

Hemodent has been specifically formulated to provide gingival retraction and hemostasis. It is intended for use in restorative and operative dentistry, optionally with gingival retraction cord, prior to taking an impression, cementation, cavity preparation or wherever hemostasis and retraction is required. Helps facilitate the insertion of the cord into the sulcus.

DIRECTIONS FOR USE

- 1) Soak a cotton-tipped applicator or pledget of cotton with Hemodent.
- 2) Apply to bleeding area with pressure until hemorrhaging is arrested. Should a small dark coagulum appear, flush it away with water. This will have no effect on hemostatic action.
- 3) When hemostasis has been achieved, remove Hemodent by thoroughly rinsing using an air/water spray to clean and dry preparation.
- 4) Check for hemostasis. If bleeding occurs, repeat step 2. Recheck with air/water spray until bleeding has stopped.

IMPRESSIONS

- 1) Displace tissue by packing size-appropriate Knit-Pak™ retraction cord or other displacement cord into sulcus.
- 2) For optimum displacement and hemostasis, place a small amount of Hemodent in a dappen dish and soak cord prior to packing.
- 3) Wait 1-3 minutes before removing Hemodent-impregnated cord.
- 4) Remove cord, rinse with a firm air/water spray, check for hemostasis, and make impression
- 5) Examine treatment site to verify complete removal prior to taking the impression.

CONTRAINDICATIONS:

- Do not use on persons with known allergy to aluminum chloride
- Do not use in patients with significant periodontal disease or furcation involvement.

PRECAUTIONS:

- Be sure all solution is removed before taking impression to avoid inhibition of polymerization of impression material.
- To avoid cross contamination, use new cotton-tipped applicators and pledget of cotton for additional volumes.

STORAGE: Store at room temperature <28°C / 82°F.

To obtain a SDS, visit Premier Dental at www.premusa.com or call 610-239-6000.

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