# IntegraBond™

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To obtain an SDS visit premierdentalco.com or call Premier at 610-239-6000.

Rx ONLY

24°C/75°F

Made in Israel.

DFU# 33011 • 0619074 Rev7 BJM

US / UK

## IntegraBond™ Bonding Agent

Universal Light-Cure Bonding Agent

### Indications for Use:

IntegraBond™ is a single-step, universal bonding agent indicated for all direct composite / compomer restorations including amalgam bonding, all indirect restorations including cementation of crown and bridge, inlays and onlays, posts and veneers. (indirect restorations that require use with a luting resin such as IntegraCem™)

### Note:

When applying multiple coats of bonding agent, it is only necessary to wait for 15 seconds after the initial application of the adhesive. Subsequent applications require solvent evaporation but can be applied in turn without waiting.

### Precaution:

Caution- U.S. Law restricts this device for sale by or on order of a dental professional.

### Instructions for Use

1. Clean all freshly instrumented enamel and dentin with a spray of oil-free water followed by a spray of oil free air to dry surface. Be careful not to desiccate the tooth.

2. Isolate the tooth from oral fluids. A rubber dam is recommended.

3. Acid etch the instrumented enamel and dentin for 10 to 15 seconds with Premier Etch, 37% phosphoric acid. Caution should be used when etching thin dentin.

4. Thoroughly rinse the tooth surface for at least 20 seconds with a spray of oil-free water followed by a spray of oil free air to dry surface. Be careful not to desiccate the tooth.

5. Dispense IntegraBond into a dappen dish or directly onto a microbrush. Apply first coat on prepared tooth surface. Allow IntegraBond to penetrate prepared enamel and dentin for 15 seconds then dry with a full blast of oil-free air. Apply an additional 2 to 3 more coats of IntegraBond; dry with a full blast of oil-free air after the final application. Look for light-reflection (shiny surface) from the bonded surface as an indication of sufficient coating. If necessary, apply additional coats of IntegraBond until light-reflection is achieved.

6. Light-cure IntegraBond for 20 seconds after final application. Compatible with all LED, and halogen curing lights.

### When Light-Curing is Not an Option: Endodontic Post Cementation

In step 5, add equal drops of Auto-cure Activator with IntegraBond in a clean mixing well / dappen dish and combine for 2 seconds. Continue with step 5 using the IntegraBond/Activator mixture. Use a paper point to wet the deepest portions of the canal. Avoid pooling of the adhesive. Proceed to post cementation.

### Note:

The use of IntegraBond with Auto-cure Activator can accelerate the working time of self- and dual-cure cements.

### Contraindications:

Do not use on patients with known sensitivity or allergy to resins. Wash exposed area completely to remove material. If an allergic reaction is observed, consult a physician immediately and discontinue use with this patient.

### Storage:

Do not store IntegraBond over 75°F (24°C). Refrigerate for maximum shelf-life. Allow material to warm to room temperature before use.

Shelf life: 2 years when stored at room temperature 75°F (24°C)

To obtain an SDS, visit Premier Dental at premierdentalco.com or call 610-239-6000.

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