EN	Impregnated Knitted Cord	IT	Filo a maglia impregnato
DE	Imprägnierter Strickfaden	GR	Εμποτισμένο πλεκτό νήμα
FR	Fil tricoté imprégné	LT	Impregnuota nerta virvutė
ES	Hilo tejido impregnado	PL	Pleciona nitrakcyjna impregnowana

EN

## Indications

A prescription-only, Aluminum Chloride Hexahydrate impregnated cord is used for temporary chemo-mechanical retraction of gingival tissues and hemostasis of the gingival margin around the tooth. It is intended for accurate and enhanced results when acquiring dental impressions, for cavity preparation or wherever hemostasis and retraction is required.

## Contraindications

- Do not use on persons with known allergy to aluminum chloride
- Do not use in patients with significant periodontal disease or furcation involvement.
- Do not leave the cord in the gum for more than 20 minutes as this may cause a lasting recession of the gum.

## Caution

Reuse of single-use medical devices may result in exposing your patient(s) to cross-contamination / cross-infection.

- Firmly close the cap to cut a length of cord that will equal or exceed the circumference of the tooth.
- Starting mid-buccal or mid-lingual, use a cord packer to place one end of the cord into the sulcus.
- Continue packing until the entire cord is placed within the sulcus, around the tooth.
- Leave the cord in place for 3-5 minutes.
- Cord may be removed prior to impression or left in the sulcus during the impression if placed below the finish line.
- In all cases, retraction cord must be removed before completion of the procedure

## Warranty and Limitation of Liability

Premier Dental Products Company ("Company") warrants that the Product will be free from defects in material and manufacture. Company makes no other warranties. In particular, and without limitation, Company makes no warranty, implied or otherwise, as to the Product's merchantability or fitness for any particular purpose.

For any patient on whom use of the Product is contemplated, user is solely responsible for determining whether the Product is suitable for such use according to the information set forth in the enclosed directions for use.

If the Product is defective within the warranty period, the sole remedy available to user, and the sole obligation of Company, shall be the Product's repair or replacement.

Unless otherwise required by law, Company will not be liable for any loss or damage arising from this Product or its use, whether such loss or damage is direct, indirect, special, incidental or consequential, and regardless of the theory of loss or damage asserted (which theories include, without limitation, warranty, contract, negligence and strict liability). Unless otherwise required by law, no party other than the Company is authorized to provide to any other party any information concerning the Product which deviates from the information set forth in the enclosed directions for use.

To obtain a SDS, visit Premier® Dental at [premierdentalco.com](http://premierdentalco.com) or call 610-239-6000.

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