NexTemp® Temporary Cement

Non-Eugenol, Resin Based Temporary Cement containing Fluoride, Chlorhexidine and Potassium Nitrate.

Indications for Use:
Premier® NexTemp® Temporary Cement is recommended for the cementation of temporary crowns, bridges, inlays and onlays.

Contraindications:
Premier® NexTemp® Temporary Cement is not intended as a temporary filling material nor is it to be used for pulp capping. Do not use on patients with known sensitivity or allergy to resins. Wash exposed area completely to remove material. If an allergic reaction is observed, consult a physician immediately and discontinue use with this patient.

Note:
• Disposable tips are designed for single patient use only.
• Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

Directions for Use:
1. Complete tooth preparation and fabrication of the temporary restoration before extruding the mixed cement.
2. Dry and isolate the prepared tooth. Be careful not to desiccate or over-dry the preparation.
3. Remove cap from double barrel automix syringe and set aside. Extrude a very small portion of the unmixed material to insure unobstructed flow from both the base and catalyst components of the respective barrels.
4. Attach automix dispensing tip and lock in place with a quarter turn.
5. Extrude a thin layer of the automixed cement on to the cleaned, inside surface of the temporary restoration. You have approximately 45 seconds of working time after extruding the mixed cement.
6. Firmly seat the temporary restoration in place. Initial gel-set occurs within 2 to 2.5 minutes after seating. Cleaning of excess cement should be initiated immediately after this gel-set phase.
7. Final cement set occurs between 4 and 5 minutes after seating of the temporary restoration. Final set time is determined intraorally. Ambient set times vary and are not representative of clinical conditions.
8. Remove used mixing tip from double barrel automix syringe and replace with original cap. If cap is misplaced, the used tip can be left in place until the next use.

Optional Hand-Mixing Directions:
1. Complete tooth preparation and fabrication of the temporary restoration before extruding the mixed cement.
2. Dry and isolate the prepared tooth. Be careful not to desiccate or over-dry the preparation.
3. Remove cap from double barrel automix syringe and set aside. Extrude a very small portion of the unmixed material to insure unobstructed flow from both the base and catalyst components of the respective barrels.
4. Place dispensing end of dual barrel syringe onto clean glass slab or clean mixing pad. Extrude the appropriate amount of base and catalyst onto the selected mixing surface. Be sure to avoid cross contaminating the openings of the respective base and catalyst barrels. Recap syringe with original closure.
5. Using a clean spatula or mixing stick, mix the two components into a homogenous paste.
6. Using the mixing stick or brush, apply a thin layer of the mixed cement on to the cleaned, inside surface of the temporary restoration. You have approximately 45 seconds of working time from start of mix.
7. Firmly seat the temporary restoration in place. Initial gel-set occurs within 2 to 2.5 minutes after seating. Cleaning of excess cement should be initiated immediately after this gel-set phase.
8. Final cement set occurs between 4 and 5 minutes after seating of the temporary restoration. Final set time is determined intraorally. Ambient set times vary and are not representative of clinical conditions.

Precautions: Though rare, some individuals may be sensitive to this product. If an allergic reaction occurs, discontinue use, remove from patient and refer patient to a physician.

Important Application Information: Only a small bead of NexTemp around the margin is necessary to securely hold your provisional. This is especially true with highly retentive temporaries. Using less cement saves money and allows for easy retrievals!

Eugenol-containing materials could inhibit the polymerization of NexTemp®.

To obtain an SDS, visit Premier Dental® at premierdentalco.com or call 610-239-6000.

Storage: Do not store NexTemp over 75°F (24°C); refrigerate for maximum shelf-life. Allow material to warm to room temperature before use. Do not freeze.

Shelf life: 2 years when stored at room temperature 75°F (24°C)

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