Premier® Implant Cement™

Non-Eugenol, Resin Cement for Implant Retained Crowns

Directions For Use

FR	Mode d'emploi	FI	Käyttöohjeet	NL	Gebruiksaanwijzing
ES	Indicaciones de uso	GR	Οδηγίες χρήσης	KR	사용설명서
DE	Gebruuchsanweisung	HU	Használati útmutató	NO	Retningslinjer for bruk
IT	Istruzioni per l'uso	LT	Naudojimo nurodymai	DK	Brugsanvisning
TR	Kullanım Talimatları	PL	Sposób użycia	RU	Инструкции по применению
CZ	Pokyny k použití	SE	Bruksanvisning	CN	使用说明

US / UK

Premier® Implant Cement™

Non-Eugenol, Resin Cement for Implant Retained Crowns

Indications for Use: Premier® Implant Cement is a non-eugenol temporary cement for luting implant-retained crowns.

Note: Premier Implant Cement is suitable for temporary cementation of provisional restorations where long-term or increased retention is desired.

Chemical-Cure Physical Properties:
Working time: at least 1.50 minutes at room temperature
Gel phase: 2 to 2.50 minutes at 98.6° F (37° C)
Final set: 4 to 5 minutes at 98.6° F (37° C) from start of mix.

Precautions:
1. When using Premier Implant Cement as provisional cement for a temporary crown over a freshly placed composite resin core, Premier recommends lightly lubricating the core material, avoiding the restoration margins, with a thin-film of water-based lubricant to prevent sticking. This step is not absolutely indicated, but certain brands of composite core may have unpolymorized regions for several hours after curing, which could cause adhesion to Premier Implant Cement.

2. Eugenol-containing temporary cements could inhibit the polymerization of Premier Implant Cement. No residue should remain on abutments prior to using Premier Implant Cement.

3. Caution- U.S. Law restricts this device for sale by or on order of a dental professional. 4. Disposable tips are designed for single patient use only. 5. Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

Directions for Use:
1. Check fit, occlusion and proximal contacts of the restoration. Complete all finishing procedures. 2. Dry and isolate the prepared abutments. Absolute dryness is not necessary, but obvious contamination should be avoided.

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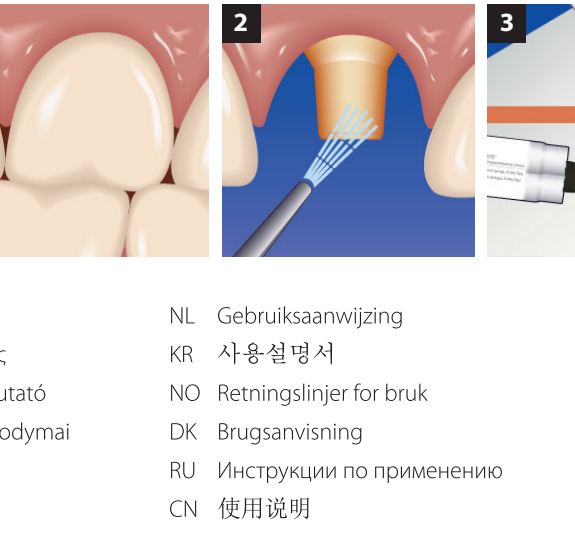
3. Bleed the syringe before installation of the automix dispensing tip. Mount a new, automix dispensing tip onto the Premier® Implant Cement dual-barrel syringe and bleed the syringe again. This assures complete flow of base and catalyst components.

4a. For firm retention, only a thin layer of Premier® Implant Cement should be applied to the cervical third of the crown (or up to the cervical half depending on the amount of retention desired). Dispense a small bead of Premier® Implant Cement onto a microbrush. Use the microbrush to place a thin film of cement along the inside wall of the cervical third of the restoration, making sure to completely coat the margin with cement. Discard excess cement. You have at least 1.5 minutes of working time at room temperature.

4b. Note: Excess cement should be avoided. Applying too much cement can cause future retrieval of the restoration to become difficult or impossible.

5. Firmly seat the crown and either hold in place or have the patient gently close on a cotton roll until the initial set which occurs in 2 – 2.50 minutes. The initial set is characterized by a distinct gel phase. Safely remove excess cement without scratching the implant or abutment after 2.50 minutes, but before final set. Note: Safe removal of excess cement can be achieved by using an implant friendly instrument such as a Premier® Implant Scaler.

6. Final cement set is between 4 – 5 minutes, depending on conditions.

Contraindications:
Do not use on patients with known sensitivity or allergy to resins. Wash exposed area completely to remove material. If an allergic reaction is observed, consult a physician immediately and discontinue use with this patient.

Storage:
Do not store Premier Implant Cement over 75°F (24°C). Refrigerate for maximum shelf-life. Do not freeze. Allow material to warm to room temperature before use.

Shelf life:
24 months from date of manufacture. Refrigerate for maximum shelf-life.

To obtain an SDS, visit Premier Dental at premierdentalco.com or call 610-239-6000.

REF | Ordering Information:
3001450 – 5ml automix syringe, 10 automix tips, mixing pad
3001455 – Three 5ml Automix syringes and 25 automix tips
3001451 – 25 automix tips

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