

Tear Here To Open

premier[®]

Pro Clean[™]

Professional Tartar and Stain Remover
for Dentures, Partials, Retainers
and Night Guards

Fresh Mint Scent

REF 9011103 17g/packet - Box of 25

Made in the U.S.A.

In-Office Directions:

1. Add contents of packet into a beaker or resealable plastic bag containing 16 oz. of very warm (100-125°F/38-52°C) water.
2. Immerse beaker or sealed plastic bag in ultrasonic bath for 1-5 minutes, depending on the severity of the stain.
3. **Important: Rinse thoroughly** with tap water after cleaning to remove residual cleaner from appliance.

Use one packet per patient.

At-Home Directions:

1. Add contents of packet into a container with 16 oz. of very warm (100-125°F/38-52°C) water.
2. Allow appliance to soak for at least two hours.
3. **Important:** Rinse thoroughly with tap water after cleaning to remove residual cleaner from appliance.

Clean dental appliances weekly or as needed.

Caution: Contains sulfamic acid – may be harmful if swallowed. Moderate eye & skin irritant. Avoid eye and prolonged skin contact.

First Aid:

EYES: Flush with large volumes of water for 15 minutes. If irritation persists, seek medical attention.

SKIN: Flush with large volumes of water.

INGESTION: Drink large volumes of water. Contact physician promptly.

KEEP OUT OF REACH OF CHILDREN.



Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462
www.premusa.com

For SDS visit www.premusa.com or call Premier at 610-239-6000.



MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany



0218009 Rev6 DCP