## Perfecta® REV! The 15-Minute, Once-A-Day Tooth-Whitening Treatment

US Doctor's Instructions for Professional Tooth Whitening

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Manufacturer: Premier® Dental Products Company  
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.  
(888) 670-6100 • (610) 239-6000 • premierdentalco.com • Made in U.S.A.

DFU# 40201

0519050 Rev9 NKP15M

US

### Perfecta® REV!®

The 15-Minute, Once-A-Day Tooth-Whitening Treatment

#### Doctor Instructions for Professional Tooth Whitening. Patient must be given patient instructions, gel and finishing rinse.

Use these instructions to administer a complete dentist-supervised tooth-whitening treatment. Perfecta gels provide a more conservative approach for treating non-vital and vital teeth vis-à-vis crowns or veneers.

**Distributed by:**  
Premier® Dental Products Company  
1710 Romano Drive – Box 4500  
Plymouth Meeting, PA 19462 USA (888) 670-6100  
(610) 239-6000 • Fax (888) 627-6160 • premierdentalco.com

**Indications and Usage:**  
Perfecta® REV!® professional tooth-whitening gel and finishing rinse are indicated for the whitening of non-vital and vital teeth. This includes discoloration from aging, food beverages, tobacco, fluorosis, trauma, tetracycline and other medications. Perfecta REV! Gel and Finishing Rinse must be used together to ensure optimal results without attendant sensitivity.

Perfecta REV! Finishing Rinse is used by the patient immediately after every daily 15-minute gel treatment is completed. The rinse is designed to:

1. Potentiate the whitening effect of the Perfecta REV! Gel.
2. Avoid sensitivity by removing any residual gel from the teeth and soft tissue after treatment.
3. Serve as a mouth wash with a pleasant cool mint flavor that will leave your patient's mouth with a clean, refreshed feeling.

Perfecta REV! Gel should be dispensed only into custom trays fabricated for the patient by a dental professional. **The use of this product is recommended only under the direct supervision of a licensed dentist.**

**Contraindications:** This product is not recommended for pregnant or lactating women. While no known hazard is associated with tooth whitening, no specific clinical studies have been conducted with pregnant or lactating women.

**Precautions:** Patients should avoid use of tobacco during tray-wear. Also, to minimize risk of thermal hypersensitivity, avoid hot or cold beverages for 30 minutes after tray removal.

**Adverse Reactions:** Prolonged gel contact with soft tissue may cause gingival irritation; this includes inflammation, redness, soreness or blanching. Occasionally, patients may experience transient thermal hypersensitivity. Normally, this is alleviated by discontinuing tray-wear or reducing wear-time.

**Active Ingredient:** Hydrogen peroxide in a predominant water base.

**Storage:** Refrigeration is recommended for optimal shelf-life and product stability. Do not freeze.

**Instructions for Perfecta® REV!® Professional Tooth Whitening Gel**

#### Non-vital Teeth

**It is recommended that the use of this product for the internal bleaching of non-vital teeth be performed under the direct supervision of the licensed dentist.**

**Instructions to whiten non-vital, endodontically treated teeth:**

Internal/External Procedure  
(This procedure is recommended for treating non-vital anterior teeth after the successful completion of root-canal therapy.)

1. Open lingual access of tooth by removing all existing restorative material.
2. Remove up to 2.0mm of root-filling material below the pulp chamber.
3. Insert a small quantity (1.5mm-2.0mm) of glass ionomer on top of the remaining root-filling to form a protective barrier. Cure per manufacturer's instructions.
4. Using a microtip syringe filled with clean tap water, irrigate pulp chamber.
5. Insert a small amount of gel into exposed root and pulp chamber.
6. Load custom tray with whitening gel and insert into patient's mouth.
7. Patient wears loaded tray for the entire prescribed treatment time (15 minutes for REV!).
8. Be sure to instruct patient of wear-time and length of treatment. (Multiple treatment sessions may be necessary to achieve optimal whitening.)

#### 9. Reschedule patient in one week to review progress.

### For Vital Teeth

#### First Appointment (Week 1 Schedule)

1. Prior to any whitening treatment, perform a complete exam of the patient's teeth and soft tissue; perio status; pregnancy status; health history (including any prior tooth-whitening treatment.) Failed restorations, caries or exposed root surfaces may cause patient to experience mild to severe sensitivity. These concerns should be restored or remedied before commencement of the treatment.

2. Record and file: x-rays, tooth shades, shade anomalies, perio-probing results, and "before" treatment photos. Compare records after treatment is completed. Any areas of concern should be restored before commencement of treatment.

3. Chart defective fillings for replacement post-whitening. Note isolated anterior crowns or bonding which may not bleach.

4. Have patient read and sign informed consent form.

5. Take upper and lower impressions for models and construct whitening trays. Block-out on model is optional, not required. Save models for lost tray replacement.

6. Trim tray on model with heated lab knife, or trim the model with C&B scissors. Tray border should end short of tissue. Tray should have no tissue contact.

7. Dental professional should try whitening trays in patient's mouth and check for extensions and smoothness. Shorten distal of the tray for comfort if the patient will not benefit from molar whitening.

8. Give the patient a one-week supply of gel (one syringe) and patient instructions. Demonstrate how to place a small (1/8"/0.3cm diameter) bead of gel in the tray. Reinsert loaded tray in patient's mouth. Have patient complete first treatment in the office.

9. Recommended wear schedule. (Start upper arch first.) 15 minutes once a day for 2 weeks.

10. After each 15-minute treatment, have patient dispense 1-2 capfuls (5ml) of Finishing Rinse in a clean cup.

11. Patient should vigorously swish Finishing Rinse between teeth for 20 seconds and expectorate. Do not swallow.

**Reappoint in 1 week. Monitor weekly until whitening is complete.**

12. Patient should avoid food, drink and tobacco for at least 30 minutes after tray wear. Transient thermal sensitivity may arise during the first week. To treat, have the patient wear trays every other day. If sensitivity does not minimize with reduced wear, ibuprofen, aspirin, or potassium nitrate toothpaste worn in the whitening tray may relieve symptoms.

#### Second Appointment (Week 2 Schedule)

13. Examine soft tissue and adjust tray(s) if tissue is irritated.

14. Note transient thermal sensitivity, TMJ soreness, or occlusion changes. These symptoms are normally mild and transient. (See #12 above.)

15. Using a value modified shade guide, advise patient of shade change. Record shade changes by tooth. Take this opportunity to compliment and motivate patient.

16. Lower arch: Begin tray wear using schedule in steps #9-11.

17. Reappoint patient in one week for short appointment to record final tooth shades and review other potential dental restorations.

#### Final Whitening Appointment

18. Record final tooth shades and other relevant data.

19. Reiterate that relapse is possible, especially with caffeine or tobacco use. A whitening treatment or touch-up may be needed every 2 to 3 years.

20. Recommend routine use of a whitening toothpaste frequently enough to prevent relapse. As stated in the consent form, relapse is variable and prevention is the patient's responsibility.

21. Maximum stability normally requires 2 weeks of whitening treatment on each arch. Tetracycline stains frequently require additional extended treatment.

#### If necessary for difficult stains:

22. If extra whitening is necessary to obtain desired shade, consider using additional gel and extending wear schedule accordingly.

#### Recommended Usage Per Treatment:

Complete 2-arch (14-day) Treatment-  
Two 3cc syringes; 4 oz bottle of Finishing Rinse

Touch-Up 2-arch (7-day) Treatment –  
One 3cc syringe; 4 oz bottle of Finishing Rinse

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