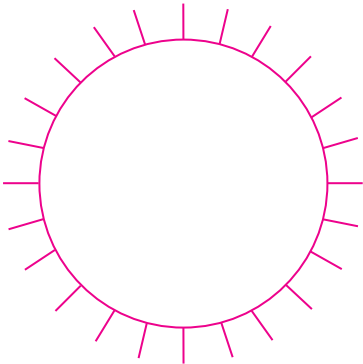
premier

Removalon®-I DRY PAK

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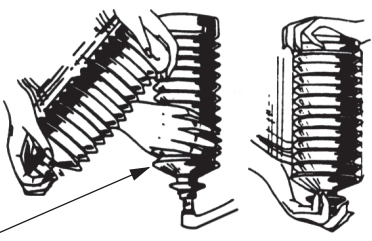
Instructions for use:

1. Pop open cap.

2. Hold bottle up-right and away from face to avoid potential dust inhalation. Pull up to height.

3. Unscrew cap and fill with warm tap water to here.

4. Shake well (15 to 20 seconds).

Without Ultrasonic Machine: Pour Removalon®-I into a metal, glass or plastic container to a depth which will cover the areas of the instruments to be cleaned. Place instruments, etc. into Removalon-I and allow to soak. The length of time required for dissolution depends on the degree of setting and the thickness of the cements. Average time is 15 minutes. *Prolonged immersion will cause surface discoloration. Remove after 15 minutes.*

With Ultrasonic Machine: Fill the beaker or bath receptacle with Removalon-I. Immerse those instruments to be cleaned, certain that areas containing cement are covered with Removalon-I. Average time for cement dissolution is 5 minutes. After removal from skin, wash thoroughly with water. Removalon-I can be reused, the used solution should be discarded when a sufficient amount of debris has accumulated.

Read SDS for important safety information before using product.
To obtain an SDS, visit Premier® Dental at www.premusa.com or call 610-239-6000.

premier

Premier® Dental Products Company

1710 Romano Drive • Plymouth Meeting, PA 19462 U.S.A.
888-670-6100 • 610-239-6000
www.premusa.com • dentalinfo@premusa.com

CE

EC REP

101543 Rev5 DCP

Warning

premier


Removalon®-I DRY PAK

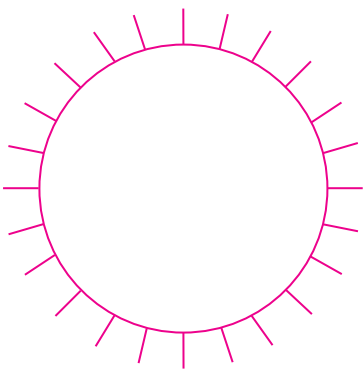Water Activated Instrument Cleaner

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