Traxodent®

FR Système de rétraction en pâte Hemodent®
ES Sistema de retracción de pasta Hemodent®
DE Hemodent® Pasten-Retraktionssystem
IT Sistema di retrazione in pasta Hemodent®
TR Hemodent® Macun Retraksiyon Sistemi
CZ Retrakční systém s pastou Hemodent®
FI Hemodent®-tahna retraktiojärjestelmä
GR Σύστημα σύμπτυξης οδοντιατρικής πάστας Hemodent®
HU Hemodent® krémvisszahúzó rendszer
LT Hemodent® retrakinės pastos sistema
PL Pasta Hemodent® do systemu retrakcyjnego
PT Sistema de retracção com pasta Hemodent®
SE Hemodent® Pastaretraktionssystem
NL Hemodent® pasta-retractiesysteem
KR Hemodent® 페이스트링 치은압배 시스템
DK Hemodent® pastaoptrækningsystem
IL HEMODENT משרת משהח לדחיקת תכנים
RU Система ретракции на основе пасты Hemodent®

US / UK

Traxodent®

Hemodent® Paste Retraction System

Product Description: Traxodent® Hemodent® Paste Retraction System is a soft paste containing aluminum chloride to provide gingival retraction and hemostasis. The product is typically dispensed thru a syringe and a single-use dispensing tip.

Indications: Traxodent® has been specifically formulated to provide gingival retraction and hemostasis. It is intended for use prior to taking an impression, cementation, cavity preparation or wherever hemostasis and retraction is required.

Directions for Use:

1) Remove the cap from the syringe. Be sure to retain cap for subsequent storage of the material.

2) Place annealed luer dispensing canula onto end of the syringe tip, twist-lock into place. The soft metal of the annealed canula allows bending of dispensing tip for optimal access. A mirror handle can assist in bending tip to desired angle.

3) Rinse and air dry prepared tooth. Extrude Traxo-dent paste slowly into the sulcus while maintaining the dispensing tip just above the sulcus and aligned approximately parallel to the axial plane of the tooth preparation. Be careful not to jam dispensing tip into sulcus.

4) Ensure you extrude sufficient material into the sulcus to achieve adequate tissue retraction.

Note: For subgingival margins and maximum deflection of soft tissue, use a Premier Retraction Cap after dispensing Traxodent. Press Retraction Cap onto prepared tooth. Have the patient bite down or apply pressure to cap if patient lacks opposing dentition. Optionally, a single strand of retraction cord can be placed prior to dispensing Traxodent.

5) Allow Traxodent paste to remain in the sulcus for at least one to two minutes. Often, the marginal gingiva will grow pale when retraction is achieved.

6) After hemostasis has been achieved, the Traxodent should be removed by thoroughly rinsing using air-water syringe and saliva ejector or suction tip.

7) Examine treatment site to verify complete removal of the Traxodent prior to taking the impression.

Remove canula dispensing tip and dispose of properly after use. Re-cap Traxodent syringe for storage in seal-able foil bag.

Contraindications:

- Do not use on persons with known allergy to aluminum chloride
- Do not use in patients with significant periodontal disease or furcation involvement.

WARNING:

- Do not use on persons with known allergy to aluminum chloride

Precautions:

- The syringe's purpose is Traxodent primary packaging and it is not to be cleaned and reused for any other purpose.
- Disposable tips are designed for single patient use only.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients and staff.
- Do not insert canula dispensing tip into the sulcus.
- Be sure all paste is removed before taking impression to avoid inhibition of polymerization of impression material.

Caution-U.S. Law restricts this device to sale by or on order of a dental professional.

Storage:

- Store at room temperature <28°C / 82°F.
- Do not freeze.

To obtain an SDS, visit Premier Dental at premierdentalco.com or call 610-239-6000.

REF	9007091 Traxodent® Value Pack
	25 syringes and 50 bendable dispensing tips

REF	9007093 Traxodent® Starter Pack
	7 syringes and 15 bendable dispensing tips

REF	9007092 Dispensing Tips
	50 bendable dispensing tips

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