PL

Vac Attak™

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US / UK

Vac Attak™

High Performance Evacuation System Cleaner

Cleans and Deodorizes • Safe • Non-Corrosive • Environmentally friendly • Non-Foaming • Organically Chlorinated

Concentrated - Produces 100 Evacuation Cleanings - Fresh Lemon Scent

Directions for Use:
1. Fill Evacuation sink or other container with 32oz. (946 ml) of warm tap water.
2. Remove one scoop (8gm) of cleaning powder from container.
3. Sprinkle powder into water and allow to thoroughly dissolve (30-45 seconds).
4. Flush system with solution.
5. Flush system again with tap water after cleaning.

Attention:
Upon initial use, the powerful cleaning action of Vac Attak may cause accumulated debris to dislodge. Be sure to check the filter/trap after use for maximum efficiency.

Highly effective at removing all types of organic debris.

Use fresh solution daily.

Caution: May be harmful if swallowed. Eye and skin irritant. Avoid eye and prolonged skin contact.

First Aid:
Eyes: Flush with large volumes of water for 15 minutes. If irritation persists, seek medical attention.
Skin: Flush with large volumes of water.
Ingestion: Drink large volumes of water. Contact physician promptly.
Keep out of reach of children.

Premier® Dental Products Company
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.
(888) 670-6100 • (610) 239-6000 • www.premusa.com

Made in U.S.A. Read SDS before using product. For SDS, visit www.premusa.com or call Premier at 610-239-6000. DPU# 90117 0917088 Rev4 DCP