**Compatible with all amalgam separators!**

**premier®**

# Vacu Blast™

## Evacuation System Cleaner

---

For best results, use daily in each operatory to keep your evacuation system operating at peak performance.

- Powerful enzymatic formula
- Anti-corrosive
- Non-foaming
- Neutral pH
- Environmentally friendly
- Phosphate- and chlorine-free

**Refreshingly Clean  
Eucalyptus Breeze!**

**REF** 9011107

Net Contents: 32 fluid oz (946 ml)

Super concentrated formula

Mix ½ oz (15 ml) with 32 oz (946 ml) of water


Yields 64 treatments

0418094 Rev3 DC

Made in the USA

## US/UK Evacuation System Cleaner

### Directions for Use

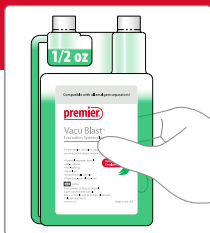Loosen cap on metered-dose chamber (do not remove). Squeeze from middle of bottle until liquid reaches  $\frac{1}{2}$  oz. mark.

Remove cap on metered-dose chamber and pour contents into 32 oz. of water. Stir to mix and use. For large offices, fill to 1 oz. mark and dilute with 64 oz. of water. Can be used with or without an atomizer.

***Aspirate the cleaning solution through your evacuation system lines daily.***

**PRECAUTION:** Avoid over dilution.

**WARNINGS:** Keep out of reach of children  
Harmful if swallowed  
May cause skin irritation, eye damage and allergic skin reactions.

**FIRST AID:** If swallowed, do not induce vomiting and call the poison control center or a physician immediately. In case of contact with eyes, flush with water for at least 20 minutes. In case of contact with skin, wash with water and soap and rinse thoroughly. See SDS for complete safety information.

To obtain an SDS sheet, visit [www.premusa.com](http://www.premusa.com) or call Premier® Customer Service at 888-670-6100 or 610-239-6000.

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EC REP MD55 GmbH