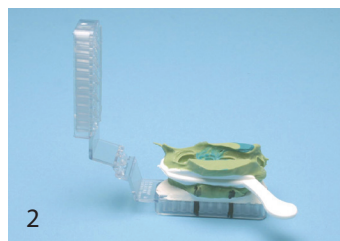
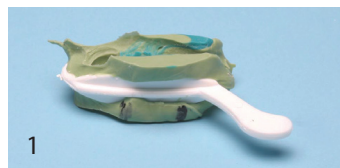
Directions for Use

The W.O.W.™ articulators can be used on all types of cases, dual-arch (double-bite), pre-poured and conventional impressions.

Dual-Arch Trays (Double-Bite)

The case is self-articulated as it is never taken out of the impression until it is poured and mounted.

Counter Impression

- 1) Remove any excess impression material from counter and master impression so that the impression tray can lay flat with the plane of the occlusion, parallel with the counter top.
- 2) Mark the centerline of the preparation on the impression to determine the best location of dies in relation to the articulator. (Figure 1)
- 3) Place one or two dowel pins in the counter side of the articulator. Position the impression tray on the counter top and pour a sufficient amount of the die stone to fill the impression completely.
- 4) Add additional die stone to the articulator so that the dowel pins and anti-rotational ridges are covered. Invert the articulator onto the tray and allow stone to set. (Figure 2)

- a) Make sure that there is enough stone between the impression and articulator to prevent over-closure. Not using enough stone can cause springiness in the finished case.

Master Impression

- 5) Select holes for the dowel pins for the prepared tooth or teeth. Place TWO dowel pins for each prep into these holes. Add pins for the non-prepped adjacent teeth. (Figure 3)

- a) MAKE SURE THAT ALL THE PINS ARE TOTALLY AND COMPLETELY INSERTED ALL THE WAY INTO THE ARTICULATOR.

A simple way to insure this is to turn the articulator over and press the pins against the counter top.

- 6) Add stone to cover the pins, pour the impression and invert the articulator over the impression.

- a) Make sure that there is enough stone between the impression and articulator to prevent springiness in the finished case. (Figure 4)

- 7) Clean up the model before stone reaches its final set.

- a) Use a lab knife to quickly and easily shape the model for a neat and attractive product. The vertical walls of the articulator act as a perfect guide for the lab knife.

- 8) After the stone has set, remove the impression tray and section the dies. (Figures 5 and 6)

Remounting the case

If the assistant or doctor has missed the bite, you can remount in less than a minute.

- 1) Use a micro torch and heat the area between the hinge and first bend until plastic softens.

- a) Be careful not to overheat or warp the hinge.

- 2) Re-establish the bite by hand. Cool with a blast of compressed air or run under cold water.

Pre-poured models

- 1) Trim the models so that they are compact and fit within the horizontal and vertical dimensions of the articulator.


- 2) Add dowel pins to the articulator and mount the opposing side model.

- 3) Sticky wax the working model to the opposing model, add dowel pins and mount the working side.

- a) Clean the model before stone sets.

Item#	Product
0069	W.O.W.™ Intro Kit
0065	W.O.W. Full Arch
0065AD	W.O.W.™2 Full Arch
0072	W.O.W. Posterior
0072AD	W.O.W. 2 Posterior
0067	Dowel pins (300)

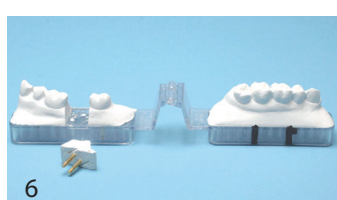Each package comes with 50 articulators and 300 dowel pins. The Intro Kit comes complete with 20 full arch, 30 posterior W.O.W. articulators and 300 dowel pins. The W.O.W.2 has the added feature of Micro-thin Flash Air Dam™ technology. This feature keeps unwanted debris from lodging in pin holes, providing a clean and professional case.

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WOW!

Self-articulated and precise models in two fast pours!

premier

W.O.W.™ & W.O.W.™2

Accurate Dies and Models

Save money and time!

□ No need to track and clean metal articulators

□ No accelerator or glue required

□ Minimizes grinding and trimming

Accurate dies and models!

□ Clear plastic view allows for accurate pin placement

□ Pin placement design eliminates rotation

□ Shallow retention grooves minimize dimensional change from stone expansion

□ Fixed hinge allows for positive occlusal relationship between upper and lower models and appropriate lateral movement

□ Remount bite, if necessary, with heat in less than a minute

premier

Premier Dental Products Company

1710 Romano Drive, Plymouth Meeting, PA 19462, USA
Phone 888-670-61001 • 610-239-6000 • Fax 610-239-6171
www.premusa.com • E-mail: dentalinfo@premusa.com

Made in the U.S.A.