ZR-Cem™

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US / UK

ZR-Cem™

Self-Adhesive Universal Resin Cement

Product Description

ZR-Cem is a self-etch, dual cure, adhesive resin cement that is specially formulated for zirconia cementation; however, it is also indicated for definitive cementation of all ceramic, composite and metal restorations.

Examples of Substrates

All Ceramic Materials	CAD/CAM milled	Metal-based
• Zirconia (low and high translucency) • Alumina • Lithium Disilicate • Zirconia-reinforced Lithium Silicate • Feldspathic and Leucite-reinforced Ceramics	• Composites • Ceramics	• Crown and Bridge Alloys • PFMs

Dual-Cure Physical Properties:

Light-cure setting times:

20 seconds with light intensities of = or > 500 mW/cm²

30 seconds with light intensities of < 500 mW/cm²

Chemical-cure

72°F (22°C) temperature

Working time: 15 – 25 minutes. Higher temperatures accelerate set.

98.6° F (37° C) temperature

Gel phase: approximately 1-15 minutes

Final set: 3.0 – 4.0 minutes. Higher temperatures accelerate set.

Indications for Use:

ZR-Cem adhesive resin cement is a prescription only, dual cured, radiopaque resin cement intended to be used for cementation of porcelain, ceramic, resin, metal-based inlays/onlays, crowns, bridges, core build up material, posts and veneers. It is also used for cementation of crown restorations to implants.

Precaution:

- Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.
- Automix tips are designed for single patient use only.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

Contraindications:

Do not use on patients with known sensitivity or allergy to methacrylate resins. Wash exposed area completely to remove material. If allergic reaction is observed, consult a physician immediately and discontinue use with this patient.

Note:

Premier Universal Primer can be used to help achieve maximum bond strength for cases that require additional retention (e.g., short and/or over-tapered preparations).

Instructions for use:

1. Pretreatment of Tooth

- 1.1 Remove the provisional restoration and all temporary cement residue. *Note: Use only temporary cements that are eugenol-free. The use of eugenol-containing temporary luting materials can inhibit the polymerization process of ZR-Cem.*
- 1.2 Mechanically clean prepared tooth (e.g. with pumice paste).
- 1.3 Rinse preparation thoroughly and air dry. Do not desiccate - over drying can lead to post-operative sensitivity.

- *Tips: Tooth preparation should leave the tooth surface evenly moist. Dry preparation (air-dried or alcohol dried), and wet preparations (with pooled surface water), can reduce adhesion.*
- *The adjacent teeth and/or the external surfaces of the restoration may be lubricated with a water soluble medium to ease cleanup of excess cement.*

- 1.4 Try in the restoration to ensure proper fit.

2. Prepare Restoration

- 2.1 Verify that the laboratory has pre-treated the final restoration in accordance with the manufacturer's instructions.
- 2.2 Clean intaglio surface of restoration with alcohol (or per restoration manufacturer's instructions), then dry well with oil-free air.

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