
US/UK

### Triple Tray® Technique

After Completion of Tooth Preparations and Gingival Retraction

1. Place unloaded TRIPLE TRAY® into proper position. When using the posterior, quadrant or sideless TRIPLE TRAY, make sure the crossbar of the tray is distal to the last maxillary tooth and the tray is centered buccally/lingually.
2. Have the patient "close" on the unloaded tray and check the opposite side of the patient's mouth to make sure that the patient has closed into the correct occlusion.
3. Paint impression adhesive on tray walls only, not on webbing. Overload both sides of TRIPLE TRAY.
- Note:** Triple Tray must be used with a firm setting material. It also may be used with hard-setting Heavy Body/Light Body one-step techniques, not two-step techniques.
4. Remove retraction cord, if used. Syringe impression material around important cervical, interproximal and edentulous areas.
5. With syringed impression material in place and still unset, place loaded Triple Tray into proper position. (Make sure crossbar is correctly placed.)
6. Have patient close properly.
7. When set, gently remove from mouth. Inspect final impression. No white plastic pressure areas should be visible.
8. Disinfect.

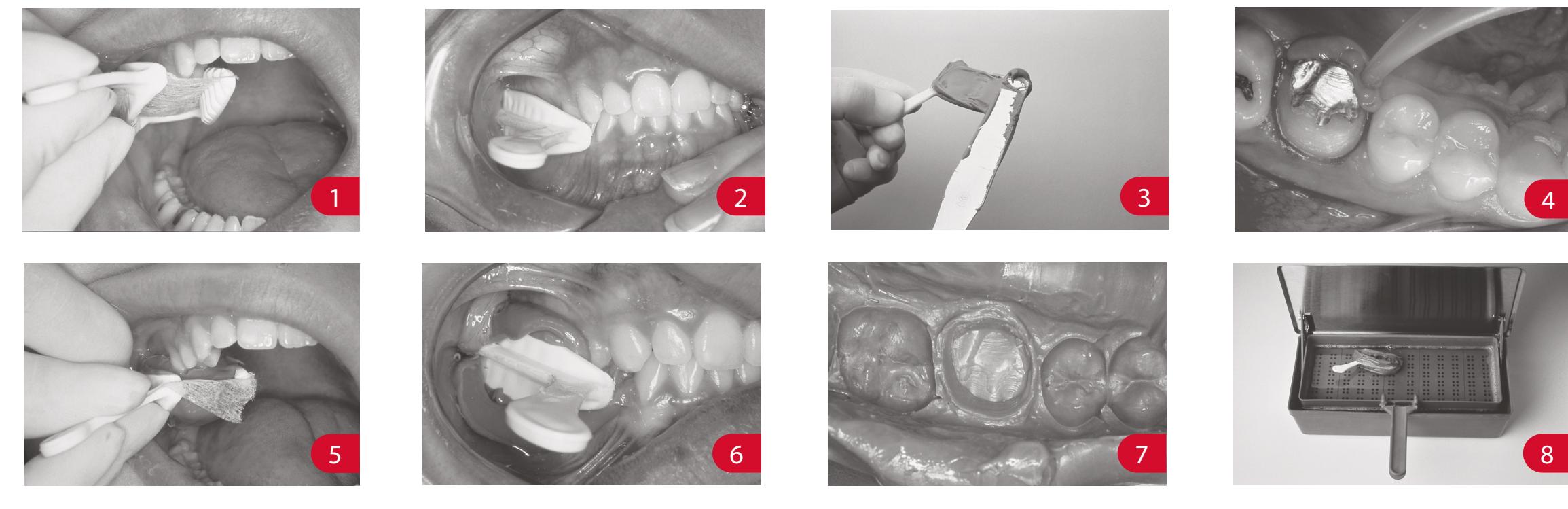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# Triple Tray®

**US/UK 3 Procedures in One**  
Master Impression • Counter Impression • Bite Registration  
Saves Chairtime!

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**US/UK - Laboratory technique for pouring Triple Trays**

Evaluate impression for any visible white plastic pressure areas which may affect the accuracy of the impression.

- Trim excess impression material from the borders of the impression. **Helpful hint:** Before pouring, mark the handle of the upper arch. This will eliminate articulating the model upside down.
- Box impression containing die with wax and position dowel pin. Seal off any holes that stone may leak through. First, the working side is poured, adding stone or epoxy in small increments. Next, the counterside is poured. Allow to set. Pour base and allow to set. Remove boxing wax.
- Articulate model and let plaster set.
- Separate casts by pulling apart carefully.
- Continue with normal crown and bridge procedures.

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## US/UK - Increase your Accuracy with Triple Tray®

- Thin mesh webbing - To assure a close, accurate bite
- Contoured to the arch of the mouth
- Finger Purchase for a firm grip and easy removal

Item #	Description	Quantity
1006203	Posterior	Box of 48
1006213	Anterior	Box of 35
1006223	Quadrant	Box of 40
1006233	Assorted	Box of 35
1006243	Sideless Posterior	Box of 48
1006253	Extended Quadrant	Box of 35
1006273	Three-Quarter Arch	Box of 35
1006283	Wide Body Posterior	Box of 48

**Intended Use:** This product is used to take a dental impression with the closed-arch technique. Triple Trays also can be used for bite registration.

**Storage Conditions:** No special storage conditions are required.

Inspired solutions  
for daily dentistry®

**Manufacturer:**  
Premier® Dental Products Company  
1710 Romano Drive, Plymouth Meeting, PA 19462 U.S.A.  
(888) 670-6100 • (610) 239-6000 • premierdentalco.com

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